Saljol Rückengurt

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Saljol back belt

for Saljol Carbon Rollator and Allround Rollator

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Article number: 4058789013595

Product overview

- 1 Reflective stripes
- 2 Recess for better visibility of the floor in front of you
- 3 Breathable mesh lining
- 4 Fastening rail
- 5 Screws for length adjustment

Mounting

Insert the two fastening rails of the back strap into the holders on the back of the handles of your Saljol Carbon/Allround Rollator.

When fitting the back belt, always make sure that the reflective strip is on the outside.

Caution!

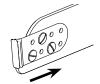
- Make sure that the back strap is fully inserted into the holder at all times. Serious injuries may otherwise occur or damage may be caused to the rollator.
- The maximum user weight must not exceed 120 kg.

Length adjustment of the back strap

The back strap is delivered as standard with the longest possible adjustment. If you would like the strap to be shorter, adjust the fastening rail on one or both sides. To do this, take the strap out of the rollator's holders and place it in front of you. You will need two slotted screwdrivers for this.

 Loosen all three screws with a screwdriver. Always hold the second screwdriver against it from the other side. Remove the screws and put them aside.

Move the fastening rail on the back strap inwards, screw all three screws back into the holes provided in the strap and tighten.

Additional adjustment of the back belt

If the handles of your rollator are set very high, the height of the back strap may therefore not provide a comfortable sitting feeling. In this case, the back strap can also be fitted with the opening recess facing downwards. To do this, take the strap out of the rollator's holders and place it in front of you. You will need two slotted screwdrivers for this. Make sure that the fastening rail is attached to the inside of the belt.

 Loosen all three screws with a screwdriver. Always hold the second screwdriver against it from the other side. Remove the screws and fastening rail and put them aside.

 Turn the belt 180 degrees, reposition the fastening rail on the inside of the belt to the correct length for you and screw all three screws back into the holes provided in the strap and tighten.

Information correct at time of printing. Technical specifications are subject to change without notice. Technical drawings and photos may differ from reality. ©2023 Saljol GmbH: all rights reserved. Reprints, including excerpts, are only permitted with the permission of Saljol GmbH.

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Manufacturer: Saljol GmbH