Saljol

Duschhocker | Shower Stool

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Instruction manual

Preface

Thank you for purchasing a Saljol Shower Stool! With your new shower stool, you will be able to shower safely, independently and comfortably. The shower stool has been developed with a lot of attention to safety and will support you in your everyday life.

Please read the instruction manual thoroughly before using your shower stool for the first time. It contains important safety information, as well as useful tips for use and care.

Content

1.	Identification.....	17
2.	General notes.....	18
3.	Notes.....	19
3.1	Safety instructions.....	19
3.2	Instructions for the safe use of the product.....	19
4.	Scope of delivery and product overview.....	20
4.1	Scope of delivery.....	20
4.2	Marking/Label.....	21
4.3	Product overview.....	22
5.	Operation.....	22
5.1	Inventory.....	22
5.2	Shower stool placement.....	23
6.	Maintenance.....	24
7.	Cleaning.....	24
8.	Storage and shipping.....	25
9.	Disposal.....	25
10.	Technical data.....	25
11.	Warranty and guarantee.....	26
11.1	Warranty.....	26
11.2	Guarantee.....	26
12.	Note on the obligation to report incidents.....	27
13.	Test protocol for reuse.....	27

1. Identification

Manufacturer:

Product:

Type:

Article name:

Version:

Saljol GmbH

Saljol Shower Stool

Shower Stool

Saljol Shower Stool

2022

Manufacturer:

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D-82216 Maisach

Phone: +49 (0)8141 317740

Fax: +49 (0)8141 3177411

Mail: hallo@saljol.de

Web: www.saljol.de

Swiss representative and importeur:

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Formalities of the operating instructions

Version/Revision: 2022.09
Creation date: 12.09.2022

Information is correct at the time of going to press.
We reserve the right to make technical changes. Technical drawings and photos may differ from reality.
©2022 Saljol GmbH: all rights reserved. Reprinting, including excerpts, only with the permission of Saljol GmbH

2. General Notes

Purpose

The shower stool from Saljol was developed for home and stationary areas, mainly for use in shower/washing rooms. For people with limited mobility it serves as a shower aid with four wheels for independent showering/washing and sitting. The shower stool is designed exclusively for indoor use and a max. user load of 150 kg.

Indication

The Saljol Shower Stool is exclusively intended as an aid to showering and for use in the bathroom for persons with the following limitations:

- Unsteadiness and/or inability to stand
- Illnesses that affect the ability to stand
- As a postoperative aid

Any other use is not allowed.

Contraindication

The Saljol Shower Stool is not intended for people with severe balance disorders, weak muscle tone, impaired cognition, inability to sit and a user weight higher than 150 kg. The use and possible applications of the Saljol Shower Stool should be assessed and instructed by a doctor or therapist.

Service life

When used as intended, regular care and maintenance by a specialized company and compliance with the safety instructions, the expected service life of the shower stool is up to 4 years. If after 4 years the shower stool is still in proper condition, it can be used again.

Responsibility

As the manufacturer, we can only consider ourselves responsible for the safety, reliability and usability of the shower stool if changes, expansions, repairs and maintenance work are carried out regularly and only by authorized specialists and if the shower stool is used properly in compliance with all instructions for use.

Declaration of conformity

Saljol, as the manufacturer of the product, declares that the shower stool fully complies with the requirements of Medical Device Regulation (2017/745) (status 2022). A CE Declaration of Conformity can be found on our homepage www.saljol.de.

The manufacturer declares that the following products are approved with Product designation Saljol Shower Stool, Version 2022

Saljol Shower Stool DH51, Velvet Grey	4058789014356
Saljol Shower Stool DH55, Velvet Grey	4058789014363
Saljol Shower Stool DH51, Diamond Silver	4058789014370
Saljol Shower Stool DH55, Diamond Silver	4058789014387
Saljol Shower Stool DHK59, Velvet Grey	4058789014394
Saljol Shower Stool DHK59, Diamond Silver	4058789014400

comply with the following harmonised standards:

ISO 17966:2016-01 Assistive products for personal hygiene that support users – Requirements and test methods

ISO 10993-1/ISO 10993-5 Biological evaluation of medical devices

3. Notes

3.1 Safety instructions

The instruction manual contains safety notes that are marked twice: by a symbol and by a signal word. The symbol informs you about the type of notice. The signal word explains more details, for example the severity of an impending risk.

Caution/Important

Possibly dangerous situation (minor injuries or material damage), commitment to a particular behaviour or activity for the safe use of the shower stool

Note

Application tips and useful information

3.2 Instructions for the safe use of the product

For safe use of the product, please keep the following in mind:

Restrictions on use

- The maximum user weight must not exceed 150 kg.
- The shower stool may only be used indoors and on a flat, solid surface.
- The shower stool must stand securely on all four wheels.
- If there is a shift in balance due to body movements, the risk of tipping can increase.
- Never use in the presence of flammable/combustible and burning substances.

Accident prevention

- Only use the shower stool for the defined purpose.
- Only use the shower stool if it is in perfect technical condition.
- It is prohibited to make any constructive changes to the shower stool, unless you have the written consent of the manufacturer, Saljol.
- All four wheels must be in contact with the ground at all times. Only in this way the shower stool is safely balanced which prevents accidents.
- When moving (shifting balance), make sure that you sit securely on the shower stool.
- Avoid jerky rotational movements on surfaces with tile grouts.
- Bath additives or detergents can alter the floor surface and thus influence the slip resistance.
- In case of slight dizziness while sitting, please hold on to the side handles.

Caution

Toes, feet or other body parts of the user and/or a third party helper could be pinched when turning the shower stool or adjusting the hygiene cut out. Therefore, when using the device, be extremely attentive and careful. Be aware of the wheel movement to protect yourself from pinching your toes and other parts of your body.

4. Scope of delivery and product overview


4.1 Scope of delivery

1x Saljol Shower Stool
1x instruction manual

4.2 Marking/Label

The product label is on the the underside of the frame.

It contains important information about the product: EAN code, product name, production date, product dimensions, maximum user weight, seat height, manufacturer information of Saljol and the serial number. The label must not be removed.

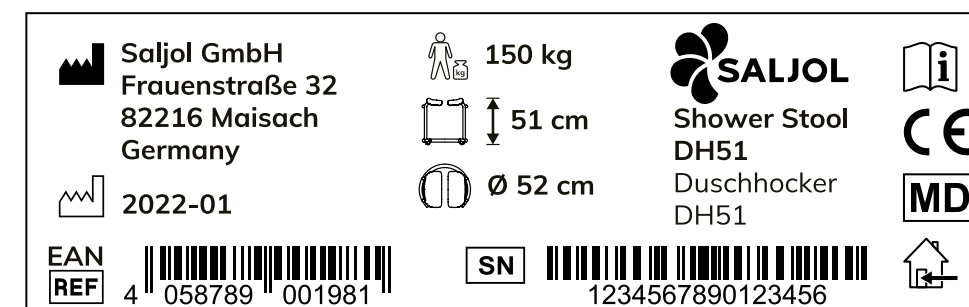The following symbols can be found, partly in combination with further information, on the marking of the shower stool and on the packaging to identify important properties.

Manufacturer of the product

Production date

Order number/item number

Maximum user weight

Seat height

Required space for turning radius

Serial number

Observe instruction manual

CE marking

Medical device

Product suitable for indoors

4.3 Product overview


5. Operation

5.1 Inventory

Take the Saljol Shower Stool out of the box and check if anything is missing. Compare with figure above and with the scope of delivery, described in 4.3 on page 20. If any part is missing or damaged, contact your dealer or Saljol immediately.

5.2 Shower stool placement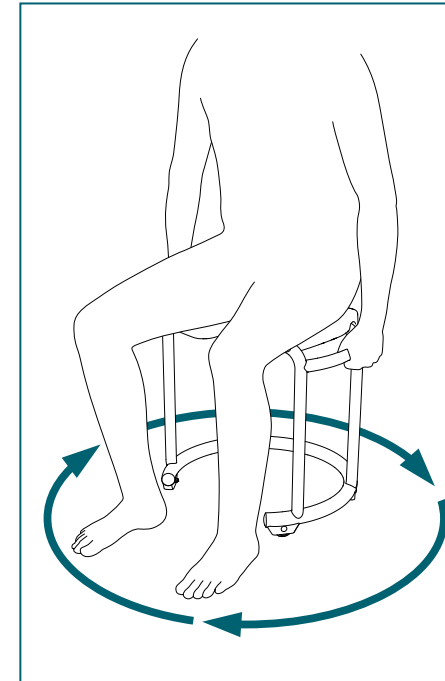

Place the Saljol Shower Stool on a level floor and take a seat for a first seating test. Make sure that you always


- sit upright in the middle of the shower stool
- do not lean forward or backward excessively
- there is always enough room for movement.

Turn from left to right together with the shower stool while keeping both legs on the floor. If you are being cared for, hold on to the handles. The person caring for you can turn you gently. For all-round care.

Turn on the spot

The wheels of the shower stool are arranged in a circle and equipped with a non-slip rubber coating. Therefore, body parts which can only be reached with difficulty can be cleaned easily. You have the choice to place your feet comfortably on the frame or to place them on the floor for a secure feeling.

For the version with wedge seat, the legs must be on the side of the frame opening.

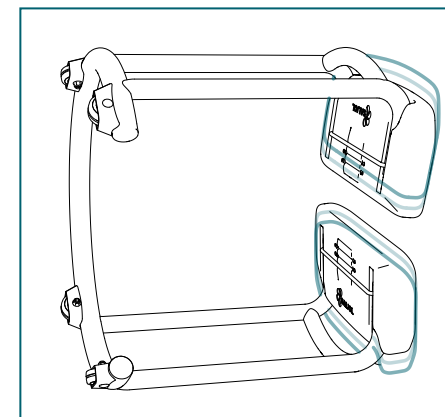Seat width adjustment

The width of the hygiene cut-out can be adapted to your physique without tools. To do this, place the stool on its side as described on the left and move the seats according to your wishes. Place it on a soft underground like a carpet to avoid scratches. The three seat widths 1, 2 and 3 engage perceptibly. Please note that both sides must be set to the same level.

Important

When sitting down and standing up, please make sure that your feet are always in contact with the floor and are turned slightly outwards in front of the lower frame opening.

While rotating, please make sure that you perform the movement slowly with continuous muscle tension.

If you are using the version with wedge seat your legs have to be on the side of the frame opening.

6. Maintenance

The shower stool is maintenance-free. If faults or defects occur during use, the shower stool should be returned immediately to the dealer/Saljol for repair.

7. Cleaning

Frame

Cleaning can be done with warm water and a sponge. In case of stubborn soiling, a commercial cleaning agent can be added to the water. Disinfectants can be used without any problems.

Important

Never use aggressive cleaning agents such as solvents/abrasives or hard brushes.

Wheels

Clean the wheels with warm water and a mild solvent. If necessary, use a plastic brush, never a wire brush.

Seat surface

Cleaning can be done with warm water and a sponge. In case of stubborn soiling, a commercial cleaning agent can be added to the water. Furthermore, the surfaces of the seat should be occasionally disinfected with a commercially available disinfectant. The Robert Koch list of approved disinfectants can be found at <http://www.rki.de/en>. The surfaces of the seat are also dishwasher safe (max. 85° C).

Important

- Never clean the shower stool with a high-pressure/steam cleaner!
- When disinfecting, observe the instructions for use and processing of the respective manufacturer.
- After cleaning, rinse sufficiently with clear water to completely remove any remaining product adhesions (cleaner).
- Personal care product residues such as liquid soaps, shampoos and shower gels can also cause damage. After use, rinse residues carefully with clear and warm water.

8. Storage and shipping

When not in use, keep the shower stool upright, in a dry place, out of reach of children or third parties. If the shower stool is to be stored or shipped, it is advisable to keep the original packaging and store it for these purposes so that it is available when needed. Then your shower stool is optimally protected against environmental influences during storage or transport.

9. Disposal

Dispose materials separately by type in the designated collection systems. Please dispose hazardous waste in the designated collection systems.

10. Technical data

Shower stool	DH51	DH55	DHK59
Recommended user height	1.50-1.70 m	>1.70 m	>1.50 m
Adjustable hygiene cut-out, setting 1/2/3	5/8/11 cm	5/8/11 cm	5/8/11 cm
Total width max. from handle to handle	52 cm	52 cm	52 cm
Seat height	51 cm	55 cm	59 cm (middle)
Wheelbase	32 cm	32 cm	32 cm
Turning circle Ø – Space requirement	48 cm	48 cm	48 cm
Max. user weight	150 kg	150 kg	150 kg
product weight Velvet Grey/Diamond Silver	5.8/4.4 kg	6.0/4.6 kg	6.0/4.7 kg

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Saljol GmbH | Frauenstraße 32 | 82216 Maisach