

# How to find the right size for your Saljol Indoor Rollator

The right size of the everyday helper for your home depends on the situation in which you use the rollator most often. **If you tend to use the indoor rollator in a seated position, your lower leg length is the decisive factor. If you mainly use the rollator to walk and lean on it, your height is relevant.** Below you will find tips on how to determine the right size for your new favourite vehicle.

- Do you need the rollator to save energy and do a lot of things sitting down? Then you will find the right size under „**1. Mainly for sitting**“.
- Do you want to use the rollator mainly as a rollator, i.e. as a replacement for a handrail? Then read „**2. Mainly as a rollator**“ to find out which size is right for you.

**Note:** When using the upholstered seat accessory, please note that the seat height increases by 1.5 cm. The height of the WR52 then increases to 53.5 cm, the WR55 to 56.5 cm and the WR58 to 59.5 cm.



# 1. Mainly for sitting

Do you need the rollator to push yourself around the home in a seated position (tripping)? Then you need the following measurements:

## Your lower leg length (A)


Measure the distance from the floor to the start of the knee bend, preferably with the shoes you normally wear at home. Your legs should be straight when standing and at a right angle when sitting. The entire soles of your feet should touch the floor. Let someone help you with this, it is easier with two people.

Compare your measurement with the following table and you will find the right rollator for you.

Lower leg length (A)	< 48 cm	< 52 cm	> 52 cm	> 52 cm
Your suitable rollator	WR52	WR55	WR58	WRXL

Are your measurements not exactly matched to a rollator size?

Our sales and service team will be happy to advise you on +49 8141 317740 or send an e-mail to [hallo@saljol.de](mailto:hallo@saljol.de).

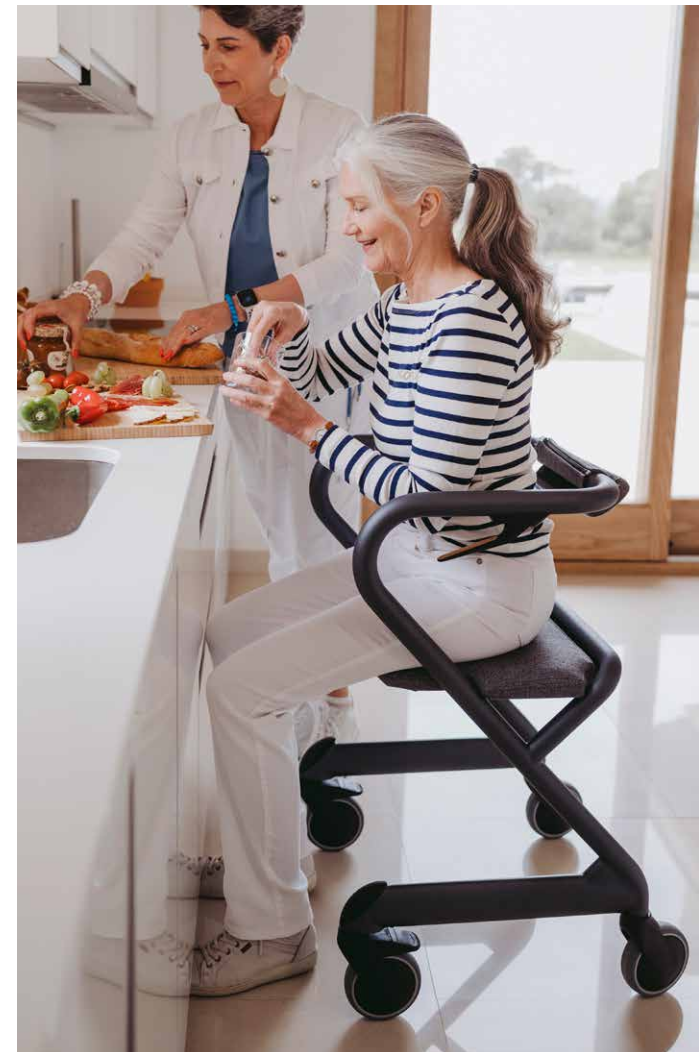

## 2. Mainly as a rollator

Would you like to use your rollator primarily for pushing and walking in your home? Then you need the following measurements:

### Your body height (B)

Measure your total height, preferably with the shoes you normally wear at home. Let someone help you with this, it is easier with two people.

Compare your measurement with the following table and you will find the right indoor rollator for you.

Height (B)	145–165 cm	150–180 cm	170–200 cm	170–210 cm
Your suitable rollator	WR52	WR55	WR58	WRXL

Are your measurements not exactly matched to a rollator size?

Our sales and service team will be happy to advise you on +49 8141 317740 or send an e-mail to [hallo@saljol.de](mailto:hallo@saljol.de).



## Technical data

Indoor Rollator	WR52	WR55	WR58	WRXL
Recommended height	145–165 cm	150–180 cm	170–200 cm	170–210 cm
Maximum user weight	150 kg	150 kg	150 kg	200 kg
Handle height	80 cm	86 cm	92 cm	95 cm
Seat height	52 cm	55 cm	58 cm	58 cm
Bottom width (wheel)	56 cm	56 cm	56 cm	68 cm
Top width	51 cm	51 cm	51 cm	65 cm
Distance between the handles	43 cm	43 cm	43 cm	60 cm
Weight Velvet Grey & Ivory White without accessories	10.4 kg	10.8 kg	11.2 kg	13.4 kg
Weight Diamond Silver without accessories	9.3 kg	9.5 kg	9.7 kg	
Compatible with accessories	✓	✓	✓	

The WRXL Indoor Rollator is available in Velvet Grey and Ivory White on request.

Are your measurements not exactly matched to a rollator size?

Our sales and service team will be happy to advise you on +49 8141 317740 or send an e-mail to [hallo@saljol.de](mailto:hallo@saljol.de).