# Saljol Wohnraumrollator

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## Instruction manual

### Preface

Thank you for purchasing a Saljol Indoor Rollator! With your new rollator, you will be able to lead a mobile and self-determined life in style. The Indoor Rollator has been developed with a lot of attention to safety and will support you in your everyday life.

Please read the instruction manual thoroughly before using your Indoor Rollator for the first time. It contains important safety information, useful tips for use and care, as well as a list of recommended accessories.

## Content

1.	Identification.....	23
2.	General notes.....	24
3.	Notes.....	25
3.1	Safety instructions.....	25
3.2	Instructions for the safe use of the product.....	25
4.	Scope of delivery and product overview.....	26
4.1	Scope of delivery.....	26
4.2	Marking/Label.....	26
4.3	Product overview.....	27
5.	Operation.....	28
5.1	Insert seat board.....	28
5.2	Grip and support positions when walking and standing inside the rollator.....	28
5.3	Grip and support positions when walking and standing outside the rollator.....	29
5.4	Standing up and sitting down.....	29
5.5	Brake system.....	30
5.5.1	Braking and locking the brake with the brake handles.....	30
5.5.2	Braking and locking the brake with the brake bar.....	31
5.5.3	Disassembling/assembling the brake bar.....	31
5.5.4	Adjusting and checking the brake system.....	32
5.6	Locking the direction.....	33
5.7	Tripping with the rollator.....	33
6.	Maintenance.....	33
6.1	Cleaning.....	33
6.2	Disinfection.....	34
6.3	Inspection.....	34
7.	Storage and shipping.....	34
8.	Disposal.....	34
9.	Technical data.....	35
10.	Warranty and guarantee.....	36
10.1	Warranty.....	36
10.2	Guarantee.....	36
11.	Note on the obligation to report incidents.....	37
12.	Accessories.....	38
13.	Test protocol for reuse.....	40

## 1. Identification

Manufacturer: Saljol GmbH  
 Product: Indoor Rollator  
 Type: Indoor rollator, serving trolley and commode chair  
 Article name: Indoor Rollator  
 Version: 2023

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Formalities of the operating instructions

Version/Revision: 2023.03

Creation date: 23.03.2023

Information is correct at the time of going to press.

We reserve the right to make technical changes. Technical drawings and photos may differ from reality.

©2023 Saljol GmbH: all rights reserved. Reprinting, including excerpts, only with the permission of Saljol GmbH

Model Indoor Rollator	Color	Article number
Saljol Indoor Rollator WR52	Velvet Grey	4058789014288
Saljol Indoor Rollator WR55	Velvet Grey	4058789014295
Saljol Indoor Rollator WR58	Velvet Grey	4058789014301
Saljol Indoor Rollator WRXL	Velvet Grey	4058789013045
Saljol Indoor Rollator WR52	Ivory White	4058789014431
Saljol Indoor Rollator WR55	Ivory White	4058789014448
Saljol Indoor Rollator WR58	Ivory White	4058789014455
Saljol Indoor Rollator WRXL	Ivory White	4058789013052
Saljol Indoor Rollator WR52	Diamond Silver	4058789014325
Saljol Indoor Rollator WR55	Diamond Silver	4058789014332
Saljol Indoor Rollator WR58	Diamond Silver	4058789014349

## 2. General Notes

### Purpose

The Indoor Rollator from Saljol is designed for indoor use and even floors. It supports people with balance or gait problems and contributes to increased mobility of disabled people. On the seat board the user can pause and move him- or herself around in a seated position.

### Indication

People who need additional stability support while walking and who have sufficient walking ability, safety and strength to move around at home with the help of the rollator. The Indoor Rollator supports movement-impaired people when getting up, sitting down and transporting. The seat board enables people to rest and move around tripping on even floors.

### Contraindication

The Indoor Rollator is not suitable for people with weak arm strength, significant cognitive limitations or very poor balance. The use and application possibilities of the rollator should be evaluated and instructed by a doctor or therapist.

### Service life

When used as intended, regular care and maintenance by a specialized company and compliance with the safety instructions, the expected service life of the Indoor Rollator is up to 4 years. If after 4 years the rollator is still in proper condition, it can be used again.

### Responsibility

We can only consider ourselves responsible as manufacturer for the safety, reliability and usability of the Indoor Rollator, if changes, extensions, repairs and maintenance work are carried out regularly and only by authorized specialists and if the rollator is used in accordance with all instructions for use.

### Declaration of conformity

Saljol GmbH as manufacturer of the product declares that the Indoor Rollator in all color and size variations meets the requirements of the European Parliament and Council Regulation 2017/745 of the harmonized Standards ISO 11199-2:2005 – Walking aids manipulated by both arms – Requirements and test methods – Part 2: Rollators and ISO 11199-3:2005 – Walking aids manipulated by both arms – Requirements and test methods – Part 3: Walking tables. Handles meets the Biological evaluation of medical devices ISO 10993-5:2009-10.

## 3. Notes

### 3.1 Safety instructions

The instruction manual contains safety notes that are marked twice: by a symbol and by a signal word. The symbol informs you about the type of notice. The signal word explains more details, for example the severity of an impending risk.

#### Caution/Important

Possibly dangerous situation (minor injuries or material damage), commitment to a particular behaviour or activity for the safe use of the rollator

#### Note

Application tips and useful information

### 3.2 Instructions for the safe use of the product

For safe use of the product, please keep the following in mind:

#### Restrictions on use

The maximum user weight must not exceed 150 kg (200 kg for variant WRXL).

#### Accident prevention

- Only use the rollator for the defined purpose.
- Only use the rollator if it is in perfect technical condition.
- It is prohibited to make any constructive changes to the rollator, unless you have the written consent of the manufacturer Saljol.
- While adjusting the rollator, be careful not to pinch your fingers or other body parts on the moving parts.
- Make sure that the parking brake is activated when you sit down on the rollator.
- Make sure that there is no direct skin contact with the seat surface when using this product. The thighs and bottom should always be covered with clothing to avoid skin squeezing.
- Make sure that the seat board does not restrict legroom when walking. The seat board should be placed upright in the rollator, see figure on page 27.
- All four wheels must be in contact with the ground at all times. Only in this way the rollator is safely balanced which prevents accidents.
- Always apply the parking brake when you stop or park the rollator.

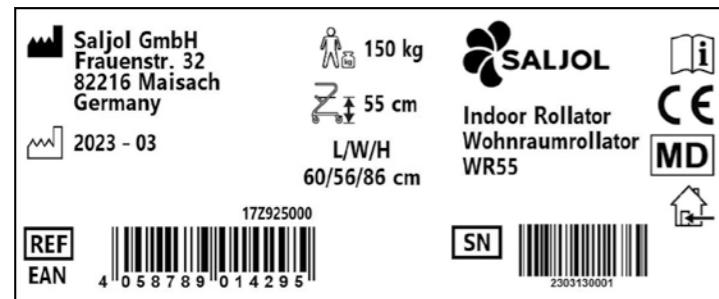
## 4. Scope of delivery and product overview

### 4.1 Scope of delivery

1x Saljol Indoor Rollator  
 1x seat board  
 1x brake bar (pre-mounted)  
 1x instruction manual

### 4.2 Marking/Label

The serial number and EAN code are on a label on the inside of the frame.  
 The label also contains important information about the product: manufacturer's details about Saljol, production date, maximum user weight, product size and seat height. The label must not be removed.

The following symbols can be found, partly in combination with further information, on the marking of the Indoor Rollator and on the packaging to identify important properties.

	Manufacturer of the product		Maximum user weight
	Production date		Seat height
	Order number/item number		L/W/H Length/width/height of the rollator
	Serial number		
	Observe instruction manual		
	CE marking		
	Medical device		
	Product only suitable for indoors		

### 4.3 Product overview

- 1 Frame**  
Secure support from all sides. Serves as mobile handrail.
- Brake system**  
For braking and locking the brakes
- 2a Brake handles for two-handed braking**
- 2b Brake bar for parking brake**
- 3 Wooden seat board**  
Move through your home while sitting.
- 4 Pedal lever for directional locking**  
Rear wheels only run straight ahead.
- 5 360° swivelling wheels**  
Rollator turns on the spot. Easy handling in narrow corridors and confined areas.

Figure: Saljol Indoor Rollator

## 5. Operation

### Inventory

Take the Saljol Indoor Rollator out of the box and check if anything is missing. Compare with the scope of delivery, described under 4.1 on page 26. If any part is missing or damaged, contact your dealer or Saljol immediately.

### 5.1 Insert seat board

#### Important

- Check that the seat board is firmly in place before you sit down.
- Never lean on the area of the seat board marked in the picture, otherwise it may fold downwards.

### 5.2 Grip and support positions when walking and standing inside the rollator

### 5.3 Grip and support positions when walking and standing outside the rollator

### 5.4 Standing up and sitting down

#### Important

When standing up and sitting down, the parking brake must be activated.

## 5.5 Brake system

### Important

When you sit on the Indoor Rollator to rest, it is absolutely necessary to lock the brakes.

As the brake handles and the brake bar are one braking system, the brake bar also moves when the brake handles are used. If you use the brake bar, both brake handles move.

### 5.5.1 Braking and locking the brake with the brake handles

**Braking**  
To brake the rollator, simply pull both brake handles upwards towards the frame.

**Lock the brake**  
To apply the parking brake, press both brake handles down until you overcome a slight resistance. Both rear wheels are now locked.

**Release the brake**  
To release the parking brake, pull both brake handles upwards.

### 5.5.2 Braking and locking the brake with the brake bar

**Braking**  
To brake the rollator, simply pull the brake bar upwards towards the frame at any point. The brake bar brakes both rear wheels simultaneously.

**Lock the brake**  
To apply the parking brake, pull the brake bar upwards until you overcome a slight resistance. Both rear wheels are now locked.

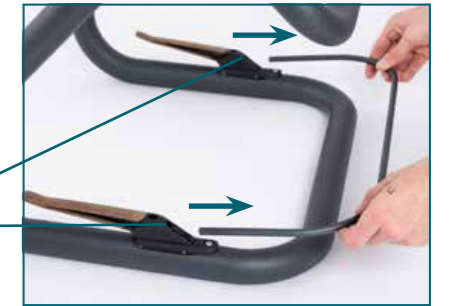
**Release the brake**  
To release the parking brake, press the brake bar down.

### 5.5.3 Disassembling/assembling the brake bar

#### Disassembly

Turn the rollator over and place it on the handrail. Now unscrew the screws on both handles with an allen key size 2 mm.

Now you can remove the brake bar.

Keep the brake bar together with the screws in case you want to reinstall the brake.


#### Assembly

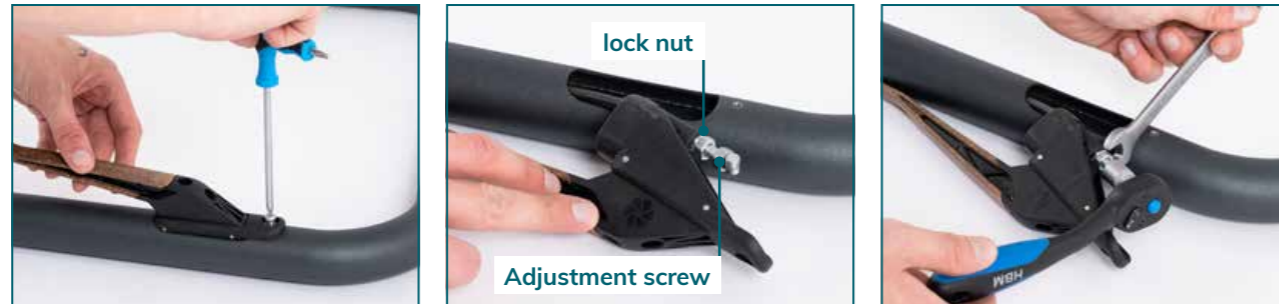To refit the brake bar, insert the brake bar into the holes provided (picture above) and tighten both screws on the brake handles.

### 5.5.4 Adjusting and checking the brake system

Turn the rollator over and place it on the handrail. Have a Torx 20 screwdriver, a 10 mm open-end spanner and a ratchet with a 10 mm socket ready.

First remove the brake bar as described in point 5.5.3, if it is mounted.

1. loosen the screw of the desired brake handle with the screwdriver and pull out the brake housing.
2. loosen the lock nut with the open-end spanner
3. unscrew the tension end sleeve with the ratchet with a 10 mm nut (brake becomes tighter) or screw it in (brake becomes smoother)
4. check brake force
5. repeat point 3-4 until the desired setting is found
6. tighten the lock nut
7. insert the brake housing and tighten the screw
8. mount the brake bar again, if desired

#### Caution

The brake is correctly adjusted if the rear wheels lock when the brake is applied and do not turn when pushed.

#### Information

Here you can find a video explaining the brake adjustment: [www.youtube.com/watch?v=7j1BEbOFeOE](https://www.youtube.com/watch?v=7j1BEbOFeOE) or scan the QR code.

### 5.6 Locking the direction

To lock the direction of the rear wheels, press down the pedal levers on both sides until you hear a soft „click“ sound. As soon as you drive straight ahead, the wheels lock automatically.

To release the directional lock, step on the top of the pedal levers.

### 5.7 Tripping with the rollator

It is permitted to move around your home while sitting on the Saljol Indoor Rollator. This is how you move safely through your home:

1. check that the seat board is firmly fixed
2. lock the brakes
3. sit down
4. release the brakes

Now you can move around the rooms with your feet

## 6. Maintenance

### 6.1 Cleaning

#### Important

- Never use aggressive cleaning agents such as solvents/abrasives or hard brushes.
- Never clean the rollator with a high-pressure/steam cleaner!

#### Frame

Cleaning can be done with warm water and a sponge. In case of stubborn soiling, a commercial cleaning agent can be added to the water. Disinfectants can be used without any problems. Plastic parts are best cleaned with a commercially available plastic cleaner. Please follow the instructions for use of the cleaning agent used.

### Wheels

Clean the wheels with warm water and a mild solvent. If necessary, use a plastic brush, never a wire brush.

### Seat board

Clean the seat board with a cloth and warm water. If necessary, use a mild detergent.

Dry the rollator thoroughly with a cloth after cleaning.

## 6.2 Disinfection

The Indoor Rollator can be disinfected with standard household disinfectants. The Robert Koch list of approved disinfectants can be found at <http://www.rki.de>. Allow the rollator to air dry and observe the exposure time of the disinfectant you are using.

## 6.3 Inspection

We recommend that the Indoor Rollator is inspected by the specialist dealer/Saljol at least once a year, ideally even twice. If faults or defects occur during use, the Indoor Rollator must be returned immediately to the specialist dealer/Saljol for repair. Saljol reserves the right to limit the guarantee claims if the rollator has not been serviced regularly.

## 7. Storage and shipping

When not in use, keep the rollator upright, in a dry place, out of reach of children or third parties. If the Indoor Rollator needs to be stored or shipped, it is advisable to keep the original packaging and store it for these purposes so that it is available when needed. Then your rollator is optimally protected against environmental influences during storage or transport.

## 8. Disposal

Dispose materials separately by type in the designated collection systems. Please dispose hazardous waste in the designated collection systems.

## 9. Technical data

Indoor Rollator	WR52	WR55	WR58	WRXL
Handle height	80 cm	86 cm	92 cm	95 cm
Seat height	52 cm	55 cm	58 cm	58 cm
Recommended body height	145–165 cm	150–180 cm	170–200 cm	170–210 cm from hip width 45 cm
Max. user weight	150 kg	150 kg	150 kg	200 kg
Length	60 cm	60 cm	60 cm	60 cm
Largest width (bottom)	56 cm	56 cm	56 cm	68 cm
Largest width (top)	51 cm	51 cm	51 cm	65 cm
Distance between the handles	43 cm	43 cm	43 cm	60 cm
Weight Velvet Grey/ Ivory White without accessories	10.4 kg	10.8 kg	11.2 kg	13.4 kg
Weight Diamond Silver without accessories	9.3 kg	9.5 kg	9.7 kg	
Frame	Ivory White and Velvet Grey: powder-coated steel Diamond Silver: electropolished stainless steel			
Wheels (diameter x width)	12 x 3 cm, grey, anti-slip TPE-wheels			
Turning circle	69 cm			85 cm

## 10. Warranty and guarantee

### 10.1 Warranty

If the purchased item is defective, the provisions of the statutory liability for defects shall apply. Insofar as you have purchased the goods from a specialist retailer, please contact your seller immediately; if you have purchased directly from Saljol, please contact Saljol directly, Tel. +49 (0)8141 317740. Visible defects must be reported in writing within ten days of delivery, hidden defects within ten days of their discovery. The above-mentioned deadlines are preclusive deadlines. Saljol shall be entitled to decide whether a defect, if any, shall be remedied by rectification or subsequent delivery („subsequent performance“). In all other respects, the provisions in the General Terms and Conditions of Saljol, available at <https://www.saljol.de/en/data-protection/>, shall apply.

### 10.2 Guarantee

In addition to the statutory guarantee claims and without limiting them, Saljol gives a guarantee of 2 years from the date of purchase on this product. You also have the option to extend this guarantee by a further 2 years if you register the rollator online no later than 14 days after the date of purchase. You can register at [www.saljol.de/en/extended-warranty/](http://www.saljol.de/en/extended-warranty/)

#### Scope of the guarantee

We guarantee that the product will be free from defects in material, workmanship and function within the guarantee period. If a defect occurs during the guarantee period, we will, at our own discretion, either repair the product free of charge or, if necessary, replace it with a new one. Excluded from the guarantee are parts subject to wear and tear such as brake mechanism and tyres.

These guarantee conditions only apply to products purchased in Germany, provided they were purchased from Saljol directly or from an authorised specialist dealer.

The seller will provide information on the guarantee conditions for products purchased abroad. Guarantee services and replacement deliveries for guarantee reasons do not result in an extension of the guarantee period.

#### Exclusion of the guarantee

The right to claim under guarantee expires if the rollator has been improperly maintained or repaired by an untrained person or if the product has been operated with non-original accessories. Incorrect set-up or installation due to non-observance of the operating instructions, use contrary to the intended purpose or improper use or care of the product will result in a loss of guarantee, as will external influences (e.g. transport damage, damage due to impacts and blows, weather influences).

## 11. Note on the obligation to report incidents

Please note that all serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority. Serious incidents in the sense of the law are all adverse events that have one of the following consequences: Death, life-threatening injury, permanent bodily damage and chronic illness.

#### In the Federal Republic of Germany, please inform:

- your local distributor, Saljol sales partner or Saljol GmbH, Frauenstr. 32, D-82216 Maisach; [www.saljol.de](http://www.saljol.de)
- the Federal Institute for Drugs and Medical Devices (BfArM), Kurt-Georg-Kiesinger-Allee 3, 53175 Bonn; [www.bfarm.de](http://www.bfarm.de)

#### In EU Member States, please inform:

- your local distributor, SALJOL sales partner or Saljol GmbH, Frauenstr. 32, D-82216 Maisach; [www.saljol.de](http://www.saljol.de)
- the authority responsible for incidents involving medical devices in your country

## 12. Accessories

### Note

The following accessories are available for the Indoor Rollator in sizes WR52, WR55 and WR58. No separate accessories are available for size WRXL.

### Tray

The stable wooden tray can be loaded up to 20 kg and has a large usable surface. The tray turns the Indoor Rollator into a serving trolley with which you can even drive over the table and safely put the tray down – without danger of crushing your fingers.

### Designer bags from Jost

Turn your indoor rollator into an eye-catcher! The designer bags from Jost are available with beige imitation leather, with a dark grey felt cover or in transparent PVC. Transport items up to 20 kg load weight safely stowed through your home. You can hang the bag both on the handle and directly under the seat.

### Back roll

The back roll can be attached to the frame of the Indoor Rollator with a zipper. This makes the rollator a comfortable movable chair. It can also be used as impact protection on the side tubes or the crossbar. Available in woven fabric grey and imitation leather dark brown.

### Upholstered seat

With the upholstered seat your Indoor Rollator becomes a comfortable tripping and transfer wheelchair. Ideal for everyone who uses the rollator, e.g. as a kitchen or work chair. The seat has a load capacity of up to 120 kg and is easy to attach and detach. Also available in synthetic leather dark brown. Available in woven fabric grey and imitation leather dark brown.

### Cabinet

The cabinet has a load capacity of 20 kg and can simply be hooked into the top of the frame of the Indoor Rollator. With its two storage compartments, the cabinet is ideal for use as a bedside table.

### Bag

The bag made of medium grey fabric can be loaded with 20 kg and is ideal for transportation.

hooked in at the top

hooked in at the bottom

### Seat cover

The danger of slipping away while sitting is reduced by the seat cover. The seat cover is simply pulled over the seat board and secured with Velcro. The grip hole remains unchanged.

### Note

You can order the accessories in our online shop [www.saljol.de](http://www.saljol.de) around the clock or by telephone via our customer service (Mon.-Fri. 9am-4pm) on the number +49 (0)8141 317740 from home. Available accessories may vary depending on the country.