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Instruction manual

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DE

EN

NL

Vorwort

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NL

Preface

Thank you for choosing to purchase a Riser Armchair from Saljol! The Saljol Riser Armchair has been designed with a strong focus on safety, aimed at making it easier for you to relax at home.

Please read the user manual thoroughly before using your chair for the first time. It contains important safety instructions, useful tips on usage and maintenance, as well as a list of recommended accessories.

Content

1. Identification.....	32
2. General notes.....	32
3. Notes.....	34
3.1 Safety instructions.....	34
3.2 Guidelines for safe use of the product.....	35
3.3 Electromagnetic field.....	37
4. Scope of delivery and product overview.....	37
4.1 Scope of delivery.....	37
4.2 Marking/label.....	38
4.3 Product description.....	39
4.3.1 Base.....	40
5. Transport.....	40
5.1 Transporting the boxes to the chair's setup location.....	40
5.2 Unpacking the chair.....	41
6. Assembly of the chair.....	42
7. Initial operation.....	46
7.1 Activating the emergency function.....	46
7.2 Moving the chair.....	46
7.3 Functions of the remote control.....	47
7.4 USB Ports.....	50
8. Deactivating the chair.....	51
9. Cleaning and maintenance.....	51
9.1 Cleaning.....	51
9.2 Disinfection.....	52
9.3 Maintenance.....	52
10. Storage and shipping.....	52
10.1 Storage conditions.....	52
11. Technical data.....	53
12. Warranty.....	54
13. Disposal.....	54
13.1 Destruction/recycling.....	54
14. Accessories.....	55

1. Identification

Manufacturer: Saljol GmbH
Product: Saljol Riser Armchair
Type: Riser Armchair
Article name: Saljol Riser Armchair
Version: 2024

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Swiss representative and importeur:

Rehabilitations-Systeme AG
Roggenstrasse 3
CH-4665 Oftringen

Formalities of the operating instructions

Version/Revision: 2025.01

Creation date: 07.08.2025

Information is correct at the time of going to press.

We reserve the right to make technical changes. Technical drawings and photos may differ from reality.

©2025 Saljol GmbH: all rights reserved. Reprinting, including excerpts, only with the permission of Saljol GmbH.

2. General notes

Intended use

The Saljol Riser Armchair is a two-motor riser chair, where the footrest and backrest can be controlled independently via an electric drive system. The two motors are operated through the remote control.

This allows you to move the chair from seated to standing position and back, saving you energy and relieving strain on your legs. Additionally, the chair can be reclined into a lying position and back, also controlled by the remote. The Saljol Riser Armchair is also equipped with an adjustable headrest.

Contraindications

The Saljol Riser Armchair is not suitable for individuals with severe orthopedic limitations, significant balance issues, or major cognitive impairments. The use and suitability of the Saljol Riser Armchair should be assessed and prescribed by a doctor or therapist.

Service life

When used as intended, with regular care and maintenance by a professional service provider, and adherence to safety instructions, the expected lifespan of the Saljol Riser Armchair is up to 4 years. If the chair remains in proper working condition after 4 years, it may continue to be used.

Material Characteristics: Upholstered furniture tends to become softer over time and with varying levels of use. Creases and indentations may form in any upholstery material under the influence of body weight. These visual changes do not affect the chair's usability, function, or lifespan.

Responsibility

As the manufacturer, we can only ensure the safety, reliability, and usability of the chair if modifications, extensions, repairs, and maintenance are carried out regularly and exclusively by authorized professionals. The chair must also be used in accordance with all provided instructions.

Declaration of conformity

- CE** Saljol, as the manufacturer of the product, declares that the Saljol Riser Armchair complies with the essential requirements of the following standards and directives:
- EMC Directive (2004/108/EC): Electromagnetic compatibility, including the testing of the USB port.
 - EN 60335-1:2012 (LVD): Safety of electrical appliances for household and similar purposes, including the testing of the USB port.
 - EN ISO 12100:2010: Safety of machinery – General principles for design – Risk assessment and risk reduction.
 - EN 1022:2018: Domestic furniture – Seating – Determination of stability.
 - EN 12520:2015 (including EN 1728:2014): Furniture – Strength, durability, and safety – Requirements for domestic seating.
 - BS 5852: Fire safety requirements.

Saljol Riser Armchair	Platinum Grey	4058789010945
Saljol Riser Armchair	Graphite Grey	4058789010952
Saljol Riser Armchair cushion set firm	Platinum Grey	4058789010969
Saljol Riser Armchair cushion set firm	Graphite Grey	4058789010983
Saljol Riser Armchair cushion set soft	Platinum Grey	4058789010976
Saljol Riser Armchair cushion set soft	Graphite Grey	4058789010990

3. Notes

3.1 Safety Instructions

The user manual employs safety instructions that are marked in two ways: with a symbol and a signal word. The symbol informs you about the type of notice, while the signal word provides further details, such as the severity of the potential risk.

Caution/Important

Potentially hazardous situation (minor injuries or property damage). Specific actions or behaviours required to ensure safe operation of the chair

Danger/Warning

Potentially imminent threat (serious injuries or death). Specific actions or behaviours required to ensure the safe use of the chair

Danger

Life-threatening electrical hazard due to high voltage

Risk

Risk of hand injury from crushing

Note

Usage tips and particularly useful information

3.2 Guidelines for safe use of the product

Read and follow the instructions in this user manual to operate the Riser Armchair safely. Do not attempt any modifications or repairs on your own.

Usage restrictions

- The Riser Armchair can be used by children aged eight and older, as well as by individuals with limited physical, sensory, or mental capabilities or a lack of experience and knowledge, provided they have received instruction on the safe use of the device and understand the associated risks.
- Children must not play with the Riser Armchair.
- Do not drop into the chair.
- Only one person should use the chair at a time.
- Ensure that no one jumps on the chair.
- Do not overload the chair with more than 150 kg.
- Do not use the armrests, backrest, or footrest as seating surfaces.
- Activate the rise/relax function only when you are seated correctly in the chair.
- Keep people and pets away from the chair while the rise and/or relax function is in operation to prevent the risk of pinching.
- Never use the chair in the presence of flammable, combustible, or burning materials.

Accident prevention

- When in the sitting/relax position, all four wheels must always be in contact with the floor. This ensures that the chair remains balanced and helps prevent accidents.
- Use the Riser Armchair only for its intended purpose.
- Only use the chair if it is in perfect technical condition to ensure operational safety.
- Ensure sufficient clearance from walls or objects when activating the rise and/or relax function.
- Use only original accessories and spare parts.
- Do not attempt to repair the chair yourself.
- Maintenance of the Riser Armchair should only be carried out by qualified or trained personnel.

Risk of pinching

- Fingers or other body parts of the user and/or others could be pinched during the assembly of the chair or when adjusting the seat depth. Therefore, be extremely careful and attentive when setting up the device. Be aware of the gaps between the chair parts to protect yourself from pinching.
- When the chair is raising or lowering, extremities could be pinched by the mechanisms under the chair and by the backrest. Ignoring these risk areas may result in injuries.

Electrical voltage

- Do not open, remove, or damage the motor housing or other drive components.
- Use only the power supply or the optional battery.
- When routing the power supply cable, ensure it does not create a tripping hazard.
- Do not run over the power supply cable with the wheels.
- Keep the chair, cable, and power supply away from heat sources.
- Disconnect the power supply before moving the chair.
- Do not use the chair if the cable, power plug, or battery is damaged.
- Magnets generate a strong and extensive magnetic field. Keep the magnetic remote control away from devices and objects (e.g., screens, credit cards, data carriers, pacemakers) that could be damaged by strong magnetic fields.
- Only qualified electricians should work on the electrical motors. Before cleaning or maintaining the chair, it must be disconnected from the power supply! Always unplug the chair from the mains before performing any work on it.

3.3 Electromagnetic field

Caution

If you use a medical device such as a pacemaker or hearing aid, we recommend consulting your doctor before purchasing or using the chair to determine if there is any risk to you or your medical device from using the chair.

4. Scope of delivery and product overview

4.1 Scope of delivery

Box 1: Chair base with side pocket

Box 2: Backrest

Box 3: 5x Cushions (1x Headrest, 1x Backrest cushion, 1x Seat/Foot cushion, 2x Armrest cushions)

1x Power supply unit

1x Remote control

1x User manual

4.2 Marking/label

The product label is affixed to the back of the frame. It contains important information about the product: manufacturer details for Saljol, product name, production date, serial number, and maximum user weight.

The label must not be removed.

The following symbols, sometimes in combination with additional information, can be found on the label of the Saljol Riser Armchair as well as on the packaging, indicating important characteristics.

2 min ON, 18 min OFF

2 minutes usage/18 minutes rest period

Protection class II

For indoor use

Refer to the user manual

Recyclable


CE marking

4.3 Product description


Figure 1: Saljol Riser Armchair

4.3.1 Base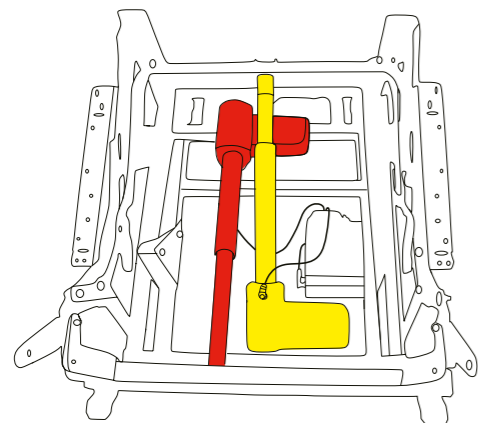

Frame and motor

The yellow-marked motor lifts the chair into the standing position or moves the footrest. The red-marked motor controls the movement of the backrest.

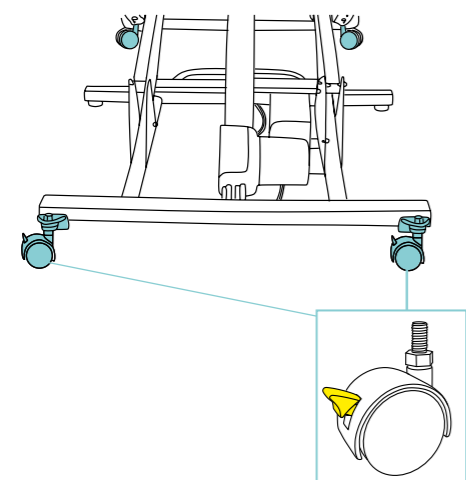Castors

The base with castors allows for easy movement of the chair. Parking brakes (yellow-marked in the image) on the castors prevent the chair from rolling during use.

Danger

Moving the chair while someone is seated is prohibited!

Important

Once the chair is in the desired position, the parking brakes must be engaged!

5. Transport

5.1 Transporting the boxes to the chair's setup location

Warning

Risk of impact and crushing when lifting and transporting the chair.

- The best transport method for the chair is a pallet truck or a sack truck.
- Determine the best transport route.
- Walk the route beforehand and check for obstacles.
- Remove any obstacles if necessary.
- Decide where the chair will be placed. Ensure a power outlet is nearby for connecting the power supply cable.
- Move and transport the three packaging boxes individually.
- Always lift the boxes carefully.
- Only lift them slightly off the ground.
- Transport the boxes as close to the ground as possible to the setup location.
- Lower the boxes slowly and gently.
- Ensure the boxes are placed with the marked side facing upwards.

5.2 Unpacking the chair

Follow packaging guidelines and transport instructions.

Do not cut through the packaging.

Improper opening of the packaging may damage the contents. To avoid damaging the chair, proceed with the unpacking as follows:

- Place the box on the floor with the arrows pointing upwards.
- Tilt the box onto its side.
- Lay the box on its side (tilt 90°) and then open the bottom flaps of the box.
- Fold the three free flaps completely outwards.
- To fully open the fourth flap, stand the box upright again so the arrows point upwards.
- Now lay the box on the other side (tilt 90°). You can now fully open the fourth flap.
- Tilt the box back so that the direction arrows are pointing upwards.
- Simply lift the box away.
- The chair is now unpacked and exposed.

6. Assembly of the chair

Note

We recommend assembling the chair with two people


Warning

Danger to life due to electric current! The motor housing or other drive components must not be opened, removed, or damaged. During the assembly of the chair, the power supply cable must be disconnected from the mains!

First, unpack the seat unit


- Position it where the chair will later be used. Consider the required area of the chair when the backrest is lowered and the footrest is extended.
- Ensure there is enough space around you to assemble the parts without bumping into furniture or walls.
- Then engage the parking brake on the seat unit to prevent the chair from rolling away during assembly.

Assembling the backrest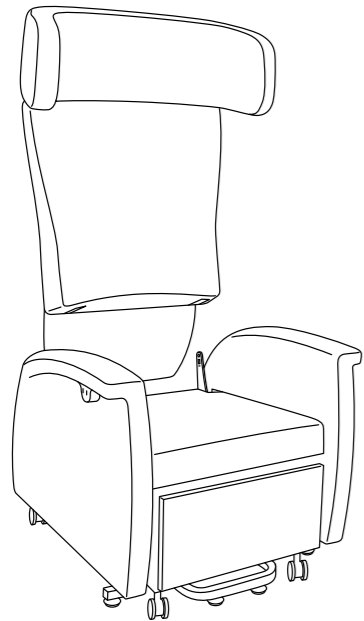

- Your chair consists of a seat unit and the backrest, which need to be connected.
- Hold the backrest with its rear facing you in both hands and stand at the back of the seat unit.
- Choose your preferred seat depth setting:
 - a) Short Seat Depth
Attach the backrest so that the metal plates on the seat unit click securely into the “Kurz” slot.
 - b) Long Seat Depth
Attach the backrest so that the metal plates on the seat unit click securely into the “Lang” slot.
- Slide the backrest onto the protruding metal plates of the seat unit, ensuring that the metal plates are on the inside.
- Slowly lower the backrest until you feel resistance. Press firmly on the backrest to ensure the fittings lock into place.

Connecting the power supply

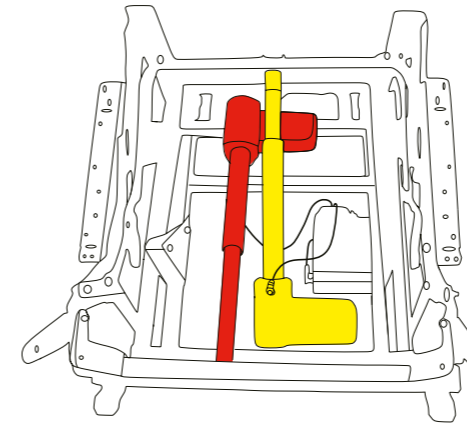The electrical components of the chair are located in the base of the seat unit. The yellow-marked motor lifts the chair into the standing position or moves the footrest. The red-marked motor controls the movement of the backrest.

To connect to the power supply, use the attached cable. Simply plug it into a household power outlet.

Note

Once the power supply is connected, the device will only switch on after a pause. Therefore, wait at least 20 seconds before pressing any buttons and operating the chair.

Warning

The cables must not be crushed, bent, or damaged! Avoid creating tripping hazards and ensure the cable is positioned correctly!

Caution

Before proceeding, check that:

- the seat unit and backrest are securely connected,
- the area of the backrest and footrest is clear,
- the parking brakes are engaged,
- the cables are undamaged and connected to the power supply.

Assembling the upholstery


Info

Be sure to keep the velvet ribbons that are attached to the Velcro fasteners on the cushion covers. If you want to wash the covers, stick the velvet ribbons back onto the Velcro fasteners before washing to avoid scratches.

1. Initial position

Once the seat unit and backrest are assembled and the power supply is connected, you can proceed to attach the upholstery.
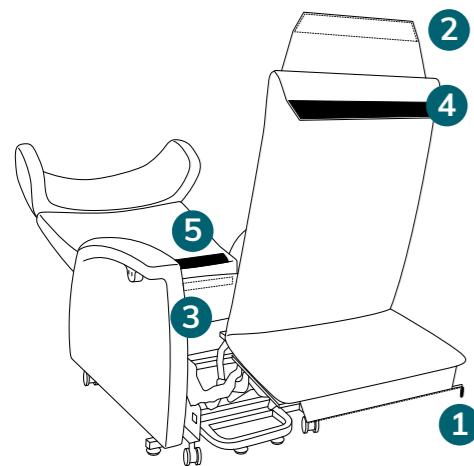


2. Attaching the seat cushion

Position the chair in the relax position by holding down the marked button on the remote control until it stops.

To attach the seat cushion, place it on the footrest so that the fabric strip with the zipper (1) is facing downwards and can be closed underneath the footrest. Zip up the fabric strip under the footrest. Place the cushion on the seat unit, pull it tight, and fasten the Velcro fastener (2) behind the seat surface of the seat unit (3).



Finally, secure the Velcro fastener (4 – Note that there is no Velcro fastener (4) for the “Soft” upholstery variant) to the lower part of the backrest (5).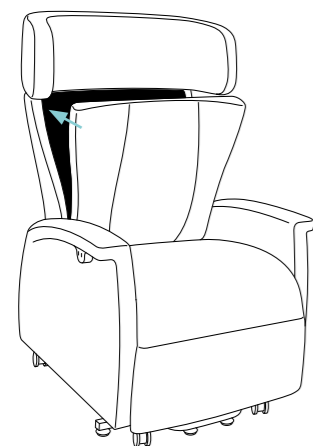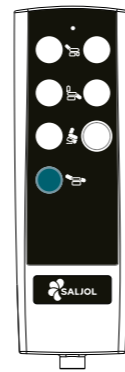

3. Attaching the back cushion


Bring the chair to the upright sitting position by pressing the marked button on the remote control until it stops.

Align the narrower part of the back cushion with the rear of the seat unit and place the cushion against the backrest. Smooth it down firmly so that the Velcro fastener adheres properly. Ensure there is no gap visible between the back cushion and the seat cushion.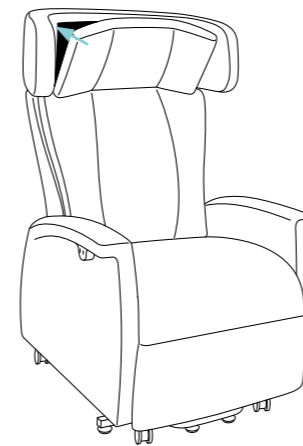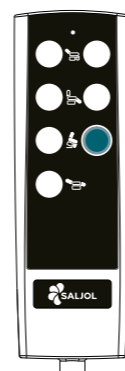

4. Attaching the head cushion

Position the head cushion so that the zipper on the back is facing downwards. Smooth the cushion firmly to ensure the Velcro fastener connects properly. The head cushion should be installed in a way that creates a seamless connection between the head cushion and the back cushion.

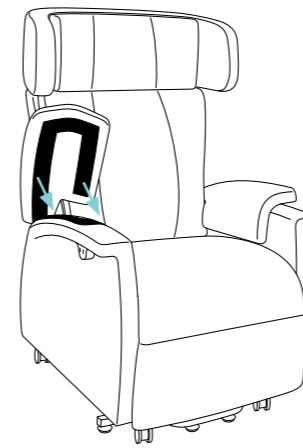5. Attaching the arm cushions

Finally, attach the arm cushions as shown on the armrests and secure them with the Velcro fastener. It is best to start by fastening the long outer Velcro strip and then pressing the upholstery inwards over the backrest.

Once all the cushions are in place, your riser armchair is fully assembled. You can now adjust the headrest by tilting it forward or backward with force to achieve the desired position.

Info

You can find a video on how to assemble the cushions at <https://vimeo.com/1011246110> or by scanning the QR code.

7. Initial operation


Inspection

Perform a visual inspection and conduct a test run with no one in the chair. Please ensure the following:

- Check that the power supply cable is properly plugged into the socket.
- Verify that all components of the chair are fully and correctly assembled.
- Conduct a test run to ensure all parts of the chair move freely.
- Ensure that no one is in the area of the chair.

7.1 Activating the emergency function

In the unlikely event of a power outage, the emergency function can move the chair through a few cycles. To activate this, insert 2x 9-volt block batteries into the 12-volt power supply unit during assembly.

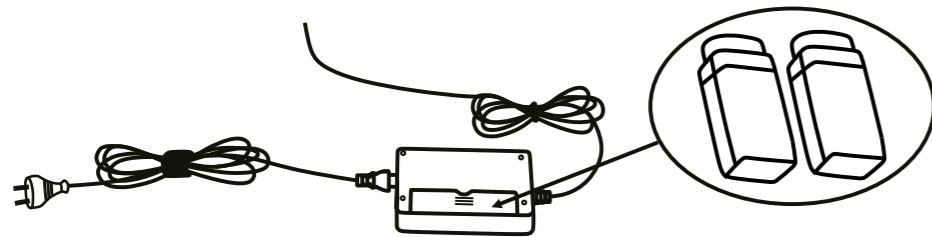7.2 Moving the chair

Before moving the chair, make sure not to move it too far from the power outlet. It is generally advisable to unplug the power cable when relocating the chair. If necessary, store the power cable in the side pockets of the chair and avoid any obstacles.


Danger

Before moving the chair, ensure that:

- the chair is in the neutral position (backrest upright, seat in sitting position, footrest retracted),
- the power supply is unplugged and stored in the side pocket,
- the path for moving the chair is clear of obstacles such as furniture, cables, people, etc.
- the chair cannot be adjusted while being moved, and
- the chair is only pushed forward.

7.3 Functions of the remote control


Remote control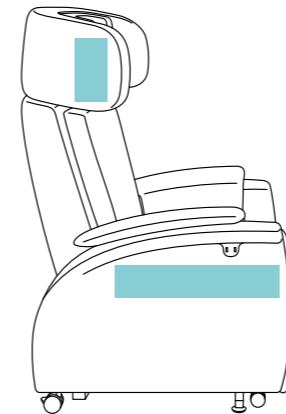


The magnetic remote control can be attached to the side of the headrest or to the outer sides of the armrests (see marked areas in the image).

Components of the remote control:


- Operation light: Illuminates when a function is activated
- Function buttons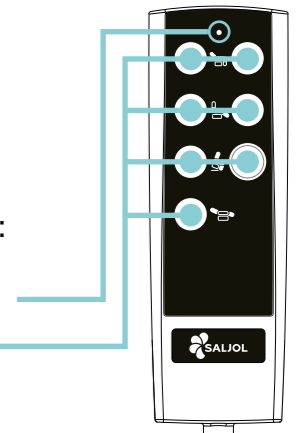

Sitting down safely in the chair

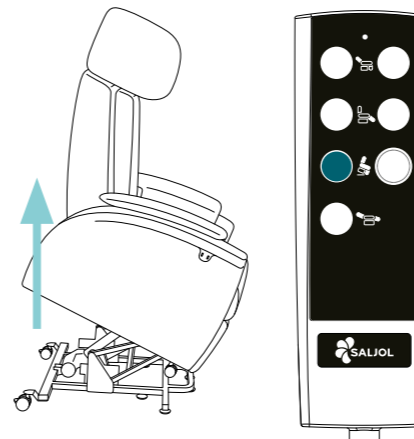
- Stand in front of the chair.
- Press the marked button on the remote control.
- The chair will tilt forward and upward.
- Keep the button pressed until the seat is in the optimal position for you to lean against it with your buttocks.
- Lean back into the chair with your buttocks.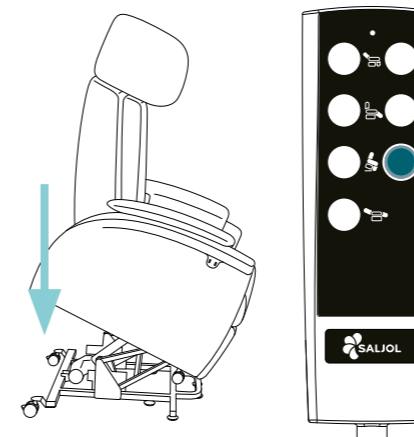

- Press the marked button on the remote control.
- The chair will slowly lower with you in it.
- Once the chair is level with the floor, release the button.

Standing up safely from the chair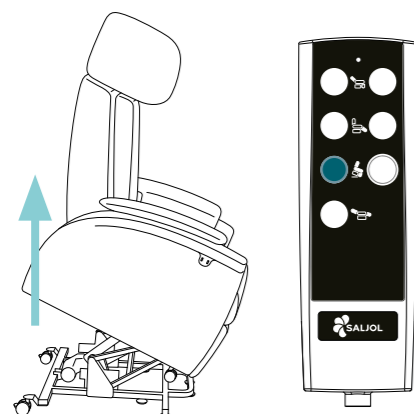


- Press the marked button on the remote control.
- Hold the button down.
- The chair will rise and bring you to a standing position.
- Once the chair reaches the ideal height for you, release the button.
- You can then move away from the chair.

Note!


- As the chair rises from the relax position, it first moves to the upright sitting position before transitioning to the standing position.
- You can hold onto the armrests while the chair is moving.
- You can leave the chair in the upright position in which you left it, making it easier to lean back and sit down again later.

Bringing the chair to the relaxing position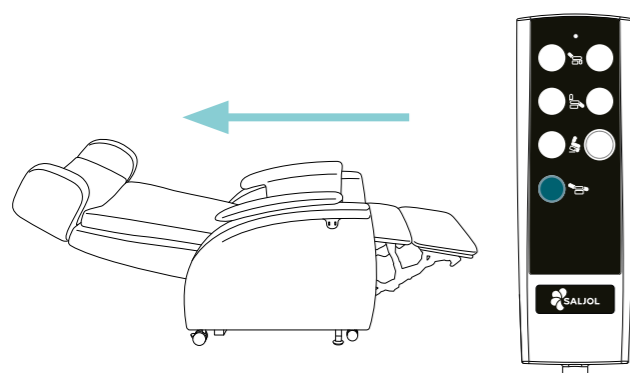


- Press the marked button on the remote control.
- Hold the button down.
- The footrest will extend, and the backrest will tilt backward.
- Once the relaxing position is reached, release the button.

From relaxing position to upright sitting position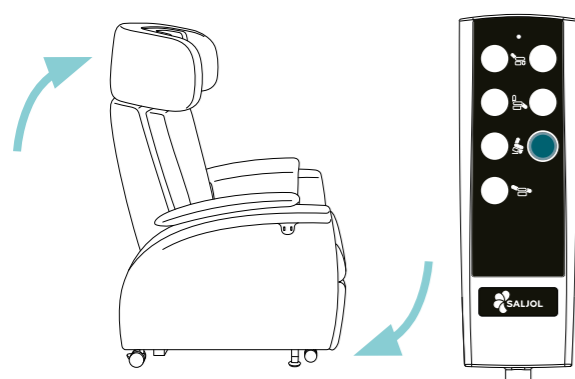


- Press the marked button on the remote control.
- Hold the button down.
- The backrest will move upward, and the footrest will retract.
- Once the chair reaches the upright sitting position, release the button.

Individual operation of the footrest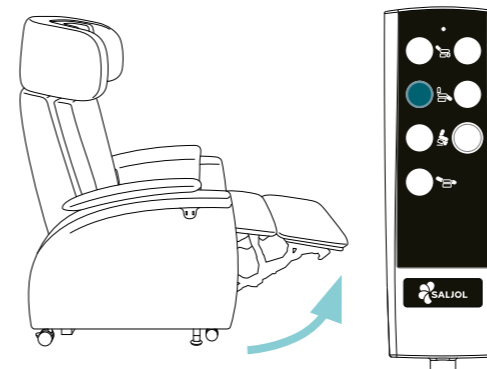

Extending the footrest


- Press the marked button.
- The footrest will extend upward.
- Keep the button pressed until you reach a comfortable position.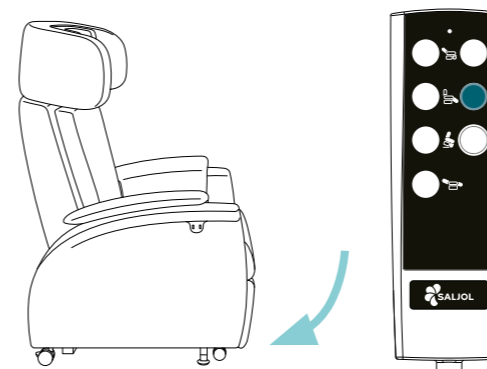

Retracting the footrest


- Press the marked button.
- The footrest will retract downward.
- Keep the button pressed until the motor stops.

Individual operation of the backrest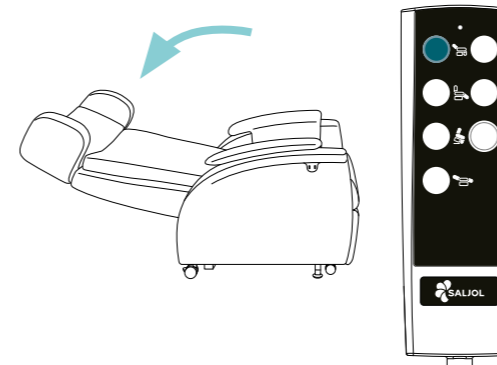

Tilting the backrest backward


- Press the marked button on the remote control.
- The backrest will tilt backward and downward.
- Keep the button pressed until the motor stops or you find a comfortable resting position.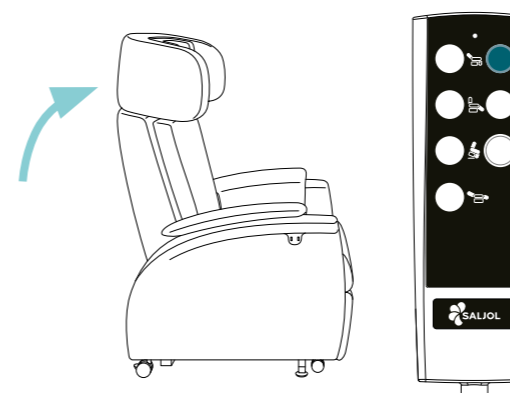

Tilting the backrest upward

- Press the marked button to tilt the backrest upward or to sit up slightly.
- The backrest will move upward.
- Keep the button pressed until you reach the desired height or the motor stops.

Resetting the chair's control unit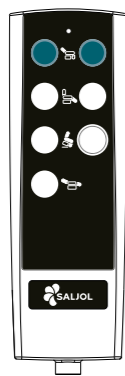


In rare cases, it may be necessary to reset the control unit of the riser armchair. This might occur if the chair has not been used for a long period or if it was abruptly disconnected from the power source.

To reset the chair, follow these steps:

- Press and hold the two marked buttons until the chair moves to the sitting position.
- Once the LED light on the remote control blinks three times, the chair has been successfully reset and can be used again.

7.4 USB Ports

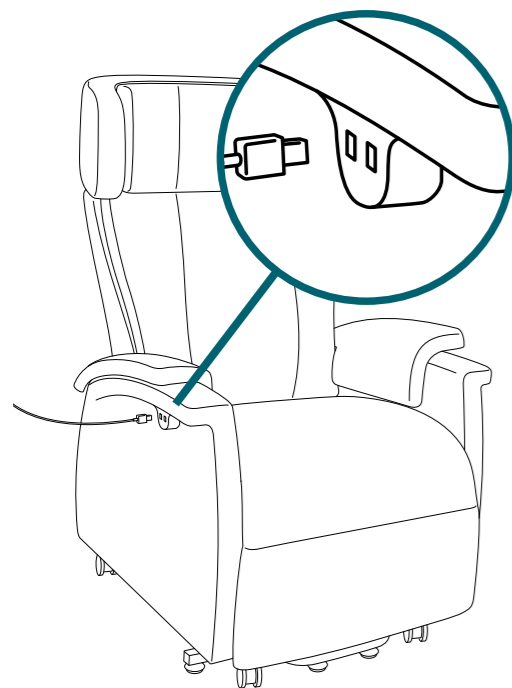Two USB Type-A ports are located on the side panel below the armrest. This allows you to charge your phone or tablet directly from the chair, so you don't need to get up when the battery is low.

Ensure the cable is long enough to connect your devices without being too short or too long.

To charge a smartphone or mobile device connect one end of the USB cable to your device's USB port and the other end to one of the two USB Type-A ports under the right armrest of the chair.

Please disconnect the USB cable after charging is complete or if the device becomes hot. This helps prevent battery damage, overheating, and unnecessary power consumption, thereby extending the life of the battery.

8. Deactivating the chair

If you are no longer using the chair, deactivate it by unplugging the power cable from the socket.

If you are going to be away for an extended period, such as during a holiday, you can temporarily deactivate the chair.

Simply unplug the power cable from the socket.

Note!

If you have temporarily deactivated your chair and wish to use it again, you will need to follow the initial operation steps described in Chapter 7.

9. Cleaning & maintenance

Note!

All cushion covers can be easily removed and washed in the washing machine at up to 60°C. However, please be aware that washing may reduce or remove fire and flame retardant properties.

In the event of a defect or if you wish to switch to a different upholstery option, the cushions can be replaced without difficulty.

9.1 Cleaning

Fabric covers (100% polyester):

Surfaces can be vacuumed. For stubborn stains, a standard upholstery cleaner can be used. Be sure to follow the specific instructions for the cleaning product and test it first on an inconspicuous area.

Caution:

Never use solvents such as petrol, alcohol, or similar substances for cleaning!

9.2 Disinfection

Cushions covered with leatherette can be easily disinfected using standard household surface disinfectants.

9.3 Maintenance

The chair should undergo an inspection by a specialist at least once a year. If malfunctions or defects occur, the chair must be immediately handed over to the dealer for inspection.

Only qualified electricians are allowed to service the electrical connections and components of the motor, including auxiliary and control connections. The chair must be disconnected from the power source during maintenance or repairs. The power cable must be unplugged from the electrical outlet and secured to prevent accidental reconnection.

All maintenance and repair tasks must be performed by trained personnel familiar with the chair. The motor housing and other drive components must not be opened, removed, or damaged. Always disconnect the chair from the power supply before performing any work.

10. Storage and shipping

If the riser armchair needs to be stored or shipped, it is advisable to keep the original packaging for these purposes. This ensures that the chair is optimally protected against environmental influences during storage or transport.

10.1 Storage conditions

The riser armchair should be stored in dry, well-ventilated, and preferably temperature-controlled rooms, protected from light, so that no external pressure is exerted.

Protect the fabric of the chair from light exposure and fading by covering it with a protective cover, ideally the original packaging.

Temperature: 0° to +35°C
Humidity: 45% to 60%

11. Technical data

Saljol Riser Armchair		RA cushion firm	RA cushion soft
Mechanical data	Seat width	47 cm	42 cm
	Seat height	50cm	50 cm
	Seat depth	55.9/61 cm	55.3/60.4 cm
	Recommended body height	ca. 160–190 cm	ca. 160–190 cm
	Seat length in resting position	190.5 cm	190,5 cm
	Maximum load	150 kg	150 kg
	Space requirement	200 x 80 cm	200 x 80 cm
	Chair height	122 cm	122 cm
	Chair width	77.5 cm	77.5 cm
Backrest	Chair weight	65 kg	66 kg
	Inclination	approx. 160°	approx. 162°
Electrical data	Operating voltage	230 Volt	230 Volt
	Control voltage	24 Volt	24 Volt
	Power consumption	6 Amp	6 Amp
	Rated voltage	100–240 V	100–240 V
	Rated frequency	50–60 Hz	50–60 Hz
	Rated power consumption	100 W	100 W
Rechargeable battery	Runtimes	30 switching cycles (from sitting position -> lying position -> stand-up position)	30 switching cycles (from sitting position -> lying position -> stand-up position)
Installation site		Closed rooms	Closed rooms
Operating mode ¹⁾ with max. nominal load		10 %/ max. 5 switching cycles	10 %/ max. 5 switching cycles
Max. adjustment speed ²⁾		44 mm/s	44 mm/s

¹⁾ Operating mode = intermittent operation DOWN 2 min/18 min, i.e. run for a maximum of 2 minutes under nominal load, after which a pause of 18 minutes must be observed. Failure to do so may result in malfunction!

²⁾ Adjustment speed = speed at which the fork head can be moved without load (the adjustment speed varies according to the load).

12. Warranty

If the purchased item is defective, the provisions of the statutory liability for defects shall apply.

Insofar as you have purchased the goods from a specialist retailer, please contact your seller immediately; if you have purchased directly from Saljol, please contact Saljol directly, Tel. +49 (0)8141 317740. Visible defects must be reported in writing within ten days of delivery, hidden defects within ten days of their discovery. The above-mentioned deadlines are preclusive deadlines. Saljol shall be entitled to decide whether a defect, if any, shall be remedied by rectification or subsequent delivery („subsequent performance“). In all other respects, the provisions in the General Terms and Conditions of Saljol, available at www.saljol.de/en/general-terms-and-conditions/, shall apply.

13. Disposal

Dispose of the materials separately by type in the designated collection systems. Please dispose of hazardous waste in the designated collection systems.

Electric motors, batteries and power supply units are hazardous waste, DO NOT dispose of in household waste!

13.1 Destruction/recycling

Before the chair can be taken away, it must be decommissioned. This is described in chapter 8 Decommissioning.

Dismantle the chair for transport:

1. move the backrest to an upright position.
2. stand behind the chair.
3. pull firmly on the backrest until it disengages from the bracket.
4. carefully lift the backrest and place it next to the chair.

Follow the transport instructions in chapter 5.

Important!

- Separate all parts and operating materials.
- Disposal must be carried out in accordance with local regulations and guidelines.

Note!

The motor and power supply unit are hazardous waste. If you have any questions about destruction and recycling, please contact the manufacturer directly.

Electric motors, batteries and power supply units are hazardous waste, DO NOT dispose of in household waste

14. Accessories

Swivel table

This table is securely mounted under the chair and serves as an ideal surface and support for standing up. The robust swivel table follows the movements of the chair and is available in light and dark wood finishes.

Head pillow

The removable head pillow easily sits on the backrest and can be individually adjusted to fit the neck. Available in Platinum Grey and Graphite Grey.

Incontinence pad set

The absorbent pad for the seat can hold up to 500 ml of liquid and comes in a pack of two. It can be quickly and easily replaced and washed at up to 40°C. Available in Platinum Grey and Graphite Grey.

Lumbar pillow

The pillow supports the lumbar spine and is also suitable for cushioning the feet and heels as well as adjusting seat widths. Available in Platinum Grey and Graphite Grey.

Tablet holder

A practical holder for your phone or tablet with a swivel arm for optimal alignment. Devices up to 18 cm can be secured (phones vertically, tablets horizontally).

Leatherette cushion set

The disinfectable leatherette cushion set offers a hygienic and easy-care solution for your chair. The cushions are made from high-quality leatherette and can be treated with household surface disinfectants. Ideal for environments where cleanliness and disinfection are of utmost importance.

Arm cushion narrow

The arm cushions narrow are perfect for users who desire more space and comfort without sacrificing the support of the armrests. Available in Platinum Grey and Graphite Grey.

Rechargeable battery

With the rechargeable battery, the chair can be operated as usual without needing to be plugged in. This eliminates the hassle of cables that could become a tripping hazard.

Note

You can order the accessories from our online shop at www.saljol.de around the clock or by phone through our sales and service team (Mon.–Fri. 9am–4pm) at +49 (0)8141 317740, all from the comfort of your home. Available accessories may vary depending on the country.

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