Saljol Wohnraumrollator

Bedienungsanleitung | Instruction manual | Manuel d'utilisation
Istruzioni per l'uso | Gebruiksaanwijzing | Manual de instrucciones

DE

EN

FR

IT

NL

ES

Vorwort

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www.saljol.de

Introduction

Thank you for purchasing an Indoor Rollator from Saljol! Your stylish new rollator will enable you to lead a mobile and self-determined life. The Saljol Indoor Rollator has been developed with safety in mind and will provide valuable assistance in your everyday life.

We would ask you to read the Instruction Manual carefully before using your indoor rollator for the first time. It contains important safety instructions and useful tips on use and care.

Contents

1. Identification 15

2. General notes..... 15

3. Notes..... 16

 3.1 Safety instructions..... 16

 3.2 Information for safe use of the product 16

4. Scope of delivery and product overview..... 17

 4.1 Scope of delivery..... 17

 4.2 Marking..... 17

 4.3 Product overview 17

5. Operation 18

 5.1 Inserting the seat board 18

 5.2 Grip and support positions for walking and standing in the rollator..... 18

 5.3 Grip and support positions for walking and standing outside the rollator..... 18

 5.4 Standing up and sitting down 18

 5.5 Brake system 19

 5.5.1 Braking and engaging the brakes with the brake levers..... 19

 5.5.2 Braking and engaging the brakes with the brake bar..... 19

 5.5.3 Disassembling/assembling the brake bar 19

 5.5.4 Adjusting and checking the brake system 19

 5.6 Locking the direction..... 20

 5.7 Shuffling with the rollator 20

6. Maintenance and reuse..... 20

 6.1 Maintenance..... 20

 6.2 Maintenance schedule 21

 6.2.1 Daily visual inspection..... 21

 6.2.2 Functional tests..... 21

 6.2.3 Cleaning 21

 6.2.4 Disinfection..... 22

 6.3 Reuse..... 22

7. Storage and shipping 22

8. Disposal..... 22

9. Technical data and list of materials..... 22

 9.1 Technical data 22

 9.2 Materials used..... 23

10. Warranty and guarantee 23

 10.1 Warranty..... 23

 10.2 Guarantee..... 23

11. Note on the obligation to report incidents..... 24

1. Identification

Manufacturer: Saljol GmbH

Product: Indoor Rollator

Type: Indoor rollator, walking frame, serving trolley and commode chair

Article name: Indoor Rollator

Version: 2025

Manufacturer: Saljol GmbH, Frauenstraße 32, 82216 Maisach

Phone: +49 8141 317740

Fax: +49 8141 3177411

Mail: hallo@saljol.de, Web: www.saljol.de

Authorised Swiss representative and importer:

CH REP

Rehabilitations-Systeme AG
Roggenstrasse 3, CH-4665 Oftringen

Formalities of the Instruction Manual

Version/revision: 2026.01

Creation date: 20/04/2026

Information was current at the time of going to press. We reserve the right to make technical changes. Technical drawings and photos may differ from reality.

©2026 Saljol GmbH: all rights reserved. Reprinting, including extracts, is only permitted with the authorisation of Saljol GmbH.

2. General notes

Intended use

The Indoor Rollator by Saljol has been developed for indoor use and is only suitable for use on firm ground. With the help of the Indoor Rollator by Saljol, people can stand and walk again and the user can rest on the integrated seat and move around in a shuffling gait. The Saljol Indoor Rollator allows users to perform everyday routine tasks without having to rely on assistance from others. Applications: The Indoor Rollator by Saljol is designed to cover a wide range of health problems It was conceived specifically for people who require additional support to maintain stability when walking.

Indication

The Indoor Rollator from Saljol is intended exclusively for use as a walking aid and is particularly suitable for patients with limited mobility due to physical impairments, limitations of the active or passive musculoskeletal system and illnesses such as arthritis, osteoporosis, muscle weakness, stroke or Parkinson's disease.

Contraindication

The Indoor Rollator from Saljol is not suitable for persons with very limited arm strength, significant cognitive impairments or very poor balance. Use and potential scenarios for the Saljol Indoor Rollator should be assessed by a doctor or therapist and instruction must be provided.

Service life

The Indoor Rollator has an expected service life of up to 4 years if used as intended, regular care and maintenance is provided by a specialist company and it is operated in compliance with the safety instructions. The rollator can remain in use after 4 years if it is still in good condition.

Responsibility

As the manufacturer, we can only consider ourselves responsible for the safety, reliability and suitability for use of the rollator if modifications, extensions, repairs and maintenance work are performed regularly and only by authorised specialist personnel and the rollator is used as intended in accordance with all instructions for use.

Declaration of Conformity

CE

As the manufacturer of the product, Saljol declares that the Indoor Rollator fully complies with the requirements of the Medical Devices Regulation (2017/745) (as of 2022). A detailed CE Declaration of Conformity can be found on our homepage at www.saljol.de.

The manufacturer/authorised representative declares that the following products with the product designation Indoor Rollator, version: 2025.

Saljol Indoor Rollator WR43	Velvet Grey	4058789015834
Saljol Indoor Rollator WR50	Velvet Grey	4058789015803
Saljol Indoor Rollator WR53	Velvet Grey	4058789015810
Saljol Indoor Rollator WR56	Velvet Grey	4058789015827
Saljol Indoor Rollator WR43	Champagne Gold	4058789015872
Saljol Indoor Rollator WR50	Champagne Gold	4058789015841
Saljol Indoor Rollator WR53	Champagne Gold	4058789015858
Saljol Indoor Rollator WR56	Champagne Gold	4058789015865

comply with the harmonised standards as listed hereafter: EN ISO 11199-2:2021. – Walking aids manipulated by both arms – Requirements and test methods – Part 2: Rollators; seat and handles meet the biological assessment according to ISO 10993-5:2009-10

3. Notes

3.1 Safety instructions

The Instruction Manual contains safety notes that are marked twice, namely by a symbol and by a signal word. The symbol informs you of the type of notice. The signal word explains more details, for example the severity of an impending risk.

Caution/important

Possibly dangerous situation (minor injuries or material damage), commitment to a particular behaviour or activity for the safe use of the rollator

Note

Application tips and particularly useful information

3.2 Information for safe use of the product

To ensure safe use of the product, please keep the following in mind:

Restrictions on use

- The maximum user weight must not exceed 150 kg for WR50, WR53 and WR56 and 120 kg for WR43.
- The rollator was developed for indoor use only.
- Never use in the presence of flammable/combustible and burning substances.

Accident prevention

- Only use the rollator for its defined purpose.
- Only use the rollator if it is in a flawless technical condition.
- It is prohibited to make any constructive changes to the rollator, unless you have the written consent of the manufacturer, Saljol.
- Be careful not to pinch your fingers or other body parts on the moving parts when adjusting the rollator.
- Make sure that the parking brake is engaged when you sit down.
- Make sure that there is no direct skin contact with the seat board when using this product. The thighs and buttocks should always be covered with clothing to prevent bruising.
- Make sure that the seat board does not restrict legroom when walking. The seat board should be placed upright in the rollator.
- All four wheels must be in contact with the ground at all times. This is the only way to balance the rollator safely and prevent accidents.
- Always engage the parking brake when you stop or park the rollator.
- Do not use the product to transport people. The product is not designed to carry people while it is in motion. Improper use can lead to dangerous situations, loss of control or falls.

4. Scope of delivery and product overview

4.1 Scope of delivery


The features of our products may vary depending on the country and variant. This is in order to meet local requirements. The scope of delivery may therefore differ.

- 1 Saljol Indoor Rollator
- 1 seat board with felt pad
- 1 tray with felt pad*
- 1 small bag*
- 1 cork handrail padding*
- 1 brake bar
- 1 Instruction Manual

*May not be included in the scope of delivery, depending on the variant

4.2 Marking

The product label and the UDI-DI label are located on the lower frame between the wheels on the left and right, respectively. The UDI-DI label (Unique Device Identification – Device Identifier) is added to the product marking for compliance with regulatory requirements. They contain important information about the product: manufacturer’s details about Saljol, production date, maximum user weight, product height, seat height, product width, product name, serial number and UDI-DI. The labels must not be removed. The following symbols are found on the labelling of the Indoor Rollator and on the packaging – sometimes together with other information – and indicate important features.

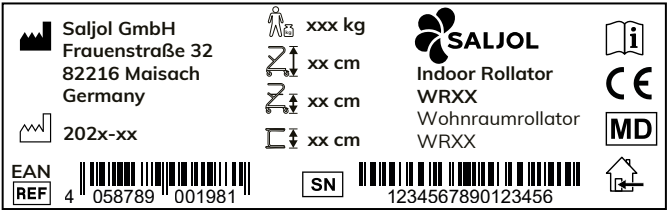	Manufacturer of the product
	Production date
	Order/item number
	Maximum user weight
	Product height
	Seat height
	Product width
	Serial number
	Observe the Instruction Manual
	CE mark
	Medical device
	Product only suitable for indoor use and even floors
	UDI-DI (Unique Device Identification – Device Identifier)

4.3 Product overview

Saljol Indoor Rollator*

- 1 Brake lever
- 2 Small bag
- 3 Seat board with felt pad
- 4 Tray with felt pad
- 5 Cork handrail pads
- 6 Brake bar
- 7 Pedal lever for directional locking

*Tray with felt pad, small bag and cork handrail pads are not standard in every country

5. Operation

Inventory

Remove the rollator out of the box and check that none of the parts are missing. Compare the parts with the illustration in 4.3 and the scope of delivery described in 4.1. Contact your specialist dealer or Saljol immediately if parts are missing or components are damaged.

Important

No special technical skills are required for these adjustments. Always make sure that the brakes are engaged (see 5.5) before you sit down or park the rollator.

5.1 Inserting the seat board

Important

- Check that the seat board is firmly in place before you sit down.
- Never lean on the surface of the seat board marked in the picture, as it may otherwise fold downwards.


5.2 Grip and support positions for walking and standing in the rollator

5.3 Grip and support positions for walking and standing outside the rollator

5.4 Standing up and sitting down

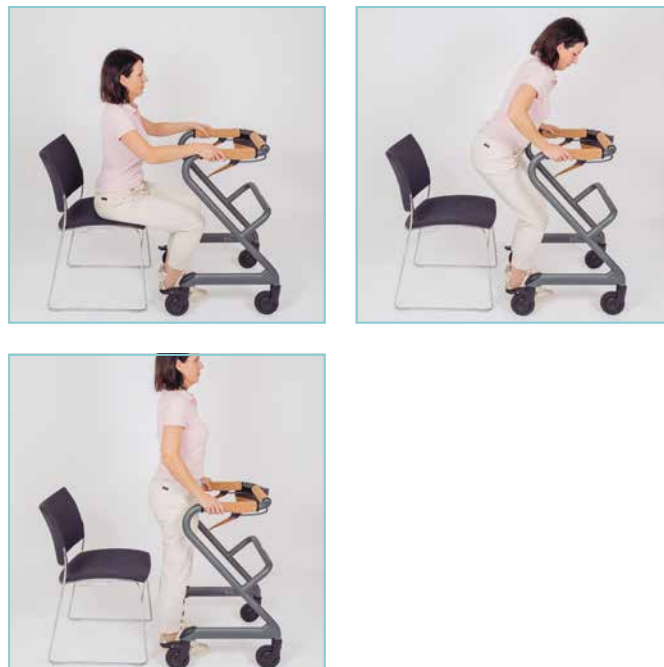5.5 Brake system

Note

The two-handed brake and one-hand brake belong to the same brake system, so the brake bar in the one-hand brake also moves when the brake levers are pressed. Using the brake bar will move the two brake levers as well.

5.5.1 Braking and engaging the brakes with the brake levers

Braking

- Pull the brake levers upwards to engage the brake.
- Keep the brake levers pulled to stand safely in the rollator.
- Simply let go of the brake levers to release the brake.

Engaging and releasing the brake


- To engage the parking brake, press the brake levers downwards until you have overcome a slight resistance. Both rear wheels are now locked.
- Simply pull the brake levers upwards again to release the parking brake.

5.5.2 Braking and engaging the brakes with the brake bar

Braking

- To brake, pull any part of the brake bar upwards towards the frame.
- Both rear wheels are braked simultaneously.
- Keep the brake bar engaged to stand safely in the rollator.
- Simply let go of the brake bar to release the brake.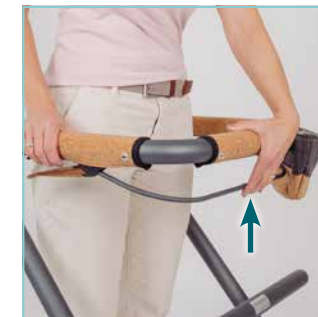

Engaging and releasing the brake

- To engage the parking brake, pull the brake bar upwards until you have overcome a slight resistance. Both rear wheels are now locked.
- Press the brake bar downwards to release the parking brake.

5.5.3 Disassembling/assembling the brake bar

Disassembly

- Turn the Indoor Rollator over and place it on the handrail.
- Next unscrew the screws on both handles using a size 3 mm Allen key.
- You can now remove the brake bar.

Keep the brake bar together with the screws in case you want to reuse the bar.

Assembly

To refit the brake bar, insert the bar into the matching holes (see picture above) and tighten both screws on the brake bar.


5.5.4 Adjusting and checking the brake system

Preparation

- Turn the Indoor Rollator over and place it on the handrail.
- Keep a size 4 mm Allen key, a size 10 open-end spanner and a ratchet with a size 10 socket ready.
- If fitted, first remove the brake bar as described in section 5.5.3.

Adjusting and checking

- Use the screwdriver to loosen the screw on the brake lever and pull out the brake housing
- Use the open-end spanner to loosen the lock nut.
- Unscrew the ferrule using the ratchet with a 10 mm socket (brake becomes tighter) or screw it in (brake becomes smoother)
- Check the braking force
- Repeat points 3-4 until you are happy with the adjustments
- Tighten the lock nut
- Insert the brake housing and tighten the screw
- Reassemble the brake bar, if preferred

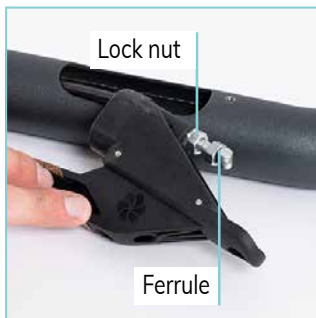You must press the central locking lever on both sides again to release the directional lock.

5.7 Shuffling with the rollator

It is permitted to shuffle around the home with the Saljol Indoor Rollator. Here is how you can move safely around your home:

- check that the seat board is firmly in place
- engage the brake
- sit down
- release the brake

Now you can shuffle through the rooms by pulling yourself forward with your feet.

6. Maintenance and reuse

6.1 Maintenance

Although the Saljol Indoor Rollator is generally robust and durable thanks to its corrosion-resistant materials, it still requires regular cleaning and maintenance. We recommend that you regularly perform the checks stated in the maintenance schedule to ensure that your rollator can be operated safely at all times. Inadequate or neglected care and maintenance of the rollator may impair its safety and limit liability. Bear in mind, however, that the maintenance schedule does not contain precise details of the actual scope of work required on the rollator. It is therefore advisable to consider any problems or additional maintenance requirements that you notice on the rollator and to seek professional assistance if necessary. You can ensure that your rollator is always in a flawless condition and provides you with safe support by performing the recommended maintenance regularly and paying attention to any deviations. Saljol reserves the right to limit warranty claims if the rollator has not been regularly maintained.

6.2 Maintenance schedule

6.2.1 Daily visual inspection

A daily visual inspection of the rollator should be performed prior to each use. Doing so helps to check main components (frame, brakes, handles, seat and wheels) for any damage, signs of wear or other problems before they become major issues. Keep an eye on changes or missing elements and, in cases of doubt, ask your specialist dealer, Saljol sales partner or Saljol GmbH.

6.2.2 Functional tests

Functional tests help to ensure that all rollator functions are working properly. Any tested component that is not in good working order must be repaired immediately either by the user or specialised personnel. We recommend performing functional tests on the Indoor Rollator at least once a month, ideally even twice. The functional tests include:

- Brake test: Check the rollator's brakes to make sure that they are in good working order. Make sure that the rollator can be brought to a standstill quickly and safely on both sides if necessary. The parking brake should lock both rear wheels and prevent the rollator from rolling away on an inclined/uneven surface.
- Tyre test: Check the rollator's tyres to make sure that they turn easily and show no signs of excessive wear or damage. Defective tyres must be replaced immediately. Make sure that all wheels are in contact with the ground at the same time.

6.2.3 Cleaning

The entire rollator must be cleaned at least once a month, depending on the degree of soiling. Coarse dirt can also be removed with warm water and a sponge. A commercially available cleaning agent can be added to the water to deal with stubborn dirt. Disinfectants can be used without any problems. Plastic parts are best cleaned with a commercially available plastics cleaning agent. Please observe the cleaning agent's instructions for use.

Please also bear in mind the following information on individual product parts!

Important

- Never use aggressive cleaning agents such as solvents/scouring agents or hard brushes!
- Never clean the rollator with a high-pressure/steam cleaner!

Frame

Cleaning can be performed with warm water and a soft sponge. A commercially available cleaning agent can be added to the water to deal with heavy soiling. Disinfectants can be used.

Plastic parts are best cleaned and cared for with a commercially available plastics cleaning agent. Always follow the instructions for use of the respective cleaning agent.

Wheels

Clean the wheels with warm water and a mild cleaning agent. Use a plastic brush if required, but never a wire one.

Cork handrail pads/Cork-TPE

Dust and other particles can accumulate on the surface of cork products during storage and use, which may cause a so-called 'grey haze'. This is not a product defect and can be easily removed.

Correct care and cleaning of cork elements helps to extend the service life of the material. Remove loose dirt with a soft, dry cloth or a soft brush. Heavier soiling can be removed with a well-wrung, soft cloth and a mild soap solution.

Avoid vigorous rubbing and the use of abrasive cleaning agents or scouring pads, as you may otherwise damage the cork surface.

To maintain the service life of the cork elements, clean the Surfaces regularly and protect them from direct sunlight, heat sources and moisture.

Please note that cork is a natural material and can change colour over time when exposed to light and humidity.

Tray/Seat board with felt pad

Remove loose dirt with a soft, dry cloth or a soft brush. Heavier soiling can be removed with a well-wrung, soft cloth and a mild soap solution.

Avoid vigorous rubbing and the use of abrasive cleaning agents or scouring pads.

Important

The brake has been correctly adjusted if the rear wheels lock when the brake is engaged and do not turn when pushed.

Note

Here you will find a video with a step-by-step guide to adjusting the brake. Simply scan the QR code.

5.6 Locking the direction

To lock the direction of the rear wheels, press down the pedal levers on both sides until you hear a soft 'click' sound. The wheels lock automatically as soon as you move straight ahead.

6.2.4 Disinfection

Commercially available household disinfectants can be used to disinfect the rollator. For an overview of the disinfectants approved by the Robert Koch Institute, visit <https://www.rki.de/>. After disinfection, allow the rollator to dry in the air and observe the exposure time specified for the disinfectant used.

6.3 Reuse

The Saljol Indoor Rollator is suitable for reuse. The number of possible reuses depends on the condition, material wear and functional suitability of the product. You must clean and disinfect the rollator prior to each reuse. Remove all visible dirt completely and use a suitable disinfectant to ensure sterility. Then perform all the checks described in section 6.2 Maintenance schedule to ensure that the rollator is working properly.

The rollator is resistant to the cleaning agents and disinfectants applied for reuse.

7. Storage and shipping

The rollator must be stored upright, dry and out of the reach of children or third parties when not in use. The original packaging should be kept and used for storage or shipping in order to protect the rollator from environmental influences.

8. Disposal

Dispose of the materials separately according to type and in the designated collection systems. Please dispose of hazardous waste in the designated collection systems.

9. Technical data and list of materials

9.1 Technical data

Indoor Rollator (WR)	WR43	WR50	WR53	WR56
Grip height	78 cm	80 cm	86 cm	92 cm
Seat height	43 cm	50 cm	53 cm	56 cm
Recommended for body height	135–155 cm	145–165 cm	150–180 cm	170–200 cm
Max. user weight	120 kg	150 kg	150 kg	150 kg
Length	61 cm	61 cm	61 cm	61 cm
Lower width	56 cm	56 cm	56 cm	56 cm
Upper width	51 cm	51 cm	51 cm	51 cm
Distance between the grips	43 cm	43 cm	43 cm	43 cm
Weight without accessories	9.4 kg	9.6 kg	9.8 kg	10.0 kg
Max. payload of the bag	0.5 kg	0.5 kg	0.5 kg	0.5 kg
Wheels (diameter x width)	12.5 x 3 cm, grey, non-slip TPE wheels			
Turning radius	36 cm	36 cm	36 cm	36 cm

9.2 Materials used

Frame	Steel Q235
Brake bar	Aluminium 6061-T6
Brake lever	Thermoplastic elastomers (TPE) with 20-30% cork
Braking device	Plastic – polyamide
Seat board	Birch
Tray	Birch
Felt pad	90% wool + 10% polyester
Bag	Polyester + cork
Cork handrail pads	Polyester + cork
Rims	Plastic – polyamide
Wheels	Thermoplastic polyurethane (TPU)

10. Warranty and guarantee

10.1 Warranty

If the purchased item is defective, the provisions of statutory liability for defects shall apply. If you have purchased the goods from a specialist retailer, please contact your seller immediately; if you purchased the goods directly from Saljol, please contact Saljol directly on +49 8141 317740. Recognisable defects must be reported in writing within ten days of delivery, concealed defects within ten days of their discovery. Liability will not be accepted after these deadlines. Saljol shall be entitled to decide whether any defect shall be remedied by rectification or subsequent delivery ("subsequent fulfilment"). In all other respects, the provisions of Saljol's Terms and Conditions apply, which are available at <https://www.saljol.de/de/agb/>.

10.2 Guarantee

In addition to the statutory warranty claims and without limiting these, Saljol extends a guarantee of 2 years from the date of purchasing this product. The option is also available for you to extend this guarantee by a further 2 years if you register the rollator online no later than 4 weeks after the date of purchase. You can register at <https://www.saljol.de/de/garantie/>.

Scope of the guarantee

We guarantee that the product will be free from defects in material, workmanship and function within the guarantee period. If a defect occurs during the guarantee period, we will, at our discretion, repair the product free of charge or, if necessary, replace it with a new one. Wear parts such as the brake mechanism and tyres and the enclosed accessories (small bag, cork handrail pad, tray with felt pad and seat board with felt pad) are not covered by the guarantee. These guarantee conditions only apply to products purchased in Germany and only if they were purchased directly from Saljol or from an authorised specialist dealer. The seller will provide information on the guarantee conditions for products purchased abroad. Guarantee services and replacement deliveries for guarantee reasons do not extend the warranty period.

Exclusion of the guarantee

The guarantee becomes void if the rollator has been maintained or repaired improperly or by an untrained person or if the product has been operated with non-original accessories. Incorrect assembly or installation due to non-compliance with the Instruction Manual, unintended use or improper use or care of the product will invalidate the guarantee, as will external influences (e.g. transport damage, damage caused by knocks and impacts, weather influences).

11. Note on the obligation to report incidents

Please note that all serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority. Serious incidents within the meaning of the law are all adverse events that have one of the following consequences: Death, life-threatening injury, permanent physical damage and chronic illness.

If you are in the Federal Republic of Germany, please notify:

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If you are in the EU Member States, please notify:

- Your specialist dealer, Saljol sales partner or Saljol GmbH, Frauenstr. 32, D-82216 Maisach; www.saljol.de
- the competent authority for incidents involving medical devices in your country

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Bedienungsanleitung | Instruction manual | Manuel d'utilisation
Istruzioni per l'uso | Gebruiksaanwijzing | Manual de instrucciones

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