Saljol Carbon Rollator

Bedienungsanleitung | Instruction manual | Manuel d'utilisation
Istruzioni per l'uso | Gebruiksaanwijzing | Manual de instrucciones

DE

EN

FR

IT

NL

ES

Vorwort

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www.saljol.de

Introduction

Thank you for purchasing a Carbon Rollator from Saljol! Your stylish new rollator will enable you to lead a mobile and self-determined life. The Carbon Rollator has been developed with safety in mind and will provide valuable assistance in your everyday life.

We would ask you to read the Instruction Manual carefully before using your Carbon Rollator for the first time. It contains important safety instructions and useful tips on use and care.

EN

Contents

1.	Identification	16
2.	General notes.....	16
3.	Notes.....	17
3.1	Safety instructions.....	17
3.2	Information for safe use of the product	17
4.	Scope of delivery and product overview.....	18
4.1	Scope of delivery	18
4.2	Marking/labelling.....	18
4.3	Product overview	19
5.	Operation	19
5.1	Unfolding the rollator.....	19
5.2	Folding the rollator	20
5.3	Adjusting the correct handle height/support height.....	20
5.4	Using the handbrake	20
5.5	Using the parking brake	20
5.6	Adjusting the brake.....	20
5.7	Using the tilting aid	21
5.8	Using the friction brake.....	21
5.9	Attaching and removing the bag	21
6.	Maintenance and reuse.....	22
6.1	Maintenance	22
6.2	Maintenance schedule	22
6.2.1	Daily visual inspection.....	22
6.2.2	Functional tests.....	22
6.2.3	Cleaning	22
6.2.4	Disinfection.....	23
6.3	Reuse.....	23
7.	Transport description.....	23
8.	Storage and shipping	23
9.	Disposal.....	23
10.	Technical data and list of materials.....	24
10.1	Technical data	24
10.2	Materials used.....	24
11.	Warranty and guarantee	25
11.1	Warranty.....	25
11.2	Guarantee.....	25
12.	Note on the obligation to report incidents.....	25

1. Identification

Manufacturer: Saljol GmbH
Product: Carbon Rollator
Type: Lightweight rollator
Item description: Carbon Rollator
Version: 2026

Manufacturer:
Saljol GmbH, Frauenstraße 32, 82216 Maisach
Phone: +49 8141 317740
Fax: +49 8141 3177411
Mail: hallo@saljol.de, Web: www.saljol.de

Authorised Swiss representative and importer:

CH REP

 Rehabilitations-Systeme AG
Roggenstrasse 3, CH-4665 Oftringen

Formalities of the Instruction Manual
Version/revision: 2026.01
Creation date: 11/03/2026
Information was current at the time of going to press.
Technical changes reserved. Technical drawings and photos may differ from reality.
©2026 Saljol GmbH: all rights reserved. Reprinting, even in extracts, is only permitted with the authorisation of Saljol GmbH.

2. General notes

Intended use
The Carbon Rollator from Saljol was developed for indoors and outdoors and is only suitable for use on firm and even ground. The Carbon Rollator from Saljol helps people to stand and walk again. Users can also rest on the integrated seat. Users can perform everyday routine tasks with the Saljol Carbon Rollator, without being dependent on outside assistance. Applications: The Carbon Rollator from Saljol is designed to cover a wide range of health issues. It was conceived specifically for people who require additional support to maintain stability when walking.

Indication
The Carbon Rollator from Saljol is intended exclusively for use as a walking aid and is particularly suitable for patients with limited mobility due to physical impairments, limitations of the active or passive musculoskeletal system and illnesses such as arthritis, osteoporosis, muscle weakness, stroke or Parkinson's disease.

Contraindication
The Carbon Rollator from Saljol is not suitable for persons with very limited arm strength, significant cognitive impairments or very poor balance. The use and possible applications of the Saljol Carbon Rollator should be assessed by a doctor or therapist and instruction must be provided.

Service life
The Carbon Rollator has an expected service life of up to 4 years if used as intended, regular care and maintenance is provided by a specialist company and it is operated in compliance with the safety instructions. The rollator can remain in use if it is still in good condition after 4 years.

Responsibility
As the manufacturer, we can only consider ourselves responsible for the safety, reliability and suitability for use of the rollator if modifications, extensions, repairs and maintenance work are performed regularly and only by authorised specialist personnel and the rollator is used as intended in accordance with all instructions for use.

Declaration of Conformity

CE

As the manufacturer of the product, Saljol declares that the Carbon Rollator fully complies with the requirements of the Medical Devices Regulation (2017/745) (as of 2022). A detailed CE Declaration of Conformity can be found on our homepage at www.saljol.de.

The manufacturer/authorised representative declares that the following products with the product designation Carbon Rollator, version: 2026

Saljol Carbon Rollator CR62	British Racing Green	4058789016589
Saljol Carbon Rollator CR62	Midnight Blue	4058789016626
Saljol Carbon Rollator CR62	Copper Red	4058789016602
Saljol Carbon Rollator CR62	Pure Gold	4058789016640
Saljol Carbon Rollator CR54	British Racing Green	4058789016596
Saljol Carbon Rollator CR54	Midnight Blue	4058789016633
Saljol Carbon Rollator CR54	Copper Red	4058789016619
Saljol Carbon Rollator CR54	Pure Gold	4058789016657
Saljol Carbon Rollator CR62 XL	British Racing Green	4058789016671

comply with the harmonised standards as listed hereafter: EN ISO 11199-2:2025 – Walking aids manipulated by both arms – Requirements and test methods – Part 2: Rollators; seat and handles meet the biological assessment according to ISO 10993-5:2009-10

3. Notes

3.1 Safety instructions

The Instruction Manual contains safety notes that are marked twice, namely by a symbol and by a signal word. The symbol informs you of the type of notice. The signal word explains more details, for example the severity of an impending risk.

Caution/important

Possibly dangerous situation (minor injuries or material damage), commitment to a particular behaviour or activity for the safe use of the rollator

Note

Application tips and particularly useful information

3.2 Information for safe use of the product

To ensure safe use of the product, please keep the following in mind:

Restrictions on use

- The maximum user weight for the CR54 and CR62 must not exceed 150 kg. The maximum user weight for the CR62 XL must not exceed 180 kg.
- The maximum payload of the bag is 10 kg.
- It is not permitted use the rollator to move in small, rapid steps.
- The rollator was developed for both indoor and outdoor use.
- Never use in the presence of flammable/combustible and burning substances.

Caution

Fingers or other parts of the body of the user and/or a third party may be pinched when folding the rollator or adjusting the handle height. You must therefore remain extremely attentive and careful when adjusting the device. Be aware of gaps in the rollator to avoid trapping your fingers and other body parts.

Accident prevention

- Remove all cork-coloured labels before using the rollator.
- Only use the rollator for its defined purpose.
- Only use the rollator if it is in a flawless technical condition.
- It is prohibited to make any constructive changes to the rollator, unless you have the written consent of the manufacturer, Saljol.
- While adjusting the rollator, be careful not to pinch your fingers or other body parts on the moving parts.
- Always ensure that the back strap is fully inserted into the holder at all times.
- Make sure that both parking brakes are engaged when you sit down.
- Note that on sloping terrain, the rolling speed of the Saljol Carbon Rollator may be faster than your walking speed. There is an increased risk of falling. Note the use of the friction brake.

- Always be careful when crossing thresholds, rail-way tracks, gully covers with wide slits and similar. A wheel may become caught or trapped in them, stop abruptly and cause a fall.
- All four wheels must be in contact with the ground at all times. This is the only way to balance the rol-lator safely and prevent accidents.
- Be careful to walk carefully on icy ground and over small obstacles such as branchesso that you can react quickly.
- Always engage both parking brakes when you stop or park the rollator.
- Do not use the product to transport people. The product is not designed to carry persons or to be sat on while it is moving. Improper use may lead to dangerous situations, loss of control, or falls.

4. Scope of delivery and product overview

4.1 Scope of delivery

The features of our products may vary depending on the country and variant in order to meet local require-ments. The scope of delivery may therefore differ.

- 1 Saljol Carbon Rollator
- 1 bag with Fidlock magnetic fastening*
- 1 back strap*
- 1 cane holder
- 1 LED light*
- 1 bell*
- 1 care cloth*
- 1 Instruction Manual

*May not be included in the scope of delivery, depend-ing on the variant


Instructions for using the back strap, bell and LED light are found in the enclosed Instruction Manual in each case.

4.2 Marking/labelling

The product label is affixed to the inside of the left frame section. The UDI-DI label is located on the right and adds to the product labelling for compliance with regulatory requirements.

They contain important information about the prod-uct: EAN code, product name, production date, prod-uct colour and weight, product dimensions, maximum user weight, maximum load weight of the bag, seat height, manufacturer information on Saljol, serial number and UDI-DI (Unique Device Identification – Device Identifier). The labels must not be removed. The following symbols are found on the labelling of the Carbon Rollator and on the packaging – some-times together with other information – and indicate important features.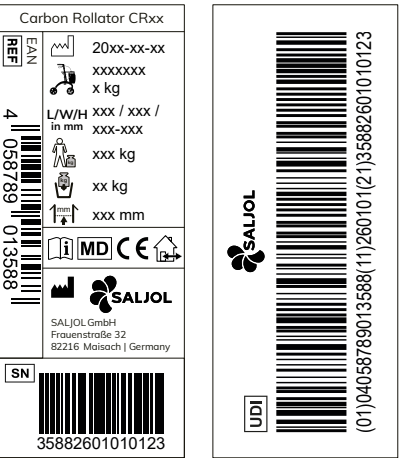

	Order/item number
	Production date
	Colour and total weight
	Length/width/height of the rollator in mm
	Maximum user weight
	Maximum payload of the bag
	Seat height
	Observe the Instruction Manual
	Medical device
	CE mark
	Suitable for indoors and outdoors
	Manufacturer of the product
	Serial number
	UDI-DI (Unique Device Identification – Device Identifier)

4.3 Product overview

Saljol Carbon Rollator*

- 1 Back strap
- 2 Push handle and armrest
- 3 Brake lever
- 4 Bell
- 5 Bowden cable
- 6 Seat net (CR54/CR62)/ Fabric seat (CR62 XL) with release strap
- 7 Handle bar
- 8 Hand screw
- 9 Brake pad
- 10 Rear wheel
- 11 Front wheel
- 12 Cane holder
- 13 Locking hook
- 14 Bag
- 15 LED light
- 16 Friction brake
- 17 Tipping aid

*Back strap, bell and LED light are not standard features in every country

5. Operation

Inventory

Remove the rollator out of the box and check that none of the parts are missing. Compare the parts with the illustration in 4.3 and the scope of delivery de-scribed in 4.1. Contact your specialist dealer or Saljol immediately if parts are missing or components are damaged.

- No special technical knowledge is required for these adjustments. Before each use, make sure that
- the locking hook is engaged when the rollator is folded,
 - the rollator is locked into place when unfolded and cannot simply be folded up,
 - the brakes are always engaged (see 5.5) before you sit down or park the rollator,

- the handle bars are tightened with the hand screw,
 - the handles are set to the correct height for you and are at the same level,
 - all accessories are firmly attached.
- A failure to observe these instructions may result in injuries.

5.1 Unfolding the rollator

- Open the locking hook.
- Grasp the rollator by its handles and push or pull them apart at the same time.
- Finally, press both seat tubes down with your hands until you hear a 'click' sound.

5.2 Folding the rollator

- Gently tug the seat's release strap upwards.
- Grasp the handles and push the rollator together until both seat tubes are close together.
- Engage the locking hook to secure the folded rollator in place.

5.3 Adjusting the correct handle height/support height

- Engage the parking brake (see 5.5).
- Then open the hand screws.
- Stand in the rollator and let your arms hang loosely so that they are slightly bent. The height of your wrists corresponds to the correct height of the push handles.
- Unscrew the hand screws and adjust the handles to the same height using the height scale on the inside of the handle bars.
- Then tighten both hand screws by hand.

5.4 Using the handbrake

- Pull the brake levers upwards to engage the brake.
- Keep the brake levers pulled to stand safely in the rollator.
- Simply let go of the brake levers to release the brake.

5.5 Using the parking brake

- To engage the parking brake, press the brake levers downwards until you hear a 'click' sound.
- Simply pull the brake levers upwards again to release the parking brake.

Important

As a rule during use, it is still important that you

- engage the brakes when you are not moving the rollator.
- always engage the parking brakes when you want to park the rollator or sit down.
- use the hand brakes on sloping surfaces to feel safer and minimise the risk of falling.
- sit with your back to the shopping bag, not on the bag.
- sit upright on the seat net, while placing your forearms comfortably on the push handles and armrests.

Information

Is the use of one of your hands impaired due to a stroke, osteoarthritis or similar? If so, we recommend converting the rollator to a one-hand brake. You can obtain the brake cable and all information on the conversion from us free of charge. Kindly contact your medical supply shop or the Saljol service team by email: kundenservice@saljol.de.



5.6 Adjusting the brake

The brake requires adjustment if the braking force decreases due to wheel wear. The brakes should be adjusted so that both wheels brake evenly. The gap between the brake pad and the wheel surface should be approx. 1 mm.

How to proceed:

Use a size 8 open-end spanner to loosen the lock nut and turn it completely upwards. Attach the ferrule at the same time.

Rotate the ferrule clockwise or anti-clockwise to adjust the braking force. Turn the wheels and pull the brake levers to check the brake function. The wheels should turn freely, but stop immediately when the brakes are applied. It must not be possible to move the wheels when the brake is engaged. Tighten the lock nut firmly again.

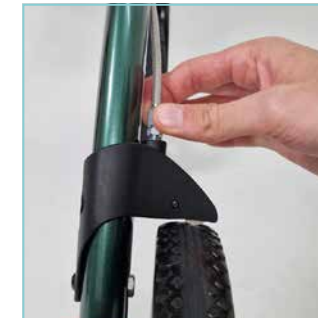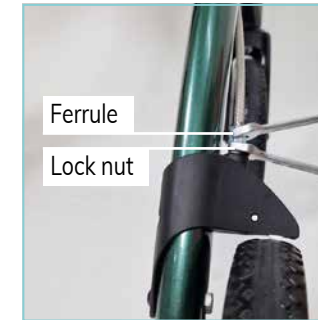Important

The brake is correctly adjusted if the rear wheels lock when the brake is applied and do not turn when pushing.

Information

Here you will find a video with a step-by-step guide to adjusting the brake. Simply scan the QR code.

5.7 Using the tipping aid

Step on the tipping aid with one foot and use the handles to pull the rollator slightly towards you. The front wheels of the rollator are now raised. This is how you overcome thresholds and kerbs.

5.8 Using the friction brake

The friction brake is recommended on gentle slopes or for people with a very restricted walking speed. Actuate the red toggle lever to engage the friction brake. Use the adjustment screw to set the rolling resistance.

5.9 Attaching and removing the bag

Important

- Stand behind the rollator. Make sure that the rollator is positioned on a level surface and that the parking brakes are engaged.
- You can hang both sides in place at the same time if your stance is secure and stable. Otherwise, support yourself with one hand on the rollator and attach one side first, then the other.

Attaching the bag

- Grasp the bag by the two straps.
- Guide the bag under the seat net or the fabric seat.
- Connect the attachment points to the designated fastening points on the frame.
- Apply pressure to the attachment points on the left and right until you hear and feel them click into place.
- Check that both sides are securely engaged.
- Make sure that the bag is hanging freely and that the brake function is not impaired.

Removing the bag

- Grasp the bag by the two straps.
- Pull the bag slightly upwards so that the attachment points are released from the fastening points.
- Do not pull the bag forwards. Doing so may damage the locking mechanism.
- Remove the bag completely from the holders..

6. Maintenance and reuse

6.1 Maintenance

Although the Saljol Carbon Rollator is generally robust and durable thanks to its corrosion-resistant materials, it still requires regular cleaning and maintenance.

We recommend that you regularly perform the checks stated in the maintenance schedule to ensure that your rollator can be operated safely at all times. Inadequate or neglected care and maintenance of the rollator may impair its safety and limit liability. Bear in mind, however, that the maintenance schedule does not contain precise details of the actual scope of work required on the rollator. It is therefore advisable to consider any problems or additional maintenance requirements that you notice on the rollator and to seek professional assistance if necessary. You can ensure that your rollator is always in a flawless condition and provides you with safe support by performing the recommended maintenance regularly and paying attention to any deviations.

Saljol reserves the right to limit warranty claims if the rollator has not been regularly maintained.

6.2 Maintenance schedule

6.2.1 Daily visual inspection

A daily visual inspection of the rollator should be performed prior to each use. Doing so helps to check main components (frame, brakes, handles, seat and wheels) for any damage, signs of wear or other problems before they become major issues. Keep an eye on changes or missing elements and, in cases of doubt, ask your specialist dealer, Saljol sales partner or Saljol GmbH.

6.2.2 Functional tests

Functional tests help to ensure that all rollator functions are working properly. Any tested component that is not in good working order must be repaired immediately either by the user or specialised personnel. We recommend performing functional tests on the Carbon Rollator at least once a month, ideally even twice. The functional tests include:

- **Brake test:** Check the rollator's brakes to make sure that they are in good working order. Make sure that the rollator can be brought to a standstill quickly and safely on both sides if necessary. The parking brake should lock both rear wheels and prevent the

rollator from rolling away on an inclined/uneven surface.

- **Tyre test:** Check the rollator's tyres to make sure that they turn easily and show no signs of excessive wear or damage. Defective tyres must be replaced immediately.
- **Stability test:** Check the rollator's connections to make sure that they move easily and show no signs of wear or damage. Connections that are too loose should be tightened immediately, connections that are too stiff should be loosened a little. Make sure that all wheels are in contact with the ground at the same time.
- **Folding test:** Perform out the folding process according to the manufacturer's instructions and make sure that all parts are correctly aligned and locked. The entire process should proceed smoothly.
- **Handle test:** Check the height adjustment of the handles to ensure that it is working correctly and that the rollator is set to the correct height for the user. The basic rule: The wrists must be level with the handles when standing upright.
- **Back strap test:** Regularly check the back strap and its fastening rail for visible damage such as tears, wear or loose seams. Make sure that the back strap is fully inserted into the holder at all times.

6.2.3 Cleaning

The entire rollator must be cleaned at least once a month, depending on the degree of soiling. To do so, use the care cloth included in the scope of delivery. Coarse dirt can be removed with warm water and a sponge. A commercially available cleaning agent can be added to the water to deal with stubborn dirt. Disinfectants can be used without any problems. Plastic parts are best cleaned with a commercially available plastics cleaning agent. Please observe the cleaning agent's instructions for use. Simple methods such as a Teflon spray can be used to stop squeaking or creaking of the folding mechanism, wheels or even the front fork.

Please also bear in mind the following information on individual product parts!

Important

- Never use aggressive cleaning agents such as solvents/scouring agents or hard brushes!
- Never clean the rollator with a high-pressure/steam cleaner!

Wheels

Clean the wheels with warm water and a mild solvent. Use a plastic brush if necessary, but never a wire one.

Cork TPE

Dust and other particles can accumulate on the surface of cork TPE products during storage and cause a so-called 'grey haze'. This is not a product defect and can be easily removed.

Correct care and cleaning of cork TPE-coated elements helps to extend the service life of the material. Remove loose dirt with a soft, dry cloth or a soft brush. Heavier soiling can be removed with a well-wrung, soft cloth and a mild soap solution. Avoid vigorous rubbing and the use of abrasive cleaning agents or scouring pads, as you may otherwise damage the cork TPE. To maintain the service life of the cork TPE, clean the surface regularly and protect it from direct sunlight, heat sources and moisture. Please note that cork is a natural material and can change colour over time when exposed to light and humidity.

6.2.4 Disinfection

Standard household disinfectants can be used to disinfect the rollator. For an overview of the disinfectants approved by the Robert Koch Institute, visit <https://www.rki.de/>.

After disinfection, allow the rollator to dry in the air and observe the exposure time specified for the disinfectant used.

6.3 Reuse

The Saljol Carbon Rollator is suitable for reuse. The number of possible reuses depends on the condition, material wear and functional suitability of the product.

You must clean and disinfect the rollator prior to each reuse. Remove all visible dirt completely and use a suitable disinfectant to ensure sterility.

Then perform all the checks described in section 6.2 Maintenance schedule to ensure that the rollator is working properly.

The rollator is resistant to the cleaning agents and disinfectants applied for reuse.

7. Transport description

The rollator is easy to fold together and therefore convenient to transport. Please follow the folding instructions (see section 5.2) if you plan to transport the rollator by car, for example. Make sure that the rollator is stowed away safely to prevent sudden and dangerous movements (e.g. braking when driving a car). The Saljol transport bag (available separately) protects your rollator from external influences and damage during transport.

8. Storage and shipping

The rollator must be stored upright, dry and out of the reach of children or third parties when not in use. The original packaging should be kept and used for storage or shipping in order to protect the rollator from environmental influences.

9. Disposal

Dispose of the materials separately according to type and in the designated collection systems. Please dispose of hazardous waste in the designated collection systems.

10. List of materials and technical data

10.1 Technical data

Carbon Rollator (CR)	CR54	CR62	CR62 XL
Recommended for body height	145 to 170 cm	160 to 200 cm	160 to 200 cm
Maximum user weight	150 kg	150 kg	180 kg
Total weight without accessories	5.3 kg	5.5 kg	5.6 kg
Bag weight	0.5 kg	0.5 kg	0.5 kg
Maximum payload of the bag	10 kg	10 kg	10 kg
Length	655 mm	655 mm	655 mm
Width (open)	610 mm	610 mm	680 mm
Width (closed)	210 mm	210 mm	210 mm
Seat width (between the push handles)	460 mm	460 mm	520 mm
Lowest height	760 mm	810 mm	810 mm
Handle height (adjustable)	760–880 mm	810–1030 mm	810–1000 mm
Seat height	540 mm	620 mm	620 mm
Brakes	2x handbrake with parking function	2x handbrake with parking function	2x handbrake with parking function
Individual friction brake	2x friction brake on the rear wheels	2x friction brake on the rear wheels	2x friction brake on the rear wheels
Wheels (diameter x width)	200 x 32 mm – black, soft polyurethane wheels with stable core	200 x 32 mm – black, soft polyurethane wheels with stable core	200 x 32 mm – black, soft polyurethane wheels with stable core
Turning radius	800 mm	800 mm	800 mm
Reflectors	31 pieces	31 pieces	31 pieces

10.2 Materials used

Frame	Carbon fibre
Folding mechanism	Aluminium 6061-T6 with black powder coating
Base for the handle	Plastic – polyamide 6 with 30% glass fibre (PA 6 GF30)
Handle	Thermoplastic elastomers (TPE) with 20-30% cork
Brake lever	Plastic – polyamide 6 with 30% glass fibre (PA 6 GF30)
Brake pads	Plastic – polyamide 6 with 15% glass fibre (PA 6 GF15)
Front fork	Plastic polypropylene with 15% glass fibre (PP GF15)
Seat net/fabric seat	100% polyester
Rims	Plastic polypropylene with 15% glass fibre (PP GF15)
Wheels	Polyurethane 40 Shore A (wheel surface) + polyurethane 65 Shore A (wheel bead)

11. Warranty and guarantee

11.1 Warranty

If the purchased item is defective, the provisions of statutory liability for defects shall apply. If you have purchased the goods from a specialist retailer, please contact your seller immediately; if you purchased the goods directly from Saljol, please contact Saljol directly on +49 8141 317740. Recognisable defects must be reported in writing within ten days of delivery, concealed defects within ten days of their discovery. Liability will not be accepted after these deadlines. Saljol shall be entitled to decide whether any defect shall be remedied by rectification or subsequent delivery ("subsequent fulfilment"). In all other respects, the provisions of Saljol's Terms and Conditions apply, which are available at <https://www.saljol.de/de/agb/>.

11.2 Guarantee

In addition to the statutory warranty claims and without limiting these, Saljol extends a guarantee of 2 years from the date of purchasing this product. The option is also available for you to extend this guarantee by a further 2 years if you register the rollator online no later than 4 weeks after the date of purchase. You can register at <https://www.saljol.de/de/garantie/>.

Scope of the guarantee

We guarantee that the product will be free from defects in material, workmanship and function within the guarantee period. If a defect occurs during the guarantee period, we will, at our discretion, repair the product free of charge or, if necessary, replace it with a new one. Excluded from the guarantee are wear parts such as handles, seat net, brake mechanism, tyres and the accessories supplied (bag, back strap, bell, LED light, cane holder). These warranty conditions only apply to products purchased in Germany and only if they were purchased directly from Saljol or from an authorised specialist dealer. The seller will provide information on the guarantee conditions for products purchased abroad. Guarantee services and replacement deliveries for guarantee reasons do not extend the warranty period.

Exclusion of the guarantee

The guarantee becomes void if the rollator has been maintained or repaired improperly or by an untrained person or if the product has been operated with non-original accessories. Incorrect assembly or installation due to non-compliance with the Instruction Manual, unintended use or improper use or care of the product will invalidate the guarantee, as will external influences (e.g. transport damage, damage caused by knocks and impacts, weather influences).

12. Note on the obligation to report incidents

Please note that all serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority. Serious incidents within the meaning of the law are all adverse events that have one of the following consequences: Death, life-threatening injury, permanent physical damage and chronic illness.

If you are in the Federal Republic of Germany, please notify:

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If you are in the EU Member States, please notify:

- Your specialist dealer, Saljol sales partner or Saljol GmbH, Frauenstr. 32, D-82216 Maisach; www.saljol.de
- the competent authority for incidents involving medical devices in your country

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Bedienungsanleitung | Instruction manual | Manuel d'utilisation
Istruzioni per l'uso | Gebruiksaanwijzing | Manual de instrucciones

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