Saljol Allround Rollator Superlight

Bedienungsanleitung | Instruction manual | Manuel d'utilisation
Istruzioni per l'uso | Gebruiksaanwijzing | Manual de instrucciones

DE

EN

FR

IT

NL

ES

Vorwort

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www.saljol.de

Introduction

Thank you for purchasing a Saljol Allround Rollator Superlight (AR Superlight) from Saljol! Your stylish new rollator will enable you to lead a mobile and self-determined life. The rollator was developed with a special focus on maximum safety and a particularly lightweight structure. It will reliably assist you in

everyday life thanks to its low weight. We would ask you to read the Instruction Manual carefully before using your AR Superlight for the first time. It contains important safety instructions and useful tips on use and care.

Contents

1. Identification 15

2. General notes..... 15

3. Notes..... 16

 3.1 Safety instructions..... 16

 3.2 Information for safe use of the product 16

4. Scope of delivery and product overview..... 17

 4.1 Scope of delivery..... 17

 4.2 Marking/labelling..... 17

 4.3 Product overview 18

5. Operation 18

 5.1 Unfolding the rollator..... 18

 5.2 Folding the rollator 19

 5.3 Adjusting the correct handle height/support height..... 19

 5.4 Using the handbrake 19

 5.5 Using the parking brake 19

 5.6 Adjusting the brake..... 19

 5.7 Using the tilting aid 20

 5.8 Attaching and removing the bag 20

6. Maintenance and reuse..... 20

 6.1 Maintenance..... 20

 6.2 Maintenance schedule 21

 6.2.1 Daily visual inspection..... 21

 6.2.2 Functional tests..... 21

 6.2.3 Cleaning 21

 6.2.4 Disinfection..... 22

 6.3 Reuse..... 22

7. Transport description..... 22

8. Storage and shipping 22

9. Disposal..... 22

10. Technical data and list of materials..... 23

 10.1 Technical data..... 23

 10.2 Materials used..... 23

11. Warranty and guarantee 24

 11.1 Warranty..... 24

 11.2 Guarantee..... 24

12. Note on the obligation to report incidents..... 24

1. Identification

Manufacturer: Saljol GmbH

Product: Allround Rollator Superlight

Type: Lightweight rollator

Article name: Allround Rollator Superlight

Version: 2026

Manufacturer:

Saljol GmbH, Frauenstraße 32, 82216 Maisach

Phone: +49 8141 317740

Fax: +49 8141 3177411

Mail: hallo@saljol.de, Web: www.saljol.de

Authorised Swiss representative and importer:

CH REP Rehabilitations-Systeme AG

Roggenstrasse 3, CH-4665 Oftringen

Formalities of the Instruction Manual

Version/revision: 2026.02

Creation date: 16/07/2026

Information was current at the time of going to press. We reserve the right to make technical changes. Technical drawings and photos may differ from reality.

©2026 Saljol GmbH: all rights reserved. Reprinting, even in extracts, is only permitted with the authorisation of Saljol GmbH.

2. General notes

Intended use

The Saljol Allround Rollator Superlight was developed for indoors and outdoors and is only suitable for use on firm and even ground. The AR Superlight helps people to stand and walk again. Users can also rest on the integrated seat. Users can perform everyday routine tasks with the Saljol Allround Rollator Superlight, without being dependent on outside assistance. Applications: The AR Superlight from Saljol is designed to cover a wide range of health problems It was conceived specifically for people who require additional support to maintain stability when walking.

Indication

The Allround Rollator Superlight from Saljol is intended exclusively for use as a walking aid and is particularly suitable for patients with limited mobility due to physical impairments, limitations of the active or passive musculoskeletal system and illnesses such as arthritis, osteoporosis, muscle weakness, stroke or Parkinson's disease.

Contraindication

The AR Superlight from Saljol is not suitable for people with very weak arm strength, significant cognitive impairments or very poor balance. The use and possible applications of the Saljol Allround Rollator Superlight should be assessed by a doctor or therapist and instruction must be provided.

Service life

The AR Superlight has an expected service life of up to 4 years if used as intended, regular care and maintenance is provided by a specialist company and it is operated in compliance with the safety instructions. The rollator can remain in use after 4 years if it is still in good condition.

Responsibility

As the manufacturer, we can only consider ourselves responsible for the safety, reliability and suitability for use of the rollator if modifications, extensions, repairs and maintenance work are performed regularly and only by authorised specialist personnel and the rollator is used as intended in accordance with all instructions for use.

Declaration of Conformity

As the manufacturer of the product, Saljol declares that the Allround Rollator Superlight fully complies with the requirements of the Medical Devices Regulation (2017/745) (as of 2022). A detailed CE Declaration of Conformity can be found on our homepage at www.saljol.de.

The manufacturer/authorised representative declares that the following products with the product designation Allround Rollator Superlight, version: 2026

Saljol Allround Rollator Superlight AR54 Superlight	Ice Blue	4058789016848
Saljol Allround Rollator Superlight AR62 Superlight	Ice Blue	4058789016831
Saljol Allround Rollator Superlight AR62 Superlight	Vulcan Grey	4058789017180

comply with the harmonised standards as listed hereafter: EN ISO 11199-2:2025 – Walking aids manipulated by both arms – Requirements and test methods – Part 2: Rollators; seat and handles meet the biological assessment according to ISO 10993-5:2009-10

3. Notes

3.1 Safety instructions

The Instruction Manual contains safety notes that are marked twice, namely by a symbol and by a signal word. The symbol informs you of the type of notice. The signal word explains more details, for example the severity of an impending risk.

Possibly dangerous situation (minor injuries or material damage), commitment to a particular behaviour or activity for the safe use of the rollator

Application tips and particularly useful information

3.2 Information for safe use of the product

To ensure safe use of the product, please keep the following in mind:

- The maximum user weight for the AR62 Superlight must not exceed 150 kg.
- The maximum payload of the mesh pocket is 10 kg.
- It is not permitted use the rollator to move in small, rapid steps.
- The rollator was developed for both indoor and outdoor use.
- Never use in the presence of flammable/combustible and burning substances.

Fingers or other parts of the body of the user and/or a third party may be pinched when folding the rollator or adjusting the handle height. You must therefore remain extremely attentive and careful when adjusting the device. Be aware of gaps in the rollator to avoid trapping your fingers and other body parts.

- Only use the rollator for its defined purpose.
- Only use the rollator if it is in a flawless technical condition.
- It is prohibited to make any constructive changes to the rollator, unless you have the written consent of the manufacturer, Saljol.
- While adjusting the rollator, be careful not to pinch your fingers or other body parts on the moving parts.
- Always ensure that the back strap is fully inserted into the holder at all times.
- Make sure that both parking brakes are engaged when you sit down.
- Note that on sloping terrain, the rolling speed of the AR Superlight may be faster than your walking speed. There is an increased risk of falling.
- Always be careful when crossing thresholds, railway tracks, gully covers with wide slits and similar. A wheel may become caught or trapped in them, stop abruptly and cause a fall.

- All four wheels must be in contact with the ground at all times. This is the only way to balance the rollator safely and prevent accidents.
- Be careful to walk carefully on icy ground and over small obstacles such as branchesso that you can react quickly.
- Always engage both parking brakes when you stop or park the rollator.

4. Scope of delivery and product overview

4.1 Scope of delivery

The features of our products may vary depending on the country and variant in order to meet local requirements. The scope of delivery may therefore differ.

- 1 Saljol Allround Rollator Superlight
- 1 mesh bag with Fidlock magnetic fastening*
- 1 Instruction Manual

4.2 Marking/labelling

The product label is affixed to the inside of the left frame section. The UDI-DI label (Unique Device Identification – Device Identifier) is found on the right and is added to the product marking for compliance with regulatory requirements. They contain important information about the product: EAN code, product name, production date, product colour and weight, product dimensions, maximum user weight, maximum load weight of the bag, seat height, manufacturer information on Saljol, serial number and UDI-DI. The labels must not be removed.

The following symbols are found on the labelling of the AR Superlight and on the packaging – sometimes together with other information – and indicate important features.

EAN REF	Order/item number
Production date	
Colour and total weight	
L/W/H in mm	Length/width/height of the rollator in mm
Maximum user weight	
Maximum payload of the bag	
Seat height	
Observe the Instruction Manual	
MD	Medical device
CE mark	CE mark
Suitable for indoors and outdoors	
Manufacturer of the product	
SN	Serial number
UDI	UDI-DI (Unique Device Identification – Device Identifier)

4.3 Product overview

Saljol Allround Rollator Superlight

- 1 Push handle
- 2 Brake lever
- 3 Bowden cable
- 4 Seat mesh with release strap
- 5 Handle bar
- 6 Hand screw
- 7 Brake pad
- 8 Rear wheel
- 9 Large front wheel
- 10 Locking hook
- 11 Mesh bag
- 12 Tipping aid

5. Operation

Inventory

Remove the rollator out of the box and check that none of the parts are missing. Compare the parts with the illustration in 4.3 and the scope of delivery described in 4.1. Contact your specialist dealer or Saljol immediately if parts are missing or components are damaged.

Important

Before each use, make sure that

- the locking hook is engaged when the rollator is folded,
- the rollator is locked into place when unfolded and cannot simply be folded up,
- the brakes are always engaged (see 5.5) before you sit down or park the rollator,
- the handle bars are tightened with the hand screw,
- the handles are set to the correct height for you and are at the same level,

- all accessories are firmly attached.

A failure to observe these instructions may result in injuries.

5.1 Unfolding the rollator

- Open the locking hook.
- Grasp the rollator by its handles and push or pull them apart at the same time.
- Finally, press both seat tubes down with your hands until you hear a 'click' sound.

5.2 Folding the rollator

- Gently tug the seat's release strap upwards.
- Grasp the handles and push the rollator together until both seat tubes are close together.
- Engage the locking hook to secure the folded rollator in place.

5.3 Adjusting the correct handle height/support height

- Engage the parking brake (see 5.5).
- Then open the hand screws.
- Stand in the rollator and let your arms hang loosely so that they are slightly bent. The height of your wrists corresponds to the correct height of the push handles.
- Unscrew the hand screws and adjust the handles to the same height using the height scale on the inside of the handle bars.
- Then tighten both hand screws by hand.

5.4 Using the handbrake

- Pull the brake levers upwards to engage the brake.
- Keep the brake levers pulled to stand safely in the rollator.
- Simply let go of the brake levers to release the brake.

5.5 Using the parking brake

- To engage the parking brake, press the brake levers downwards until you hear a 'click' sound.
- Simply pull the brake levers upwards again to release the parking brake.

Important

As a rule during use, it is still important that you

- engage the brakes when you are not moving the rollator.

- always engage the parking brakes when you want to park the rollator or sit down.
- use the hand brakes on sloping surfaces to feel safer and minimise the risk of falling.
- sit with your back to the mesh bag, not on top of it.
- sit upright on the seat mesh, while placing your forearms comfortably on the push handles.

Information

Is the use of one of your hands impaired due to a stroke, osteoarthritis or similar? If so, we recommend converting the rollator to a one-hand brake. You can obtain the brake cable and all information on the conversion from us free of charge. Kindly contact your medical supply shop or the Saljol service team by email: kundenservice@saljol.de.



5.6 Adjusting the brake

The brake requires adjustment if the braking force decreases due to wheel wear. The brakes should be adjusted so that both wheels brake evenly. The gap between the brake pad and the wheel surface should be approx. 1 mm.

How to proceed:

Use a size 8 open-end spanner to loosen the lock nut and turn it completely upwards. Attach the ferrule at the same time.

Rotate the ferrule clockwise or anti-clockwise to adjust the braking force. Turn the wheels and pull the brake levers to check the brake function. The wheels should turn freely, but stop immediately when the brakes are applied. It must not be possible to move the wheels when the brake is engaged. Tighten the lock nut firmly again.

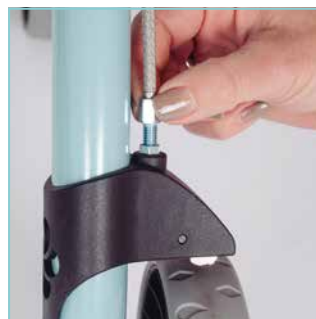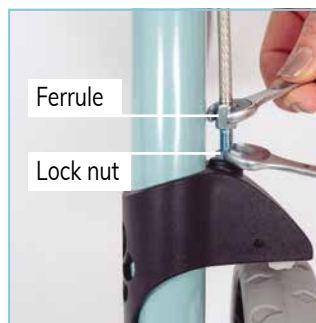Important

The brake has been correctly adjusted if the rear wheels lock when the brake is engaged and do not turn when pushed.

Information

Here you will find a video with a step-by-step guide to adjusting the brake. Simply scan the QR code.

5.7 Using the tipping aid

Step on the tipping aid with one foot and use the handles to pull the rollator slightly towards you. The front wheels of the rollator are now raised. This is how you overcome thresholds and kerbs.


5.8 Attaching and removing the mesh bag

Important

- Stand behind the rollator. Make sure that the rollator is positioned on a level surface and that the parking brakes are engaged.
- You can hang both sides in place at the same time if your stance is secure and stable. Otherwise, support yourself with one hand on the rollator and attach one side first, then the other.

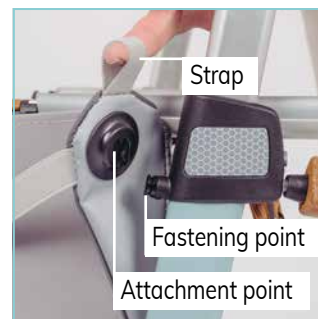Attaching the mesh bag

- Grasp the bag by the two straps.
- Guide the bag under the seat net or the fabric seat.
- Connect the attachment points to the designated fastening points on the frame.
- Apply pressure to the attachment points on the left and right until you hear and feel them click into place.
- Check that both sides are securely engaged.
- Make sure that the mesh bag is hanging freely and that the brake function is not impaired.

Removing the mesh bag

- Grasp the bag by the two straps.
- Pull the bag slightly upwards so that the attachment points are released from the fastening points.
- Do not pull the bag forwards. Doing so may damage the locking mechanism.
- Remove the mesh bag completely from the holders.

6. Maintenance and reuse

6.1 Maintenance

Although the Saljol Allround Rollator Superlight is generally robust and durable thanks to its corrosion-resistant materials, it still requires regular cleaning and maintenance. We recommend that you regularly perform the checks stated in the maintenance schedule to ensure that your rollator can be operated safely at all times.

Inadequate or neglected care and maintenance of the rollator may impair its safety and limit liability. Bear in mind, however, that the maintenance schedule does not contain precise details of the actual scope of work required on the rollator. It is therefore advisable to consider any problems or additional maintenance requirements that you notice on the rollator and to seek professional assistance if necessary. You can ensure that your rollator is always in a flawless condition and provides you with safe support by performing the recommended maintenance regularly and paying attention to any deviations. Saljol reserves the right to limit warranty claims if the rollator has not been regularly maintained.

6.2 Maintenance schedule

6.2.1 Daily visual inspection

A daily visual inspection of the rollator should be performed prior to each use. Doing so helps to check main components (frame, brakes, handles, seat and wheels) for any damage, signs of wear or other problems before they become major issues. Keep an eye on changes or missing elements and, in cases of doubt, ask your specialist dealer, Saljol sales partner or Saljol GmbH.

6.2.2 Functional tests

Functional tests help to ensure that all rollator functions are working properly. Any tested component that is not in good working order must be repaired immediately either by the user or specialised personnel. We recommend performing functional tests on the Allround Rollator at least once a month, ideally even twice. The functional tests include:

- **Brake test:** Check the rollator's brakes to make sure that they are in good working order. Make sure that the rollator can be brought to a standstill quickly and safely on both sides if necessary. The parking brake should lock both rear wheels and prevent the rollator from rolling away on an inclined/uneven surface.
- **Tyre test:** Check the rollator's tyres to make sure that they turn easily and show no signs of excessive wear or damage. Defective tyres must be replaced immediately.
- **Stability test:** Check the rollator's connections to make sure that they move easily and show no signs of wear or damage. Connections that are too loose should be tightened immediately, connections that are too stiff should be loosened a little. Make sure

that all wheels are in contact with the ground at the same time.

- **Folding test:** Perform out the folding process according to the manufacturer's instructions and make sure that all parts are correctly aligned and locked. The entire process should proceed smoothly.
- **Handle test:** Check the height adjustment of the handles to ensure that it is working correctly and that the rollator is set to the correct height for the user. The basic rule: The wrists must be level with the handles when standing upright.
- **Back strap test, if applicable:** Regularly check the back strap and its fastening rail for visible damage such as tears, wear or loose seams. Make sure that the back strap is fully inserted into the holder at all times.

6.2.3 Cleaning

The entire rollator must be cleaned at least once a month, depending on the degree of soiling. Coarse dirt can be removed with warm water and a sponge. A commercially available cleaning agent can be added to the water to deal with stubborn dirt. Disinfectants can be used without any problems. Plastic parts are best cleaned with a commercially available plastics cleaning agent. Please observe the cleaning agent's instructions for use. Simple methods such as a Teflon spray can be used to stop squeaking or creaking of the folding mechanism, wheels or even the front fork.

Please also bear in mind the following information on individual product parts!

Important

- Never use aggressive cleaning agents such as solvents/scouring agents or hard brushes!
- Never clean the rollator with a high-pressure/steam cleaner!

Wheels

Clean the wheels with warm water and a mild solvent. Use a plastic brush if necessary, but never a wire one.

Cork TPE

Dust and other particles can accumulate on the surface of cork TPE products during storage and cause a so-called 'grey haze'. This is not a product defect and can be easily removed.

Correct care and cleaning of cork TPE-coated elements helps to extend the service life of the material.

Remove loose dirt with a soft, dry cloth or a soft brush. Heavier soiling can be removed with a well-wrung, soft cloth and a mild soap solution. Avoid vigorous rubbing and the use of abrasive cleaning agents or scouring pads, as you may otherwise damage the cork TPE. To maintain the service life of the cork TPE, clean the surface regularly and protect it from direct sunlight, heat sources and moisture. Please note that cork is a natural material and can change colour over time when exposed to light and humidity.

6.2.4 Disinfection

Standard household disinfectants can be used to disinfect the rollator. For an overview of the disinfectants approved by the Robert Koch Institute, visit <https://www.rki.de/>. After disinfection, allow the rollator to dry in the air and observe the exposure time specified for the disinfectant used.

6.3 Reuse

The Saljol Allround Rollator is suitable for reuse. The number of possible reuses depends on the condition, material wear and functional suitability of the product. You must clean and disinfect the rollator prior to each reuse. Remove all visible dirt completely and use a suitable disinfectant to ensure sterility. Then perform all the checks described in section 6.2 Maintenance schedule to ensure that the rollator is working properly. The rollator is resistant to the cleaning agents and disinfectants applied for reuse.

7. Transport description

The rollator is easy to fold together and therefore convenient to transport. Please follow the folding instructions (see section 5.2) if you plan to transport the rollator by car, for example. Make sure that the rollator is stowed away safely to prevent sudden and dangerous movements (e.g. braking when driving a car). The Saljol transport bag (available separately) protects your rollator from external influences and damage during transport.

8. Storage and shipping

The rollator must be stored upright, dry and out of the reach of children or third parties when not in use. The original packaging should be kept and used for storage or shipping in order to protect the rollator from environmental influences.

9. Disposal

Dispose of the materials separately according to type and in the designated collection systems. Please dispose of hazardous waste in the designated collection systems.

10. List of materials and technical data

10.1 Technical data

Allround Rollator (AR) Superlight	AR54 Superlight	AR62 Superlight
Recommended for body height	145 to 170 cm	160 to 200 cm
Maximum user weight	150 kg	150 kg
Total weight without accessories	5.2 kg	5.3 kg
Mesh bag weight	0.2 kg	0.2 kg
Maximum payload of the bag	10 kg	10 kg
Length	670 mm	670 mm
Width (open)	610 mm	610 mm
Width (closed)	210 mm	210 mm
Seat width (between the push handles)	460 mm	460 mm
Lowest height	760 mm	810 mm
Handle height (adjustable)	760–880 mm	810–1030 mm
Seat height	540 mm	620 mm
Brakes	2x handbrake with parking function	2x handbrake with parking function
Front wheels (diameter x width)	250 x 32 mm – grey, soft rubber wheels	250 x 32 mm – grey, soft rubber wheels
Rear wheels (diameter x width)	200 x 32 mm – grey, soft rubber wheels	200 x 32 mm – grey, soft rubber wheels
Turning radius	810 mm	810 mm
Reflectors	10 pieces	10 pieces

10.2 Materials used

Frame	Carbon fibre
Folding mechanism	Aluminium 6061-T6 with black powder coating
Base for the handle	Plastic – polyamide 6 with 30% glass fibre (PA 6 GF30)
Handle	Thermoplastic elastomers (TPE) with 20-30% cork
Brake lever	Plastic – polyamide 6 with 30% glass fibre (PA 6 GF30)
Brake pads	Plastic – polyamide 6 with 15% glass fibre (PA 6 GF15)
Front fork	Plastic polypropylene with 15% glass fibre (PP GF15)
Seat mesh	100% polyester
Rims	Plastic – polyamide 6 with 15% glass fibre (PA 6 GF15)
Wheels	Thermoplastic rubber 55 Shore A

11. Warranty and guarantee

11.1 Warranty

If the purchased item is defective, the provisions of statutory liability for defects shall apply. If you have purchased the goods from a specialist retailer, please contact your seller immediately; if you purchased the goods directly from Saljol, please contact Saljol directly on +49 8141 317740. Recognisable defects must be reported in writing within ten days of delivery, concealed defects within ten days of their discovery. Liability will not be accepted after these deadlines. Saljol shall be entitled to decide whether any defect shall be remedied by rectification or subsequent delivery ("subsequent fulfilment"). In all other respects, the provisions of Saljol's Terms and Conditions apply, which are available at <https://www.saljol.de/de/agb/>.

11.2 Guarantee

In addition to the statutory warranty claims and without limiting these, Saljol extends a guarantee of 2 years from the date of purchasing this product. The option is also available for you to extend this guarantee by a further 2 years if you register the rollator online no later than 4 weeks after the date of purchase. You can register at <https://www.saljol.de/de/garantie/>.

Scope of the guarantee

We guarantee that the product will be free from defects in material, workmanship and function within the guarantee period. If a defect occurs during the guarantee period, we will, at our discretion, repair the product free of charge or, if necessary, replace it with a new one. Excluded from the guarantee are wear parts such as handles, seat net, brake mechanism, tyres and the accessories supplied (mesh bag). These guarantee conditions only apply to products purchased in Germany and only if they were purchased directly from Saljol or from an authorised specialist dealer. The seller will provide information on the guarantee conditions for products purchased abroad. Guarantee services and replacement deliveries for guarantee reasons do not extend the warranty period.

Exclusion of the guarantee

The guarantee becomes void if the rollator has been maintained or repaired improperly or by an untrained person or if the product has been operated with non-original accessories. Incorrect assembly or installation due to non-compliance with the Instruction Manual, unintended use or improper use or care of the product will invalidate the guarantee, as will external influences (e.g. transport damage, damage caused by knocks and impacts, weather influences).

12. Note on the obligation to report incidents

Please note that all serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority. Serious incidents within the meaning of the law are all adverse events that have one of the following consequences: Death, life-threatening injury, permanent physical damage and chronic illness.

If you are in the Federal Republic of Germany, please notify:

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If you are in the EU Member States, please notify:

- Your specialist dealer, Saljol sales partner or Saljol GmbH, Frauenstr. 32, D-82216 Maisach; www.saljol.de
- the competent authority for incidents involving medical devices in your country

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