Saljol Duschstuhl

Bedienungsanleitung | Instruction manual | Manuel d'utilisation
Istruzioni per l'uso | Gebruiksaanwijzing | Manual de instrucciones

DE

EN

FR

IT

NL

ES

Vorwort

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Introduction

Thank you for purchasing a shower chair from Saljol! The swivelling shower chair helps you to shower safely, comfortably and without outside assistance. The Saljol Shower Chair was developed with safety in mind and will provide valuable assistance in your everyday life.

We would ask you to read the Instruction Manual carefully before using your shower chair for the first time. It contains important safety instructions and useful tips on use and care.

EN

Contents

1.	Identification	12
2.	General notes.....	12
3.	Notes.....	13
3.1	Safety instructions.....	13
3.2	Information for safe use of the product	13
4.	Scope of delivery and product overview.....	13
4.1	Scope of delivery.....	13
4.2	Marking.....	14
4.3	Product overview	14
5.	Operation	14
5.1	Inventory	14
5.2	Fitting the EPP arm supports.....	15
5.3	Fitting the EPP backrest	15
5.4	Positioning the shower chair	15
6.	Maintenance	16
7.	Cleaning and disinfection.....	16
8.	Storage and shipping	16
9.	Disposal.....	16
10.	Technical data and list of materials.....	17
10.1	Technical data.....	17
10.2	Materials used.....	17
11.	Warranty and guarantee	17
11.1	Warranty.....	17
11.2	Guarantee.....	17
12.	Note on the obligation to report incidents	18

1. Identification

Manufacturer: Saljol GmbH
Product: Shower chair
Type: Shower chair
Article name: Saljol Shower Chair
Version: 2026

Manufacturer:
Saljol GmbH, Frauenstraße 32, 82216 Maisach
Phone: +49 8141 317740
Fax: +49 8141 3177411
Mail: hallo@saljol.de, Web: www.saljol.de

Authorised Swiss representative and importer:

CH REP Rehabilitations-Systeme AG
Roggenstrasse 3, CH-4665 Oftringen

Formalities of the Instruction Manual

Version/revision: 2026.01

Creation date: 08/06/2026

Information was current at the time of going to press. We reserve the right to make technical changes. Technical drawings and photos may differ from reality.

©2026 Saljol GmbH: all rights reserved. Reprinting, including extracts, is only permitted with the authorisation of Saljol GmbH.

2. General notes

Intended use

The Saljol Shower Chair is used in domestic and inpatient environments, primarily in shower/washrooms. It can be used by people with restricted mobility as a shower aid with five wheels, for example for independent showering/washing and sitting. The shower chair is only suitable for indoor use and a maximum user load of 150 kg.

Indication

The Saljol Shower Chair is intended exclusively as a shower aid and for use in the bathroom for people with the following restrictions:

- instability and/or inability to stand
- diseases that impair stability
- as a post-surgery aid

No other use is permitted.

Contraindication

The Saljol Shower Chair is not suitable for persons with severe balance problems, weak muscle tone, impaired perception, inability to sit and a user weight greater than 150 kg. The use and possible applications of the Saljol Shower Chair should be assessed by a doctor or therapist and instruction must be provided.

Service life

The shower chair has an expected service life of up to 4 years if used as intended, regular care and maintenance is provided and it is operated in compliance with the safety instructions. The shower chair can remain in use if it is still in good condition after 4 years.

Responsibility

As the manufacturer, we can only consider ourselves responsible for the safety, reliability and suitability for use of the shower chair if modifications, extensions and repairs are performed regularly and only by authorised specialist personnel and the shower chair is used as intended in accordance with all instructions for use.

Declaration of Conformity

As the manufacturer of the product, Saljol declares that the shower chair fully complies with the requirements of the Medical Devices Regulation (2017/745) (as of 2022). A detailed CE Declaration of Conformity can be found on our homepage at www.saljol.de.

The manufacturer/authorised representative declares that the following products with the product designation Saljol Shower Chair version: 2026

Saljol Shower Chair DS50	Diamond Silver	4058789016879
--------------------------	----------------	---------------

comply with the harmonised standards as listed hereafter: ISO 17966:2016-01 Assistive products for personal hygiene that support users – Requirements and test methods ISO 10993-5 Biological evaluation of medical devices

3. Notes

3.1 Safety instructions

The Instruction Manual contains safety notes that are marked twice, namely by a symbol and by a signal word. The symbol informs you of the type of notice. The signal word explains more details, for example the severity of an impending risk.

Caution/important

Potentially dangerous situation (minor injuries or material damage), commitment to a particular behaviour or activity for safe use of the shower chair

Note

Application tips and particularly useful information

3.2 Information for safe use of the product

To ensure safe use of the product, please keep the following in mind:

Restrictions on use

- The maximum user weight must not exceed 150 kg.
- The shower chair may only be used indoors and on a level, solid surface.
- The shower chair must be positioned securely on all five wheels at all times.
- A shift in balance due to body movements can increase the risk of the shower chair tipping over.
- Never use in the presence of flammable/combustible and burning substances.

Caution

Toes, feet or other parts of the body of the user and/or a third party may be pinched when turning the shower chair or adjusting the hygiene opening. You must therefore remain extremely attentive and careful when using the device. Be aware of the wheel movement while seated to protect yourself from pinching your toes and other body parts.

Accident prevention

- Only use the shower chair for its defined purpose.
- Only use the shower chair if it is in a flawless technical condition.
- It is prohibited to make any constructive changes to the shower chair, unless you have the written consent of the manufacturer, Saljol.
- All five wheels must be in contact with the ground at all times. This is the only way to balance the shower chair safely and prevent accidents.
- Ensure that you are seated securely on the shower chair when moving (shifting your balance).
- Avoid jerky turning movements on floor surfaces with joints.
- Bath additives or detergents can change the ground surface and therefore affect the slip resistance.
- Please hold on to the side handles if you feel slightly dizzy when seated.

4. Scope of delivery and product overview


4.1 Scope of delivery

- 1 Saljol Shower Chair with pre-assembled armrests
- 1 Backrest including screws and curved washers
- 2 EPP arm supports
- Screws for attaching the arm supports and backrest
- 1 hexagon Allen key
- 1 Instruction Manual

4.2 Marking

The product label is affixed to the underside of the frame. The UDI-DI label (Unique Device Identification – Device Identifier) is also found on the underside of the frame and is added to the product marking for compliance with regulatory requirements. They contain important information about the product: Product name, serial number, maximum user weight, product dimensions, production date, manufacturer information on Saljol, EAN code and UDI-DI. The labels must not be removed.

The following symbols are found on the labelling of the shower chair and on the packaging – sometimes together with other information – and indicate important features.

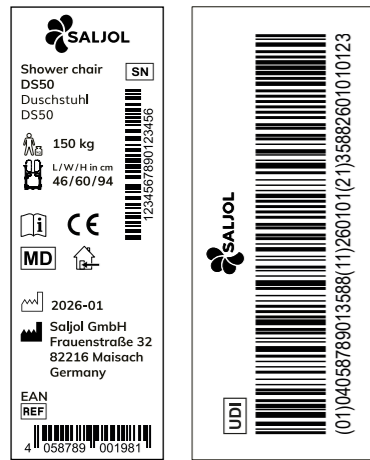	Serial number
	Maximum user weight
	Length/width/height of the shower chair in cm
	Observe the Instruction Manual
	CE mark
	Medical device
	Suitable for indoor use
	Production date
	Manufacturer of the product
	Order/item number
	UDI-DI (Unique Device Identification – Device Identifier)

4.3 Product overview

Saljol Shower Chair

- 1 Heat-reflecting backrest
- 2 Heat-reflecting armrest with brake function
- 3 Heat-reflecting seat surface
- 4 Handles
- 5 Adjustable hygiene opening
- 6 Aluminium frame for maximum stability
- 7 Non-slip wheels

5. Operation

5.1 Inventory

Remove the shower chair from the box and check that none of the parts are missing. Compare the parts with the illustration in 4.3 and the scope of delivery described in 4.1. Contact your specialist dealer or Saljol immediately if parts are missing or components are damaged.


When sitting down and standing up, please ensure that your feet are always in firm contact with the floor and are positioned slightly turned outwards in front of the opening of the lower frame.

When turning, please ensure that the movement is performed slowly with continuous muscle tension.

5.2 Fitting the EPP arm supports



- Place the EPP arm supports on the top of the armrests.
- Align the threads in the EPP arm supports with the holes in the armrests.
- Insert the supplied screws from below through the holes in the armrests.
- Tighten the screws evenly using the hexagon Allen key provided.
- Then check the tight fit of the EPP arm supports.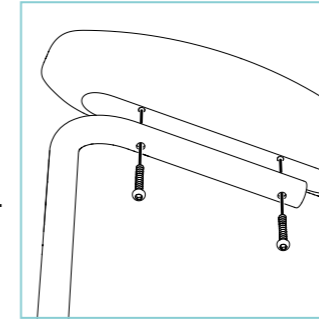

Only tighten the screws hand-tight to avoid damaging the EPP arm supports.

5.3 Fitting the EPP backrest

- Position the backrest at the fixing points provided on the underside of the shower chair frame.
- Align the holes in the backrest with the holes in the frame.
- Insert the supplied screws together with the curved washers through the holes.
- Use the hexagon Allen key provided to tighten the screws evenly.
- Then check the tight fit of the EPP backrest.

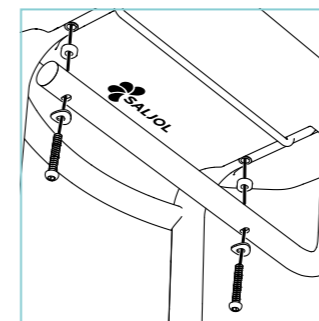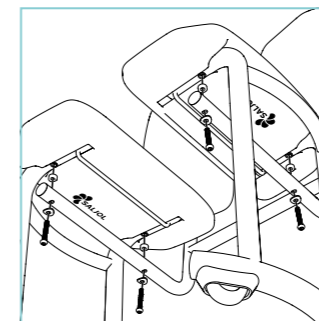
Only tighten the screws hand-tight to avoid damaging the material.

5.4 Positioning the shower chair

Before first use, check that the brake function of the armrests is working properly. When leaning on the armrests, the turning movement of the shower chair must be blocked.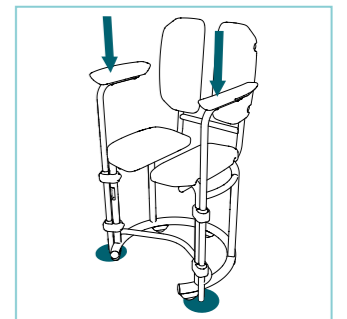

Using the shower chair


- Place the Saljol Shower Chair on a level, non-slip surface. Sit down for an initial seating test.
- Make sure you are sitting upright and in the centre. Avoid leaning forwards or backwards too much and ensure that you have sufficient freedom of movement.
- Turn the shower chair slowly with your body. Keep both feet on the floor.
- If supported by a carer, hold on to the handles. The carer can turn you carefully.

Brake function when leaning on the armrests

The turning movement of the shower chair is automatically blocked when leaning on the armrests. This ensures a secure hold when sitting down, standing up and moving to a different seat.

Turning on the spot


- The wheels allow the shower chair to be turned on the spot and are fitted with non-slip rubber coating. This makes it easier to reach different areas of the body without having to change your sitting position.
- You can either place the feet on the frame or leave them on the floor for improved stability.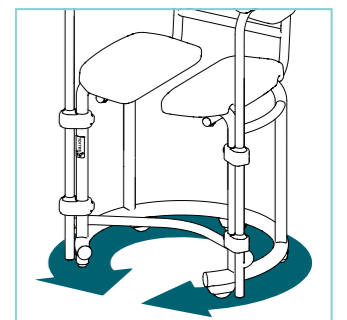

Adjusting the seat width and backrest

The seat and backrest can be adjusted without tools.

- To adjust the seat surface, place the shower chair on its side on a soft surface (e.g. carpet) to avoid scratches. The easiest way to adjust the backrest is to place the chair upright.
- Move the seat or back surfaces to the preferred position.
- The widths 1, 2 and 3 engage noticeably. Make sure that both sides are set to the same level.

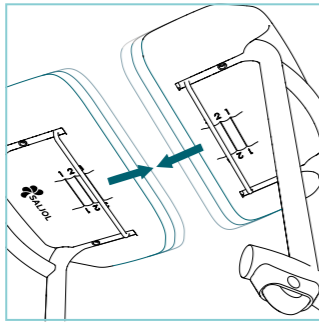6. Maintenance

The shower chair is maintenance-free. In the event that faults or defects occur during use, return the shower chair immediately to your specialist dealer or to Saljol for repair.

7. Cleaning and disinfection

Important

- Do not use any aggressive cleaning agents such as solvents or abrasive cleaners or hard brushes.
- Do not use a high-pressure cleaner or steam cleaner to clean the shower chair.
- When disinfecting, follow the instructions for use and processing provided by the manufacturer of the cleaning agent and disinfectant.
- After cleaning, rinse the shower chair with clear, warm water and dry it carefully to extend its service life.
- After use, thoroughly remove residues of personal care products such as liquid soap, shampoo or shower gel with clear, warm water.

Frame

Clean the frame with warm water and a sponge. A commercially available cleaning agent can be used in the event of heavy soiling. Disinfectants can also be used.

Wheels

Clean the wheels with warm water and a mild cleaning agent. Use a plastic brush if required. Do not use a wire brush.

Seat surface, arm supports and the backrest

Clean the seat surfaces, arm supports and backrest with warm water and a sponge. A commercially available cleaning agent can be used in the event of heavy soiling. If necessary, use a suitable disinfectant to disinfect the surfaces. A list of authorised disinfectants is found on the website of the Robert Koch Institute at <http://www.rki.de>. The seat surfaces, arm supports and backrest are dishwasher-safe up to a maximum of 85°C.

8. Storage and shipping

Keep the shower chair upright in a dry place when not in use. Store it out of the reach of children and unauthorised persons. If possible, use the original packaging for storage or shipping. Therefore, keep the original packaging in a safe place to protect the shower chair from environmental influences when necessary.

9. Disposal

Dispose of the materials separately in the designated collection systems according to the substances contained. Dispose of hazardous waste at the designated collection points.

10. Technical data and list of materials

10.1 Technical data

Shower chair (DS)	DS50
Recommended for body height	150–190 cm
Maximum user weight	150 kg
Seat height	50 cm
Hygiene opening, level 1/2/3	6/9/12 cm
Seat width	36/39/42 cm
Seat depth	37 cm
Backrest length	37 cm
Backrest width	33/36/39 cm
Space required for turning circle (product width)	60 cm
Total weight without accessories	5.5 kg

10.2 Materials used

Frame	Aluminium
Seat, armrests and backrest	Expanded polypropylene
Connecting elements	Polyamide 6
Cover caps	Polyamide 6
Wheels	Thermoplastic elastomers (TPE)

11. Warranty and guarantee

11.1 Warranty

If the purchased item is defective, the provisions of statutory liability for defects shall apply. If you have purchased the goods from a specialist retailer, please contact your seller immediately; if you purchased the goods directly from Saljol, please contact Saljol directly on +49 8141 317740. Recognisable defects must be reported in writing within ten days of delivery, concealed defects within ten days of their discovery. Liability will not be accepted after these deadlines. Saljol shall be entitled to decide whether any defect

shall be remedied by rectification or subsequent delivery ("subsequent fulfilment"). In all other respects, the provisions of Saljol's Terms and Conditions apply, which are available at <https://www.saljol.de/de/agb/>.

11.2 Guarantee

In addition to the statutory warranty claims and without limiting these, Saljol extends a guarantee of 2 years from the date of purchasing this product. The option is also available for you to extend this guarantee by a further 2 years if you register the shower chair online no later than 4 weeks after the date of purchase. You can register at <https://www.saljol.de/de/garantie/>.

Scope of the guarantee

We guarantee that the product will be free from defects in material, workmanship and function within the guarantee period. If a defect occurs during the guarantee period, we will, at our discretion, repair the product free of charge or, if necessary, replace it with a new one. Wear parts such as the seat surface, backrest and wheels are excluded from the warranty. These guarantee conditions only apply to products purchased in Germany and only if they were purchased directly from Saljol or from an authorised specialist dealer. The seller will provide information on the guarantee conditions for products purchased abroad. Guarantee services and replacement deliveries for guarantee reasons do not extend the warranty period.

Exclusion of the guarantee

The guarantee becomes void if the shower chair has been maintained or repaired improperly or by an untrained person or if the product has been operated with non-original accessories. Incorrect assembly or installation due to non-compliance with the Instruction Manual, unintended use or improper use or care of the product will invalidate the guarantee, as will external influences (e.g. transport damage, damage caused by knocks and impacts, weather influences).

12. Note on the obligation to report incidents

Please note that all serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority. Serious incidents within the meaning of the law are all adverse events that have one of the following consequences: Death, life-threatening injury, permanent physical damage and chronic illness.

If you are in the Federal Republic of Germany, please notify:

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If you are in the EU Member States, please notify:

- Your specialist dealer, Saljol sales partner or Saljol GmbH, Frauenstr. 32, D-82216 Maisach; www.saljol.de
- the competent authority for incidents involving medical devices in your country

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