Saljol Duschhocker

Bedienungsanleitung | Instruction manual | Manuel d'utilisation
Istruzioni per l'uso | Gebruiksaanwijzing | Manual de instrucciones

DE

EN

FR

IT

NL

ES

Vorwort

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Introduction

Thank you for purchasing a shower stool from Saljol! The swivelling shower stool will help you to shower safely, comfortably and without outside assistance. The Saljol Shower Stool was developed with safety in mind and will provide valuable assistance in your everyday life.

We would ask you to read the Instruction Manual carefully before using your shower stool for the first time. It contains important safety instructions and useful tips on use and care.

Contents

1.	Identification	11
2.	General notes.....	11
3.	Notes.....	12
	3.1 Safety instructions.....	12
	3.2 Information for safe use of the product	12
4.	Scope of delivery and product overview.....	12
	4.1 Scope of delivery.....	12
	4.2 Marking.....	13
	4.3 Product overview	13
5.	Operation	13
	5.1 Inventory	13
	5.2 Positioning the shower stool.....	14
6.	Maintenance	14
7.	Cleaning and disinfection.....	14
8.	Storage and shipping	14
9.	Disposal.....	15
10.	Technical data and list of materials.....	15
	10.1 Technical data.....	15
	10.2 Materials used.....	15
11.	Warranty and guarantee	16
	11.1 Warranty.....	16
	11.2 Guarantee.....	16
12.	Note on the obligation to report incidents.....	16

1. Identification

Manufacturer: Saljol GmbH
 Product: Shower Stool
 Type: Shower stool
 Article name: Saljol Shower Stool
 Version: 2026

Manufacturer:
 Saljol GmbH, Frauenstraße 32, 82216 Maisach
 Phone: +49 8141 317740
 Fax: +49 8141 3177411
 Mail: hallo@saljol.de, web: www.saljol.de

Authorised Swiss representative and importer:

CH	REP	Rehabilitations-Systeme AG Roggenstrasse 3, CH-4665 Oftringen
----	-----	--

Formalities of the Instruction Manual

Version/revision: 2026.01
 Creation date: 08/06/2026

Information was current at the time of going to press. We reserve the right to make technical changes. Technical drawings and photos may differ from reality.

©2026 Saljol GmbH: all rights reserved. Reprinting, including extracts, is only permitted with the authorisation of Saljol GmbH.

2. General notes

Intended use

The Saljol Shower Stool is used in domestic and inpatient environments, primarily in shower/washrooms. It can be used by people with restricted mobility as a shower aid with five wheels, for example for independent showering/washing and sitting. The shower stool is only suitable for indoor use and a maximum user load of 150 kg.

Indication

The Saljol Shower Stool is intended exclusively as a shower aid and for use in the bathroom for people with the following restrictions:

- instability and/or inability to stand
- diseases that impair stability
- as a post-surgery aid

No other use is permitted.

Contraindication

The Saljol Shower Stool is not suitable for persons with severe balance problems, weak muscle tone, impaired perception, inability to sit and a user weight greater than 150 kg. The use and possible applications of the Saljol Shower Stool should be assessed by a doctor or therapist and instruction must be provided.

Service life

The shower stool has an expected service life of up to 4 years if used as intended, regular care and maintenance is provided and it is operated in compliance with the safety instructions. The shower stool can remain in use if it is still in good condition after 4 years.

Responsibility

As the manufacturer, we can only consider ourselves responsible for the safety, reliability and suitability for use of the shower stool if modifications, extensions and repairs are performed regularly and only by authorised specialist personnel and the shower stool is used as intended in accordance with all instructions for use.

Declaration of Conformity

As the manufacturer of the product, Saljol declares that the shower stool fully complies with the requirements of the Medical Devices Regulation (2017/745) (as of 2022). A detailed CE Declaration of Conformity can be found on our homepage at www.saljol.de.

The manufacturer/authorised representative declares that the following products with the product designation Saljol Shower Stool version: 2026

Saljol Shower Stool DH50	Diamond Silver	4058789016886
--------------------------	----------------	---------------

comply with the harmonised standards as listed hereafter: ISO 17966:2016-01 Assistive products for personal hygiene that support users – Requirements and test methods ISO 10993-5 Biological evaluation of medical devices

3. Notes

3.1 Safety instructions

The Instruction Manual contains safety notes that are marked twice, namely by a symbol and by a signal word. The symbol informs you of the type of notice. The signal word explains more details, for example the severity of an impending risk.

Caution/important

Potentially dangerous situation (minor injuries or material damage), commitment to a particular behaviour or activity for safe use of the shower stool

Note

Application tips and particularly useful information

3.2 Information for safe use of the product

To ensure safe use of the product, please keep the following in mind:

Restrictions on use

- The maximum user weight must not exceed 150 kg.
- The shower stool may only be used indoors and on a level, solid surface.
- The shower stool must be positioned securely on all five wheels at all times.
- A shift in balance due to body movements can increase the risk of the shower stool tipping over.
- Never use in the presence of flammable/combustible and burning substances.

Caution

Toes, feet or other parts of the body of the user and/or a third party may be pinched when turning the shower stool or adjusting the hygiene opening. You must therefore remain extremely attentive and careful when using the device. Be aware of the wheel movement while seated to protect yourself from pinching your toes and other body parts.

Accident prevention

- Only use the shower stool for its defined purpose.
- Only use the shower stool if it is in a flawless technical condition.
- It is prohibited to make any constructive changes to the shower stool, unless you have the written consent of the manufacturer, Saljol.
- All five wheels must be in contact with the ground at all times. This is the only way to balance the shower stool safely and prevent accidents.
- Ensure that you are seated securely on the shower stool when moving (shifting your balance).
- Avoid jerky turning movements on floor surfaces with joints.
- Bath additives or detergents can change the ground surface and therefore affect the slip resistance.
- Please hold on to the side handles if you feel slightly dizzy when seated.

4. Scope of delivery and product overview


4.1 Scope of delivery

- 1 Saljol Shower Stool
- 1 Instruction Manual

4.2 Marking

The product label is affixed to the underside of the frame. The UDI-DI label (Unique Device Identification – Device Identifier) is also found on the underside of the frame and is added to the product marking for compliance with regulatory requirements. They contain important information about the product: Product name, serial number, maximum user weight, product dimensions, production date, manufacturer information on Saljol, EAN code and UDI-DI. The labels must not be removed.

The following symbols are found on the labelling of the shower stool and on the packaging – sometimes together with other information – and indicate important features.

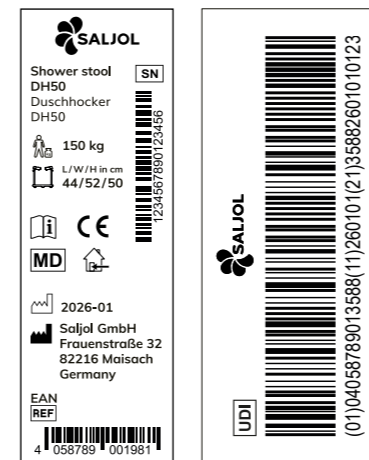	Serial number
	Maximum user weight
	Length/width/height of the shower stool in cm
	Observe the Instruction Manual
	CE mark
	Medical device
	Suitable for indoor use
	Production date
	Manufacturer of the product
	Order/item number
	UDI-DI (Unique Device Identification – Device Identifier)

4.3 Product overview

Saljol Shower Stool

- 1 Heat-reflecting seat surface
- 2 Handles
- 3 Adjustable hygiene opening
- 4 Aluminium frame for maximum stability
- 5 Non-slip wheels

5. Operation

5.1 Inventory

Remove the shower stool from the box and check that none of the parts are missing. Compare the parts with the illustration in 4.3 and the scope of delivery described in 4.1. Contact your specialist dealer or Saljol immediately if parts are missing or components are damaged.

Important

When sitting down and standing up, please ensure that your feet are always in firm contact with the floor and, depending on the sitting direction, are positioned slightly turned outwards in front of the opening of the lower frame or in front of the lower frame. When turning, please ensure that the movement is performed slowly with continuous muscle tension.


5.2 Positioning the shower stool

Using the shower stool

- Place the Saljol Shower Stool on a level, non-slip surface. Sit down for an initial seating test.
- Make sure you are sitting upright and in the centre. Avoid leaning forwards or backwards too much and ensure that you have sufficient freedom of movement.
- Turn the shower stool slowly with your body. Keep both feet on the floor.
- If supported by a carer, hold on to the handles. The carer can turn you carefully.

Turning on the spot


- The wheels allow the shower stool to be turned on the spot and are fitted with non-slip rubber coating. This makes it easier to reach different areas of the body without having to change your sitting position.
- You can either place the feet on the frame or leave them on the floor for improved stability.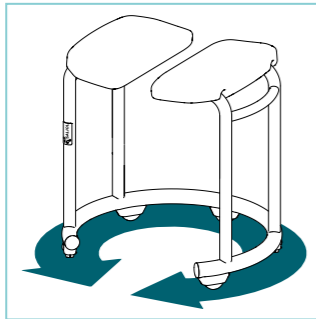

Adjusting the seat width

The hygiene opening between the seats can be adjusted without tools.

- To do so, place the shower stool on its side on a soft surface (e.g. carpet) to avoid scratches. Move the seat surfaces to the preferred position.
- The seat widths 1, 2 and 3 engage noticeably. Make sure that both sides are set to the same level.

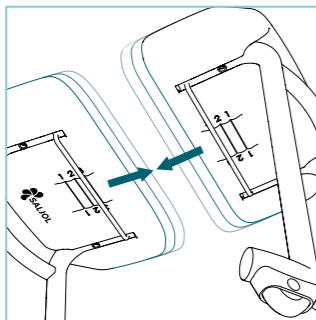6. Maintenance

The shower stool is maintenance-free. In the event that faults or defects occur during use, return the shower stool immediately to your specialist dealer or to Saljol for repair.

7. Cleaning and disinfection

Important

- Do not use any aggressive cleaning agents such as solvents or abrasive cleaners or hard brushes.
- Do not use a high-pressure cleaner or steam cleaner to clean the shower stool.
- When disinfecting, follow the instructions for use and processing provided by the manufacturer of the cleaning agent and disinfectant.
- After cleaning, rinse the shower stool with clear, warm water and dry it carefully to extend its service life.
- After use, thoroughly remove residues of personal care products such as liquid soap, shampoo or shower gel with clear, warm water.

Frame

Clean the frame with warm water and a sponge. A commercially available cleaning agent can be used in the event of heavy soiling. Disinfectants can also be used.

Wheels

Clean the wheels with warm water and a mild cleaning agent. Use a plastic brush if required. Do not use a wire brush.

Seat surface

Clean the seat surfaces with warm water and a sponge. A commercially available cleaning agent can be used in the event of heavy soiling. If necessary, use a suitable disinfectant to disinfect the seat surfaces. A list of authorised disinfectants is found on the website of the Robert Koch Institute at <http://www.rki.de>. The seats are dishwasher-safe up to a maximum of 85°C.

8. Storage and shipping

Keep the shower stool upright in a dry place when not in use. Store it out of the reach of children and unauthorised persons. If possible, use the original packaging for storage or shipping. Therefore, keep the original packaging in a safe place to protect the shower stool from environmental influences when necessary.

9. Disposal

Dispose of the materials separately in the designated collection systems according to the substances

contained. Dispose of hazardous waste at the designated collection points.

10. Technical data and list of materials

10.1 Technical data

Shower stool (DH)	DH50
Recommended for body height	150–190 cm
Maximum user weight	150 kg
Seat height	50 cm
Hygiene opening, level 1/2/3	6/9/12 cm
Seat width	36/39/42 cm
Seat depth	37 cm
Space required for turning circle (product width)	52 cm
Total weight without accessories	2.8 kg

10.2 Materials used

Frame	Aluminium
Seat	Expanded polypropylene
Cover caps	Polyamide 6
Wheels	Thermoplastic elastomers (TPE)

11. Warranty and guarantee

11.1 Warranty

If the purchased item is defective, the provisions of statutory liability for defects shall apply. If you have purchased the goods from a specialist retailer, please contact your seller immediately; if you purchased the goods directly from Saljol, please contact Saljol directly on +49 8141 317740. Recognisable defects must be reported in writing within ten days of delivery, concealed defects within ten days of their discovery. Liability will not be accepted after these deadlines. Saljol shall be entitled to decide whether any defect shall be remedied by rectification or subsequent delivery ("subsequent fulfilment"). In all other respects, the provisions of Saljol's Terms and Conditions apply, which are available at <https://www.saljol.de/de/agb/>.

11.2 Guarantee

In addition to the statutory warranty claims and without limiting these, Saljol extends a guarantee of 2 years from the date of purchasing this product. The option is also available for you to extend this guarantee by a further 2 years if you register the shower stool online no later than 4 weeks after the date of purchase. You can register at <https://www.saljol.de/de/garantie/>.

Scope of the guarantee

We guarantee that the product will be free from defects in material, workmanship and function within the guarantee period. If a defect occurs during the guarantee period, we will, at our discretion, repair the product free of charge or, if necessary, replace it with a new one. Wear parts such as the seat surface and wheels are excluded from the warranty.

These guarantee conditions only apply to products purchased in Germany and only if they were purchased directly from Saljol or from an authorised specialist dealer. The seller will provide information on the guarantee conditions for products purchased abroad. Guarantee services and replacement deliveries for guarantee reasons do not extend the warranty period.

Exclusion of the guarantee

The guarantee becomes void if the shower stool has been maintained or repaired improperly or by an untrained person or if the product has been operated with non-original accessories. Incorrect assembly or installation due to non-compliance with the Instruction Manual, unintended use or improper use or care of the product will invalidate the guarantee, as will external influences (e.g. transport damage, damage caused by knocks and impacts, weather influences).

12. Note on the obligation to report incidents

Please note that all serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority. Serious incidents within the meaning of the law are all adverse events that have one of the following consequences: Death, life-threatening injury, permanent physical damage and chronic illness.

If you are in the Federal Republic of Germany, please notify:

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If you are in the EU Member States, please notify:

- Your specialist dealer, Saljol sales partner or Saljol GmbH, Frauenstr. 32, D-82216 Maisach; www.saljol.de
- the competent authority for incidents involving medical devices in your country

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