# Saljol Carbon Rollator

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## Instruction manual

### Preface

Thank you for purchasing a Saljol Carbon Rollator! With your new rollator, you will be able to lead a mobile and self-determined life in style. The Carbon Rollator has been developed with a lot of attention to safety and will support you in your everyday life.

Please read the instruction manual thoroughly before using your Carbon Rollator for the first time. It contains important safety information, useful tips for use and care, as well as a list of recommended accessories.

## Content

1. Identification.....	23
2. General notes.....	24
3. Notes.....	25
3.1 Safety instructions.....	25
3.2 Instructions for the safe use of the product.....	25
4. Scope of delivery and product overview.....	26
4.1 Scope of delivery.....	26
4.2 Marking/Label.....	27
4.3 Product overview.....	28
5. Operation.....	29
5.1 Unfolding the rollator.....	29
5.2 Folding the rollator.....	30
5.3 Adjusting the correct handle height/support height.....	30
5.4 Use of the handbrake.....	31
5.5 Use of the parking brake.....	31
5.6 Brake adjustment.....	32
5.7 Using the tipping aid.....	32
5.8 Using the grinding brake.....	33
6. Maintenance.....	33
6.1 Cleaning.....	33
6.2 Disinfection.....	34
6.3 Inspection.....	34
7. Transport description.....	34
8. Storage and shipping.....	34
9. Disposal.....	34
10. Technical data.....	35
11. Warranty and guarantee.....	36
11.1 Warranty.....	36
11.2 Guarantee.....	36
12. Note on the obligation to report incidents.....	37
13. Accessories.....	37
14. Test protocol for reuse.....	40

## 1. Identification

Manufacturer: Saljol GmbH  
Product: Carbon Rollator  
Type: Lightweight rollator  
Article name: Carbon Rollator  
Version: 2022

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Swiss representative and importeur:

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Formalities of the operating instructions

Version/Revision: 2022.10  
Creation date: 26.10.2022

Information is correct at the time of going to press.

We reserve the right to make technical changes. Technical drawings and photos may differ from reality.

©2022 Saljol GmbH: all rights reserved. Reprinting, including excerpts, only with the permission of Saljol GmbH

## 2. General Notes

### Purpose

The Carbon Rollator from Saljol was developed for indoor and outdoor use. It serves as a mobility walking aid with four wheels, for example for standing, walking and sitting independently.

### Indication

With the help of the Carbon Rollator from Saljol, people can stand again, walk and rest on the integrated seat. The Saljol Carbon Rollator allows users to perform everyday routine tasks without having to rely on the help of third parties. Note that on sloping terrain, the rolling speed of the Saljol Carbon Rollator may be higher than your walking speed. Increased risk of falling! Please also pay attention to the use of the grinding brake.

Applications: Saljol's Carbon Rollator is specifically designed for people with a wide range of health problems who require additional stability support during walking.

### Contraindication

Carbon Rollator from Saljol is not suitable for people with very weak arm power, significant cognitive impairment or poor balance. The use of the Saljol Carbon Rollator should be evaluated and instructed by a physician or therapist.

### Service life

When used as intended, regular care and maintenance by a specialized company and compliance with the safety instructions, the expected service life of the Carbon Rollator is up to 4 years. If after 4 years the rollator is still in proper condition, it can be used again.

### Responsibility

As the manufacturer, we can only consider ourselves responsible for the safety, reliability and usability of the rollator if changes, expansions, repairs and maintenance work are carried out regularly and only by authorized specialists and if the rollator is used properly in compliance with all instructions for use.

### Declaration of conformity

Saljol, as the manufacturer of the product, declares that the Carbon Rollator fully complies with the requirements of Medical Device Regulation (2017/745) (status 2022). A CE Declaration of Conformity can be found on our homepage [www.saljol.de](http://www.saljol.de).

The manufacturer declares that the following products are approved with Product designation CARBON Rollator, Version 2021

Saljol Carbon Rollator CR62, British Racing Green	4058789013960
Saljol Carbon Rollator CR62, Midnight Blue	4058789013977
Saljol Carbon Rollator CR62, Star Silver	4058789013984
Saljol Carbon Rollator CR62, Matt Black	4058789013991
Saljol Carbon Rollator CR54, British Racing Green	4058789014004
Saljol Carbon Rollator CR54, Midnight Blue	4058789014011
Saljol Carbon Rollator CR54, Star Silver	4058789014028
Saljol Carbon Rollator CR54, Matt Black	4058789014035

comply with the following harmonised standards: ISO 11199-2:2005 Walking aids manipulated by both arms – Requirements and test methods – Part 2: Rollators; Seat and handles meets the biological assessment according to ISO 10993-5: 2009-10

## 3. Notes

### 3.1 Safety instructions

The instruction manual contains safety notes that are marked twice: by a symbol and by a signal word. The symbol informs you about the type of notice. The signal word explains more details, for example the severity of an impending risk.

#### Caution/Important

Possibly dangerous situation (minor injuries or material damage), commitment to a particular behaviour or activity for the safe use of the rollator

#### Note

Application tips and useful information

### 3.2 Instructions for the safe use of the product

For safe use of the product, please keep the following in mind:

#### Restrictions on use

- The maximum user weight must not exceed 150 kg.
- The bag may be loaded with a maximum of 5 kg.
- It is not permitted to move yourself around with your feet (tripping) while being seated on the rollator.
- The rollator is designed for both indoor and outdoor use.
- Never use in the presence of flammable/combustible and burning substances.

### Accident prevention

- Only use the rollator for the defined purpose.
- Only use the rollator if it is in perfect technical condition.
- It is prohibited to make any constructive changes to the rollator, unless you have the written consent of the manufacturer, Saljol.
- While adjusting the rollator, be careful not to pinch your fingers or other body parts on the moving parts.
- Make sure that both parking brakes are activated when you sit down on the rollator.
- All four wheels must be in contact with the ground at all times. Only in this way the rollator is safely balanced which prevents accidents.
- Always be careful when crossing thresholds, railway tracks, gully covers with wide slits and the like. A wheel can get caught or trapped in them, stop abruptly and you might fall.
- Be careful when walking on icy ground and small obstacles such as branches, walk carefully so that you can react quickly.
- Always apply both parking brakes when you stop or park the rollator.

### Caution

Fingers or other parts of the body of the user and/or a third party could be pinched when folding the rollator or adjusting the handle height. Therefore, be extremely attentive and careful when adjusting the device. Be aware of gaps in the rollator to protect yourself from trapping your fingers and other body parts.

## 4. Scope of delivery and product overview

### 4.1 Scope of delivery


- 1x Saljol Carbon Rollator
- 1x bag
- 1x cane holder (mounted)
- 1x instruction manual

### 4.2 Marking/Label

The product label is on the inside of the left frame part. It contains important information about the product: EAN code, product name, production date, product colour and weight, product dimensions, maximum user weight, maximum payload weight of the bag, seat height, manufacturer information of Saljol and the serial number. The label must not be removed.

The following symbols can be found, partly in combination with further information, on the marking of the Carbon Rollator and on the packaging to identify important properties.

- Order number/item number
- Production date
- Product colour and total weight
- Length/width/height of the rollator in cm
- Maximum user weight
- Maximum payload weight of the bag
- Seat height
- Observe instruction manual
- Medical device
- CE marking
- Product suitable for indoors and outdoors
- Manufacturer of the product
- Serial number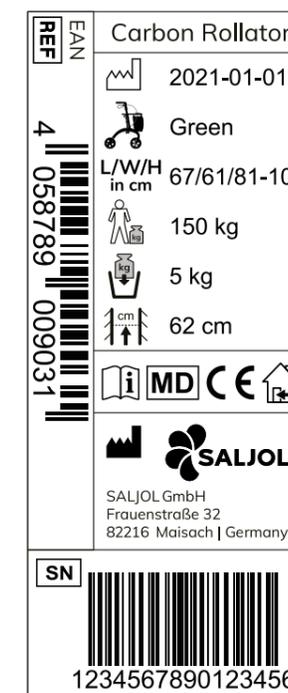

### 4.3 Product overview

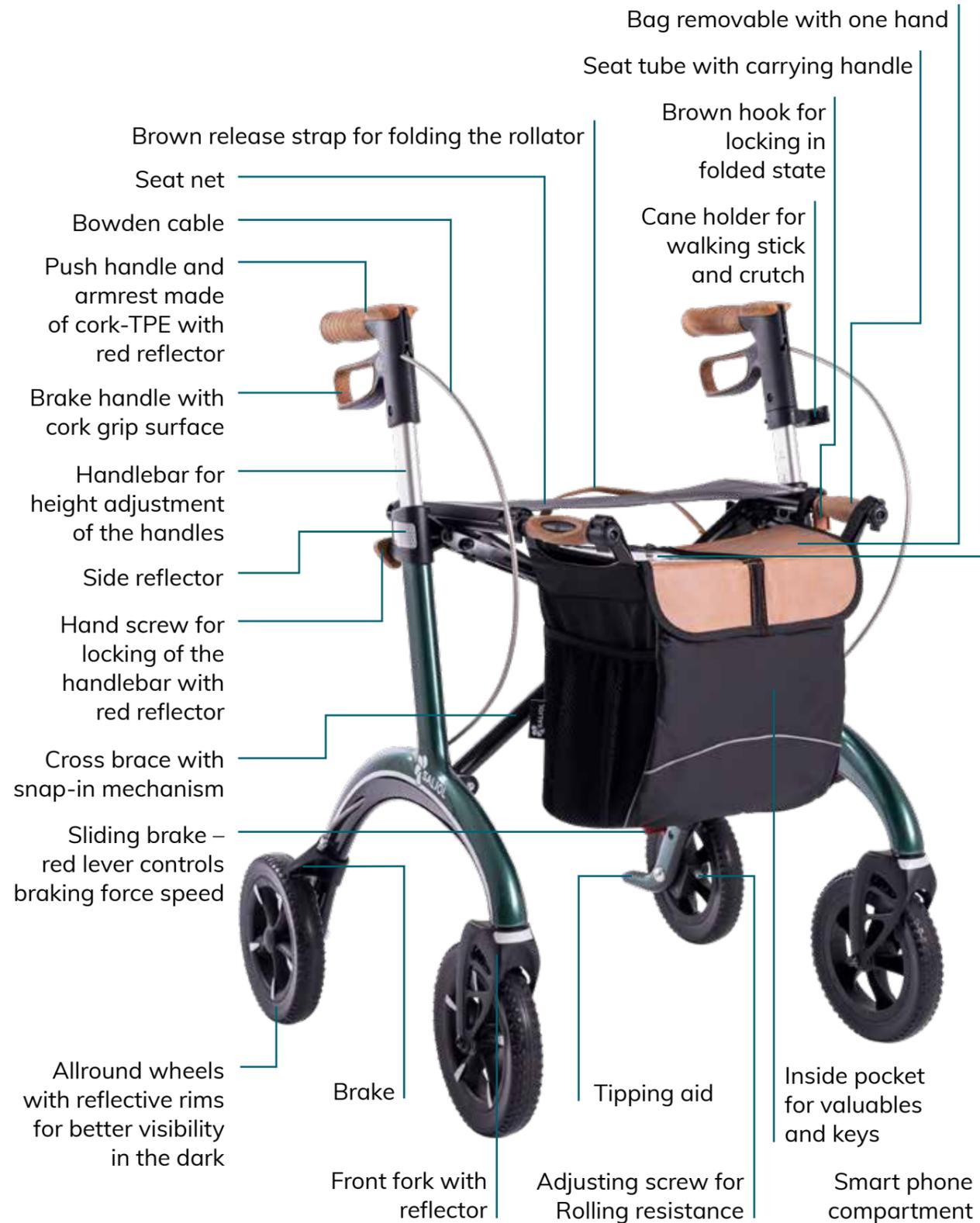
Figure 1: The Saljol Carbon Rollator

## 5. Operation

### Inventory

Take the Saljol Carbon Rollator out of the box and check if anything is missing. Compare with figure 1 on page 28 and with the scope of delivery, described in 4.1 on page 26. If any part is missing or damaged, contact your dealer or Saljol immediately.

### 5.1 Unfolding the rollator

Open the brown hook to unlock the folding mechanism.

Hold both handles and push or pull them apart in parallel.

Finally, push both seat tubes down with your hands until you hear a „click“ sound.

### Important

Now test whether you can pull the seat tubes upwards. If this does not work, the rollator is correctly and safely unfolded. If in doubt, compare with figure 1 on page 28. After unfolding the rollator, please make sure that it is securely closed with an audible click of the lock and that the seat is well tensioned. Make sure that both push handles are securely inserted in the frame and firmly fixed with the hand screw. Always test the secure fit of all accessories BEFORE use. Failure to observe these warnings may result in injury.

## 5.2 Folding the rollator

Pull the brown release strap on the seat upwards with a slight tug to fold the rollator.

Push the rollator together by the handles until both seat tubes are close together.

Secure the folded rollator with the brown hook.

## 5.3 Adjusting the correct handle height/support height

Open the hand screws on the handlebars. On the inside of the support bars you will find a height scale that makes it easier to adjust the left and right handlebars to the same height.

You can find the correct grip height by standing in the rollator, letting your arms dangle loosely and bending them slightly. The height of the ball of the thumb now corresponds to the correct height of the push handles.

Tighten both hand screws hand-tight.  
Tip: Activate the parking brake so that the rollator does not roll away when you adjust it.

### Important

Adjust both push handles to the same height to achieve a symmetrical posture when walking. Always press the parking brake (see 5.5) when you want to leave the rollator, stop or sit down.

## 5.4 Use of the handbrake

To brake the rollator, simply pull the brake handles upwards towards the push handles. To stand safely in the rollator, keep the brake handles pulled.

To release the brake and continue walking, simply release the brake handles.

## 5.5 Use of the parking brake

To apply the parking brake, press the brake handles down until you hear a „click“ sound.

To release the parking brake, pull the brake handles upwards.

### Important

- Use the brakes when you are not moving the rollator.
- Always apply the parking brake when you park the rollator or want to sit down.
- It is recommended to use the brakes on inclined grounds to get a safer feeling and minimize the risk of falling.
- Sit with your back to the shopping bag, not on the bag.
- Sit upright on the seat net while resting your forearms comfortably on the push handles and armrests.

## 5.6 Brake adjustment

If the braking force decreases due to wear of the wheels, the brake must be adjusted. The brake should be adjusted so that both wheels brake evenly. The distance between the brake pad and the wheel surface should be approx 2-3.5 mm.

Loosen the lock nut with a size 8 open-end spanner and turn it completely upwards. Fix the adjustment screw at the same time.

Adjust the braking force by turning the adjustment screw inwards or outwards. Check the brake function by turning the wheels and operating the brake handles. The wheels should turn freely but stop immediately when braking. It should not be possible to move the wheels when the brake is locked. Tighten the lock nut tightly again.

### Caution

The brake is correctly adjusted if, when the brake is applied, the rear wheels lock and do not turn when pushed.

### Info

Here you can find a video, using the example of our Allround Rollator, in which the brake adjustment is explained: <https://www.youtube.com/watch?v=-k-Ru8oiwFM> or scan the QR code.

## 5.7 Using the tipping aid

The tipping aid is located on the rear wheels. Step on the tilt lever with one foot and pull the rollator towards you gently by the handles. The walker will now tilt. This will help you overcome thresholds and curbs.

## 5.8 Using the grinding brake

The grinding brake is recommended for slight slopes or for people with severely limited walking speed. Press the red lever to activate the grinding brake. You can adjust the rolling resistance at the adjusting screw.

## 6. Maintenance

### 6.1 Cleaning

#### Frame

Cleaning can be done with warm water and a sponge. In case of stubborn soiling, a commercial cleaning agent can be added to the water. Disinfectants can be used without any problems. Plastic parts are best cleaned with a commercially available plastic cleaner. Please follow the instructions for use of the cleaning agent used. To stop squeaking or creaking of the cross struts, simple means such as Teflon spray can be used.

#### Important

Never use aggressive cleaning agents such as solvents/abrasives or hard brushes.

#### Wheels

Clean the wheels with warm water and a mild solvent. If necessary, use a plastic brush, never a wire brush. To stop the squeaking or creaking of the wheels, or the front fork, simple means such as Teflon spray can be used.

#### Important

Never clean the rollator with a high-pressure/steam cleaner!

## 6.2 Disinfection

The Carbon Rollator can be disinfected with standard household disinfectants. The Robert Koch list of approved disinfectants can be found at <http://www.rki.de>. Allow the rollator to air dry and observe the exposure time of the disinfectant you are using.

## 6.3 Inspection

We recommend that the Carbon Rollator is inspected by the specialist dealer/Saljol at least once a year, ideally even twice. If faults or defects occur during use, the Carbon Rollator must be returned immediately to the specialist dealer/Saljol for repair. Saljol reserves the right to limit the warranty claims if the rollator has not been serviced regularly.

## 7. Transport description

The rollator is easy to fold and thus convenient to store for transport. If it is necessary to transport it, e.g. in the car, please follow the instructions for folding. Make sure to store the rollator in a safe way to secure it against sudden and dangerous movements (e.g. braking when driving a car).

The Saljol transport bag as an accessory protects your rollator from external influences and damage during transport.

## 8. Storage and shipping

When not in use, keep the rollator upright, in a dry place, out of reach of children or third parties. If the rollator is to be stored or shipped, it is advisable to keep the original packaging and store it for these purposes so that it is available when needed. Then your rollator is optimally protected against environmental influences during storage or transport.

## 9. Disposal

Dispose materials separately by type in the designated collection systems. Please dispose hazardous waste in the designated collection systems.

## 10. Technical data

Carbon Rollator	CR54	CR62
Max. user weight	150 kg	150 kg
Total weight without bag	5,6 kg	5,6 kg
Weight bag	0,4 kg	0,4 kg
Max. bag load	5 kg	5 kg
Length	67 cm	67 cm
Width (open)	61 cm	61 cm
Width (closed)	21 cm	21 cm
Seat width (between push handles)	45 cm	45 cm
Lowest height (push handles retracted)	76 cm	81 cm
Handle height (adjustable)	76–88 cm	81–103 cm
Seat height	54 cm	62 cm
Brakes	2x handbrake with parking function	2x handbrake with parking function
Individual grinding brake	2x grinding brake on the rear wheels	2x grinding brake on the rear wheels
Wheels (diameter x width)	200 x 32 mm – black, soft polyurethane wheels with stable core	200 x 32 mm – black, soft polyurethane wheels with stable core
Turning radius	118 cm	118 cm
Reflectors	31 pieces	31 pieces

## 11. Warranty and guarantee

### 11.1 Warranty

If the purchased item is defective, the provisions of the statutory liability for defects shall apply.

Insofar as you have purchased the goods from a specialist retailer, please contact your seller immediately; if you have purchased directly from Saljol, please contact Saljol directly, Tel. +49 (0)8141 317740. Visible defects must be reported in writing within ten days of delivery, hidden defects within ten days of their discovery. The above-mentioned deadlines are preclusive deadlines. Saljol shall be entitled to decide whether a defect, if any, shall be remedied by rectification or subsequent delivery („subsequent performance“). In all other respects, the provisions in the General Terms and Conditions of Saljol, available at <https://www.saljol.de/en/data-protection/>, shall apply.

### 11.2 Guarantee

In addition to the statutory guarantee claims and without limiting them, Saljol gives a guarantee of 2 years from the date of purchase on this product.

You also have the option to extend this guarantee by a further 2 years if you register the rollator online no later than 4 weeks after the date of purchase. You can register at [www.saljol.de/en/extended-warranty/](http://www.saljol.de/en/extended-warranty/)

#### Scope of the guarantee

We guarantee that the product will be free from defects in material, workmanship and function within the guarantee period. If a defect occurs during the guarantee period, we will, at our own discretion, either repair the product free of charge or, if necessary, replace it with a new one. Excluded from the guarantee are parts subject to wear and tear such as brake mechanism, grinding brake, tyres, seat net, push handles and bag. These guarantee conditions only apply to products purchased in Germany, provided they were purchased from Saljol directly or from an authorised specialist dealer.

The seller will provide information on the guarantee conditions for products purchased abroad. Guarantee services and replacement deliveries for guarantee reasons do not result in an extension of the guarantee period.

#### Exclusion of the guarantee

The right to claim under guarantee expires if the rollator has been improperly maintained or repaired by an untrained person or if the product has been operated with non-original accessories. Incorrect set-up or installation due to non-observance of the operating instructions, use contrary to the intended purpose or improper use or care of the product will result in a loss of guarantee, as will external influences (e.g. transport damage, damage due to impacts and blows, weather influences).

## 12. Note on the obligation to report incidents

Please note that all serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority. Serious incidents in the sense of the law are all adverse events that have one of the following consequences: Death, life-threatening injury, permanent bodily damage and chronic illness.

#### In the Federal Republic of Germany, please inform:

- your local distributor, Saljol sales partner or Saljol GmbH, Frauenstr. 32, D-82216 Maisach; [www.saljol.de](http://www.saljol.de)
- the Federal Institute for Drugs and Medical Devices (BfArM), Kurt-Georg-Kiesinger-Allee 3, 53175 Bonn; [www.bfarm.de](http://www.bfarm.de)

#### In EU Member States, please inform:


- your local distributor, SALJOL sales partner or Saljol GmbH, Frauenstr. 32, D-82216 Maisach; [www.saljol.de](http://www.saljol.de)
- the authority responsible for incidents involving medical devices in your country

## 13. Accessories

#### Back belt

Thanks to the back belt, you can sit comfortably on the rollator and lean against it. It is adjustable and therefore also suitable for tall people. The back belt offers you more comfort when sitting in the rollator.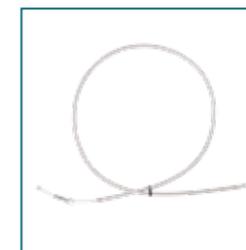

#### Long brake cable

##### for conversion to one-hand simultaneous brake

A huge advantage for those who no longer have full strength in both hands.

You can find a video on the conversion of our Allround Rollator under: <https://www.youtube.com/watch?v=MGKBGVUJyml> or scan the QR code:

#### Transport bag

Protect your rollator from external influences and damage when stowing it away. The padded bag can easily be put over the rollator from behind. Thanks to zips at the height of the rollator handles, you can manoeuvre it safely even when folded.

### Designer bag from JOST

The modern designer bag for your rollator in leopard print or black is stable and water-repellent. It can also be used as a hand or shoulder bag and can be loaded with 5 kg.

### Real wood tray

Setting the table will be easy as pie. The high-quality wooden tray for the rollator makes it easy to transport things from A to B.

### Firm seat

The padded seat improves your hold when you sit on the rollator. You sit upright and safely – no matter where.

### Parasol/umbrella

Going out in rainy weather? With the umbrella on the rollator you stay dry and can push safely with both hands. Used as a parasol, it provides shade. Easy to assemble!

### LED lamp

With the bright LED lamp on your rollator, you are clearly visible in the dark. Also suitable for illuminating the footwell – so you can see small obstacles.

### Bell

With the bell on your rollator, you can draw attention to yourself in road traffic. Simply screw it on and ring, just like on a bicycle.

### Bag

The bag, available completely made of black woven fabric or in a combination of black woven fabric and brown faux leather, can be loaded with 5 kg, can be removed with one hand and has a practical magnetic closure.

### Additional cane holder

The cane holder holds your walking aid firmly to the rollator. It can be attached to the right and left of the frame and the height adjustment of the rollator. One cane holder is already pre-mounted on the Carbon Rollator.

### Note

You can order the accessories in our online shop [www.saljol.de](http://www.saljol.de) around the clock or by telephone via our customer service (Mon.-Fri. 9am-4pm) on the number +49 (0)8141 317740 from home. Available accessories may vary depending on the country.